

Reference

Russell L Lundstrom<sup>1</sup>, Tyler Klenow<sup>1</sup>, Arri Morris<sup>1</sup>, Barbara Pobatschnig<sup>2</sup>, Karl D Hibler<sup>3</sup>, and Andreas HJ Kannenberg<sup>1</sup>

# The C-Brace<sup>®</sup> microprocessor-controlled stance and swing orthosis improves safety, mobility, and quality of life at one year: Interim results from a prospective registry

Journal of Rehabilitation and Assistive Technologies Engineering Volume 11: 1–13.  
DOI: 10.1177/20556683241269539 [Open Access](#)


Products

C-Brace

Major Findings

After 1 year following C-Brace fitting compared to baseline condition:

- **36% improvement in walking speed**  
Fast Walking Speed (FWS) improved by  $0.26 \pm 0.33$  m/s ( $p < .0001$ )
- **35% improvement in functional ability and reduced fall risk**  
Timed up and go (TUG) time reduced by  $8.1 \text{ sec} \pm 14.6 \text{ sec}$  ( $p < .0001$ )  
47% fewer patients at risk of falling (TUG time above 13.5-sec cutoff)
- **Improvement in patient-perceived balance confidence and reduced fall risk**  
ABC improved by  $+24.9 \pm 25.8\%$  ( $p < .0001$ )  
46% fewer patients at risk of falling (ABC score below 67% cutoff)
- **91% reduction in actual falls**  
Baseline:  $33 \pm 77$ . C-Brace:  $3.0 \pm 5.6$  ( $p = .0005$ )
- **33% improvement in quality of life**  
Health Utility Index (HUI) from EQ-5D-5L improved by  $+0.20 \pm 0.21$  ( $p < .0001$ )

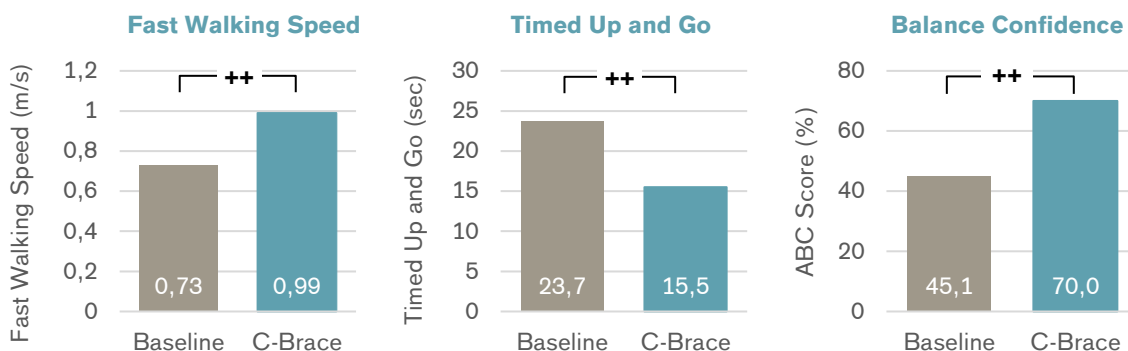

++ statistically significant ( $p < 0.05$ )

Population

Subjects: 46 (17 female) w/ indication for KAFO  
Previous orthosis: SCO: 28%, AFO: 28%, Locked KAFO: 20%, None: 11%, Polycentric or posterior offset KAFO: 7%, Knee orthosis: 4%, Unknown: 2%.


Underlying conditions: Spinal Cord Injury (SCI): 33%, Polio/Post-polio syndrome (PPS): 20%, Trauma: 7%, Iatrogenic: 7%, Muscular Dystrophy (MD): 2%, Traumatic Brain Injury (TBI): 2%, Herniated Disk: 2%, Other (n=1 each): 20%

Mean age: 51.8 ± 16.4 yrs

Involvement: Unilateral: 80%, Bilateral: 20

## Study Design

Prospective, international registry:

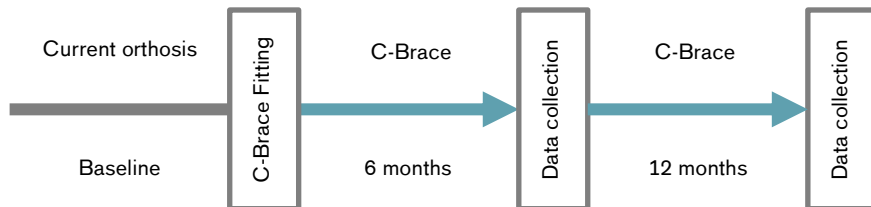

NOTE: The C-Brace Registry includes annual follow ups for 3 years. Primary outcomes were evaluated at 12 months with 6-month data carried forward for missing 12-month follow ups.

## Results

Functions and Activities						Participation	Environment
Biomechanics – Static Measurement	Biomechanics – Gait analysis	X-Rays	EMG	Functional tests	Clinical effects	Satisfaction	Health Economics

Category	Outcomes	Results for C-Brace vs. Baseline	Sig.*
Functional tests	Fast Walking Speed (FWS) measured by the 10-meter Walk Test.	<b>36% improvement in FWS, increasing by +0.26 ± 0.33 m/s, from 0.73 ± 0.34 to 0.99 ± 0.42.</b>	++  n/a
	Timed up and Go (TUG)	<b>35% improvement in TUG, decreasing by 8.1 ± 14.6 s, from 23.7 ± 17.5 to 15.5 ± 10.2.</b>  47% fewer subjects at increased fall risk	++  n/a
Clinical effects	Falls	<b>91% reduction in falls, dropping from 33 ± 77 at baseline to 3.0 ± 5.6 at follow up</b>	++
	Falls (multiple fallers)	<b>90% reduction in falls among those that fell more than once, dropping from 50 ± 90 at baseline to 4.4 ± 6.4 at follow up.</b>	++
	Activity-specific Balance Confidence (ABC)	<b>55% improvement in patient-perceived balance, ABC increasing by +24.9 ± 25.8%, from 45.1 ± 21.1 to 70.0 ± 7.5.</b>  46% fewer subjects at increased fall risk	++  n/a
	PROMIS Pain Interference (PI)	<b>14% reduction in consequences of pain, PI T-score dropping by -8.8 ± 9.7, from</b>	++

Category	Outcomes	Results for C-Brace vs. Baseline	Sig.*
		<b>61.3 ± 12.8 to 55.0 ± 9.5.</b>	
	Numeric Pain Rating Scale (NPS)	<b>23% reduction in average NPS (-1.1) driven by reductions in the affected leg (-1.4), affected ankle (-1.0), sound hip (-1.2), sound ankle (-2) and sound foot (-1.5).</b>	<b>++</b>
	Patient-specific Functional Scale (PSFS)	<b>169% improvement in PSFS score, increasing by +3.6 ± 2.3 points on a 0-10 point scale, from 2.1 ± 1.2 at baseline to 5.7 ± 2.4 at follow up.</b>	<b>++</b>
Health Economics	EQ-5D-5L QoL Health Utility Index (HUI)	<b>33% improvement in HUI, +0.20 ± 0.21, from 0.61 ± 0.24 to 0.82 ± 0.13, driven by improvements in mobility, usual activities and pain dimensions.</b>	<b>++</b>
	EQ-5D-5L QoL Visual Analog Scale (VAS)	<b>Improvement of +12.1 ± 21.4 in VAS score, from 69.1 ± 21.5 to 78.8 ± 14.7.</b>	<b>++</b>

<sup>a</sup> no difference (0), positive trend (+), negative trend (-), significant (++/--), not applicable (n.a.)

<sup>b</sup> p value after post hoc Bonferroni correction did not significance set at p<0.05; trends set at 0.1>p>0.05

effect sizes classified by authors as small (<0.3), moderate (>0.3 and <0.5) or large(>0.5)

### Author's Conclusion

“Several significant and clinically meaningful improvements were observed in a widely diverse sample of individuals with indication for a KAFO after 1-year of real-world use with the C-Brace, a microprocessor stance and swing control orthosis. Among the clinical improvements were increased fast walking speed, walking capability, and patient-perceived balance confidence which translated into a reduction in risk of falling and actual falls. Further improvements included reductions in relevant functional status limitations, pain, and pain interference, and a meaningful improvement in quality of life. The results of this prospective observational registry provide additional evidence that the C-Brace is an effective option to improve safety, mobility, and quality of life for patients needing the support of a KAFO for walking.” (Lundstrom et al., 2024)

### Author's Affiliation

<sup>1</sup>Otto Bock HealthCare LP, Austin, TX, USA.

<sup>2</sup>Ottobock Healthcare Products GmbH, Vienna, AT, USA

<sup>3</sup>Independent Statistician, Bradenton, FL, US.

©2024, Otto Bock HealthCare Products GmbH (“Otto Bock”), All Rights Reserved. This article contains copyrighted material. Wherever possible we give full recognition to the authors. We believe this constitutes a ‘fair use’ of any such copyrighted material according to Title 17 U.S.C. Section 107 of US Copyright Law. If you wish to use copyrighted material from this site for purposes of your own that go beyond ‘fair use’, you must obtain permission from the copyright owner. All trademarks, copyrights, or other intellectual property used or referenced herein are the property of their respective owners. The information presented here is in summary form only and intended to provide broad knowledge of products offered. You should consult your physician before purchasing any product(s). Otto Bock disclaims any liability related from medical decisions made based on this article summary.