

Reference

Fantini Pagani CH, Willwacher S, Kleis B, Brüggemann G-P.

Institute of Biomechanics and Orthopaedics, German Sport University Cologne, Germany.

Influence of a valgus knee brace on muscle activation and co-contraction in patients with medial knee osteoarthritis

J Electromyogr Kinesiol 2013; 23(2):490-500.

Products

Genu Arthro

Major Findings

With Genu Arthro:

→ Muscle activity of rectus femoris (RF), lateral gastrocnemius (GL) and lateral hamstring (LH) decreased

Pre-activation phase:	RF with neutral adjustment:	13.6% decrease
Loading phase:	GL with 4° valgus adjustment:	23.8% decrease
Early stance phase:	GL with neutral adjustment:	16% decrease
	GL with 4° valgus adjustment:	17.7% decrease
Late stance phase:	RF with neutral adjustment:	18.5% decrease
	RF with 4° valgus adjustment:	16.3% decrease
	LH with neutral adjustment:	35.8% decrease
	LH with 4° valgus adjustment:	31.3% decrease


→ Co-contraction ratios of the medial/lateral (M/L), flexor/extensor (F/E) muscle groups decreased

Loading phase:	F/E with 4° valgus adjustment:	15.1% decrease
Late stance:	M/L with 4° valgus adjustment:	12.3% decrease
	F/E with 4° valgus adjustment:	21.5% decrease

→ Co-contraction ratios of the lateral vastus/lateral gastrocnemius (VL/GL), lateral vastus/lateral hamstring (VL/LH) and medial vastus/medial hamstring (VM/MH) muscle pairs decreased


Pre-activation phase:	VL/LH with neutral adjustment:	16.8% decrease
	VL/LH with 4° valgus adjustment:	5.9% decrease
	VM/MH with neutral adjustment:	19.6% decrease
	VM/MH with 4° valgus adjustment:	10.4% decrease
Loading phase:	VL/GL with 4° valgus adjustment:	28.4% decrease

Rectus femoris muscle activity

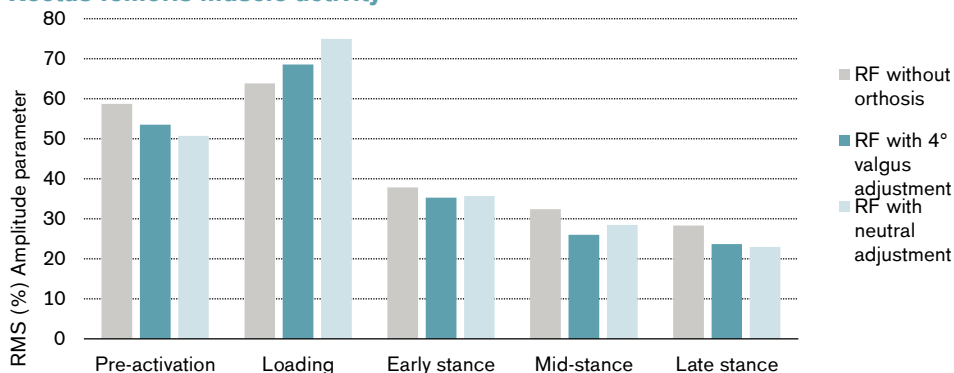

Population	Subjects:	12 patients (7 female, 5 male)
	Mean age:	56.0 ± 4.6 yrs
	Mean body mass:	80.9 ± 13 kg
	Inclusion criteria:	medial knee osteoarthritis from grade II to IV

Study Design Observational, comparative:

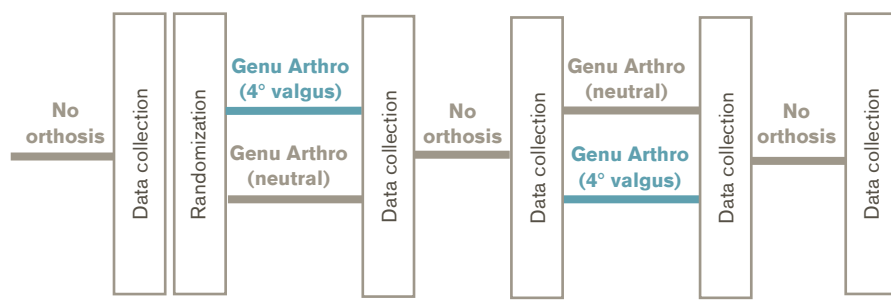

The patients were not informed about the different adjustments of the orthosis.

Results

Functions and Activities						Participation
Biomechanics – Static measures	Biomechanics – Gait analysis	X-Ray	EMG	Functional tests	Clinical effects	Satisfaction

Category	Outcomes	Results for Genu Arthro	Sig.*
EMG	Amplitude parameters – Quadriceps group	<p>In the pre-activation phase of the gait cycle the muscle activity of the rectus femoris decreased by 13.6% with the neutral adjustment.</p> <p>In the late stance phase the muscle activity of the rectus femoris decreased by 16.3% with the 4° valgus and by 18.5% with the neutral adjustment.</p> <p>In all other phases of the gait cycle no significant changes were found for the rectus femoris, vastus lateralis and vastus medialis.</p>	++
	Amplitude parameters – Hamstrings group	<p>The muscle activity of the lateral hamstring was significantly lower with the 4° valgus (31.1%) and the neutral adjustment (35.8%) in the late stance phase.</p> <p>The muscle activity in the other phases of the gait cycle and of the medial hamstring did not differ significantly.</p>	++
	Amplitude parameters – Gastrocnemii group	<p>The muscle activity of the lateral gastrocnemius was significantly lower in the loading phase with the 4° valgus adjustment (23.8%). In the early stance phase the 4° valgus (17.7%) and the neutral adjustment (16%) led to decreased muscle activity of the lateral gastrocnemius.</p>	++

Category	Outcomes	Results for Genu Arthro	Sig.*
		The muscle activity in the other phases of the gait cycle and of the medial gastrocnemius did not differ significantly.	
	Co-contraction ratios <i>Medial/lateral</i>	The co-contraction ratios were 12.3% lower with the 4° valgus adjustment in the late stance phase.	++
		Inter individual differences:	
		4° Valgus vs without orthosis	Neutral vs without orthosis
		83%*: decrease 17%: increase *of patients	50%: decrease 17%: no change 33%: increase
	Co-contraction ratios <i>Flexors/extensors</i>	The co-contraction ratios were 15.1% lower in the loading phase and 21.5% lower in the late stance phase with the 4° valgus adjustment.	++
		Inter individual difference was high.	
	Co-contraction ratios (late stance phase) <i>Muscle pairs</i>	<i>Loading phase:</i> Lateral Vastus / lateral gastrocnemius: 28.4% decrease (4° valgus)	++
		<i>Pre-activation phase:</i> Lateral Vastus / lateral hamstring: 5.9% (4° valgus) and 16.8% (neutral) decrease Medial Vastus / medial hamstring: 10.4% (4° valgus) and 19.6% (neutral) decrease	++
		No significant differences for medial vastus / medial gastrocnemius and medial hamstring / lateral hamstring.	0
Satisfaction	Questionnaire about comfort, acceptance and subjective changes in gait	Perception of comfort during rest or gait did not differ significantly between the 4° valgus and neutral condition nor the perception of changes in gait and acceptance as treatment option.	0

* no difference (0), positive trend (+), negative trend (-), significant (++/--), not applicable (n.a.)

Author's Conclusion

"In summary, significant decreases in muscle activity and co-contraction ratios were observed with the use of the knee brace in both adjustments, indicating a mechanical stabilization of the knee by the brace. The results of our study support the theory of a possible beneficial effect of knee braces in reducing knee loading by decreasing muscle activation and co-contraction levels. This additional mechanism of loading reduction in conjunction with the load reduction induced by the three-point-bending system of valgus braces could further contribute to avoid disease progression in patients with knee osteoarthritis." (Fantini Pagani et al. 2012)

© 2014, Otto Bock HealthCare Products GmbH ("Otto Bock"), All Rights Reserved. This article contains copyrighted material. Wherever possible we give full recognition to the authors. We believe this constitutes a 'fair use' of any such copyrighted material according to Title 17 U.S.C. Section 107 of US Copyright Law. If you wish to use copyrighted material from this site for purposes of your own that go beyond 'fair use', you must obtain permission from the copyright owner. All trademarks, copyrights, or other intellectual property used or referenced herein are the property of their respective owners. The information presented here is in summary form only and intended to provide broad knowledge of products offered. You should consult your physician before purchasing any product(s). Otto Bock disclaims any liability related from medical decisions made based on this article summary.