

Reference

Kahle JT, Highsmith MJ, Hubbard SL.

Westcoast Brace and Limb, Tampa, USA.

Comparison of nonmicroprocessor knee mechanism versus C-Leg on Prosthesis Evaluation Questionnaire, stumbles, falls, walking tests, stair descent, and knee preference

Journal of Rehabilitation Research & Development 2008; 45(1):1–13.

Products

C-Leg vs NMPKs

Major Findings

With C-Leg compared to NMPKs:

→ Falls are reduced by 64% and stumbles are reduced by 59%

→ Satisfaction improved by 20%

→ 74% of subjects preferred using C-Leg over NMPK


→ Improvements of walking velocity by up to 21%

short and long distance

even and uneven terrain

self-selected and fastest possible walking velocity

Improvements of walking velocity with C-Leg

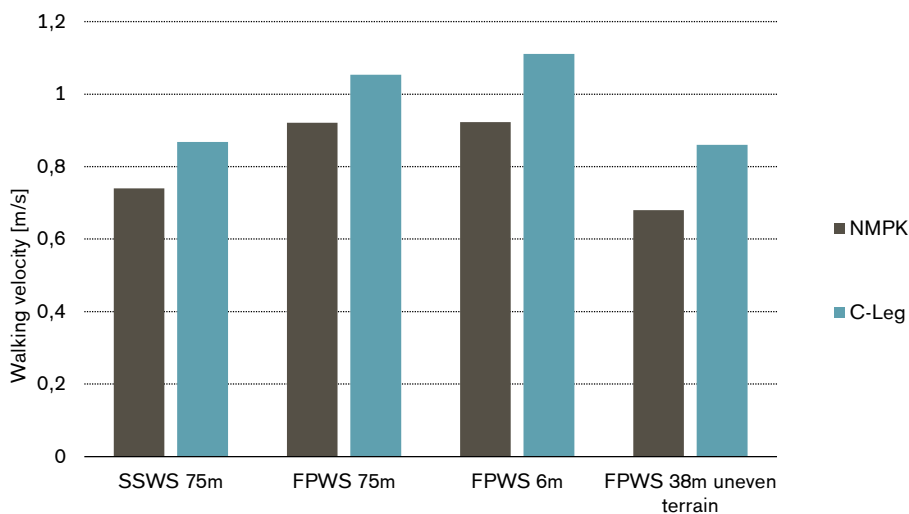

Walking velocity was measured during different walking tasks: self-selected walking speed over 75 meters (SSWS 75m), fastest possible walking speed over 75 meters (FPWS 75m), fastest possible walking speed over 6 meters (FPWS 6) and fastest possible speed over 38 meters on uneven terrain (FPWS 38m uneven terrain).

Population

Subjects: 21 unilateral, transfemoral amputees
Previous prosthesis: NMPKs
Amputation causes: 33% trauma, 19% congenital, 14% diabetes mellitus, 14% PVD, 5% sarcoma
Mean age: 51 yrs (range from 22 – 83 yrs)
Mean time since amputation: 10 yrs (4 – 20 yrs)
MFCL: 43% K2, 38% K3, 10% K4

Study Design

Interventional, pre- to post-test design:



Results

Activities								Participation	Environment
Level walking	Stairs	Ramps, Hills	Uneven ground, Obstacles	Cognitive demand	Metabolic energy consumption	Safety	Activity, Mobility, ADLs	Preference, Satisfaction, QoL	Health economics

Category	Outcomes	Results for C-Leg compared to NMPK	Sig.*
Level Walking	Self-selected (SSWS) and fastest possible walking speed (FPWS)	SSWS increased by 15 % during 75m test.	++
		FPWS increased by 12 % during 75m test.	++
		FPWS increased by 17% during 6m test.	++
Stairs	Stair descent test	63% of subjects improved in stair descent performance.	n.a.
Uneven Ground, Obstacle Course	Fastest possible walking speed (FPWS) on uneven terrain	FPWS increased by 21 % during 38m uneven terrain test.	++
Safety	Reported stumbles and falls	The number of self-reported stumbles decreased by 59%	++
		The number of self-reported falls decreased by 64%.	++
Activity, Mobility, Activities of daily living (ADLs)	Mobility level	42% of subjects improved their mobility level. 53% of subjects stayed at their mobility level. 5% of subjects decreased their mobility level.	n.a.
Preference, Satisfaction, Quality of Life (QoL)	PEQ (Prosthesis Evaluation Questionnaire)	PEQ total score improved by 20%.	++
	Preference	74% of the subjects responded that they would prefer using the C-Leg rather than their previous NMPK.	n.a.

* no difference (0), positive trend (+), negative trend (-), significant (++/--), not applicable (n.a.)

Author's Conclusion

"Using the C-Leg, most subjects demonstrated greater prosthesis function and prosthesis-related quality of life, decreased self-reported falls and stumbles, increased walking speed under several conditions, and improved stair descent. In addition, most subjects preferred the C-Leg." (Kahle et al. 2008)

© 2014, Otto Bock HealthCare Products GmbH ("Otto Bock"), All Rights Reserved. This article contains copyrighted material. Wherever possible we give full recognition to the authors. We believe this constitutes a 'fair use' of any such copyrighted material according to Title 17 U.S.C. Section 107 of US Copyright Law. If you wish to use copyrighted material from this site for purposes of your own that go beyond 'fair use', you must obtain permission from the copyright owner. All trademarks, copyrights, or other intellectual property used or referenced herein are the property of their respective owners. The information presented here is in summary form only and intended to provide broad knowledge of products offered. You should consult your physician before purchasing any product(s). Otto Bock disclaims any liability related from medical decisions made based on this article summary.