
Reference

Wong CK, Wilska J, Stern M.

Program in Physical Therapy, Columbia University Medical Center, New York, USA.

Balance, Balance Confidence, and Falls Using Nonmicroprocessor and Microprocessor Knee Prostheses: A Case Study after Vascular Amputation With 12-Month Follow-Up

Journal of Prosthetics and Orthotics 2012; 24(1):16-18.

Products**C-Leg and NMPKs**

Major Findings

With C-Leg compared to NMPKs:


→ Improved balance and decreased risk of falling

Time required completing the timed up and go (TUG) test decreased by up to 38%

The BERG balance scale score improved by up to 6 points

The activities-specific balance confidence (ABC) scale score improved by 10%

Falls during a 2 months period were reduced from 2 to 0

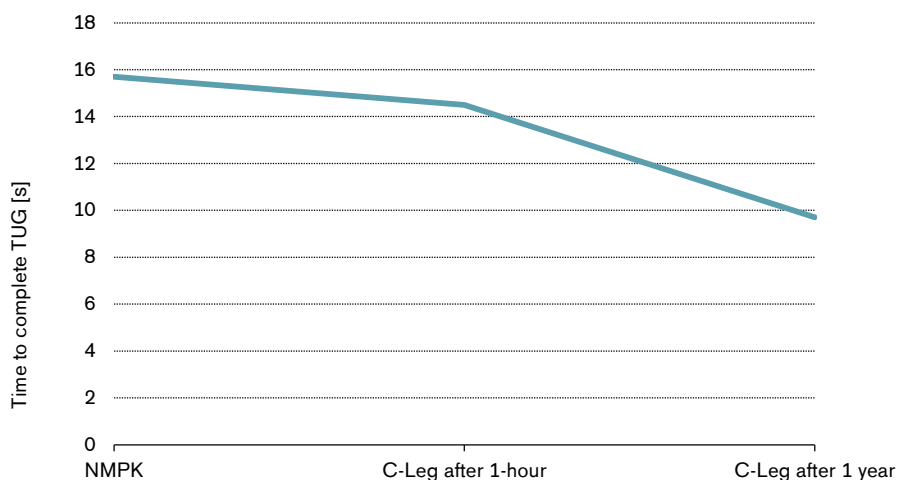→ Stair descent strategy improved from step-to-step to step-over-step**Decreased risk of falling with C-Leg**


The timed up and go (TUG) test is an indicator for risk of falling.

Population

Subjects:	1 unilateral, transfemoral amputee
Previous prosthesis:	NMPK
Amputation causes:	vascular
Mean age:	53 yrs
Mean time since amputation:	1.7 yrs
MFCL:	K3

Study Design

Case Report:

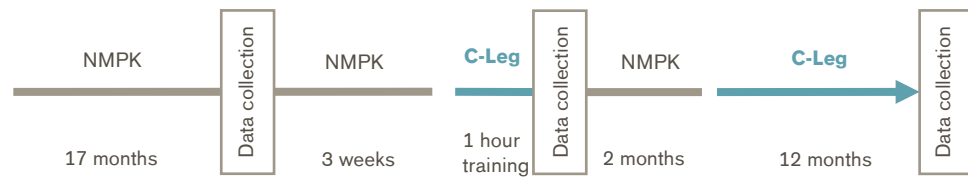

Results

Activities								Participation	Environment
Level walking	Stairs	Ramps, Hills	Uneven ground, Obstacles	Cognitive demand	Metabolic energy consumption	Safety	Activity, Mobility, ADLs	Preference, Satisfaction, QoL	Health economics

Category	Outcomes	Results for C-Leg compared to NMPKs	Sig.*
Stairs	Questionnaire	Stairs descent strategy changed from step-to-step strategy to step-over-step strategy.	n.a.
Safety	Timed up and go (TUG)	The TUG time decreased immediately following the 1-hour training session by 8% (15.7s to 14.5s) and decreased after 1 year by 38% (15.7s to 9.7s).	n.a.
	BERG balance scale	The BERG score increased immediately following the 1-hour training session by 3 points (from 46 to 49) increased after 1 year by 6 points (46 to 52 with a maximum of 56 points), representing clinically meaningful improvement. The increased BERG balance scale score represents a reduced risk of falling.	n.a.
	Activities-Specific Balance Confidence (ABC) Scale	The ABC score increased immediately following the 1-hour training by approximately 10% (from 84 to 92%) and was maintained at 1-year follow-up.	n.a.
	Questionnaire	Reported falls regarding a two months period were reduced from 2 to none.	n.a.
Activity, Mobility, Activities of daily living (ADLs)	Questionnaire	The subject returned to swimming/bowling and he participated in trial walking and jogging with amputee group.	n.a.

* no difference (0), positive trend (+), negative trend (-), significant (++/--), not applicable (n.a.)

Author's Conclusion

"For this man with transfemoral amputation because of vascular disease, balance, balance confidence, falls, and participation in leisure activities all improved during a year of MPK. Future research to identify and reduce risk of falls for people with vascular transfemoral amputation is needed before clinical conclusions can be drawn." (Wong et al. 2012)

© 2014, Otto Bock HealthCare Products GmbH ("Otto Bock"), All Rights Reserved. This article contains copyrighted material. Wherever possible we give full recognition to the authors. We believe this constitutes a 'fair use' of any such copyrighted material according to Title 17 U.S.C. Section 107 of US Copyright Law. If you wish to use copyrighted material from this site for purposes of your own that go beyond 'fair use', you must obtain permission from the copyright owner. All trademarks, copyrights, or other intellectual property used or referenced herein are the property of their respective owners. The information presented here is in summary form only and intended to provide broad knowledge of products offered. You should consult your physician before purchasing any product(s). Otto Bock disclaims any liability related from medical decisions made based on this article summary.