

Reference

Bartsch, Leonie P.¹; Schwarze, Martin¹; Block, Julia¹; Alimusaj, Merkur¹; Schiltewolf, Marcus¹; Jaber, Ayham¹; Wolf, Sebastian I.¹

Varus knee limits pain relief effects of laterally wedged insoles and ankle-foot orthoses in medial knee osteoarthritis

Journal of Rehabilitation Medicine (2022), 54, 1129.

DOI: 10.2340/jrm.v54.1129. [Open Access](#).

Products

Agilium Freestep

Major Findings

With Agilium Freestep, an ankle-foot orthosis (AFO), compared to laterally wedged insoles (LWI):

→ Both aids reduced pain significantly compared to baseline

→ Greater varus malalignment (higher mechanical axis deviation (MAD)) was significantly correlated with less pain reduction for both aids

→ A cut-off value of 14-15mm MAD (i.e. >3° knee varus) was identified to predict pain response

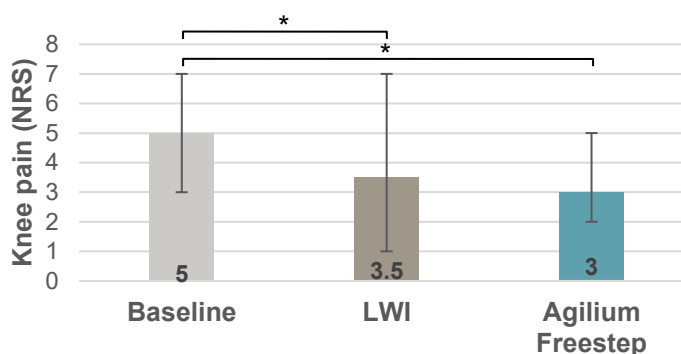


Figure 1: Median (quartiles) of knee pain in the last seven days. NRS: numerical rating scale (0 – no pain, 10 – worst pain)


Population

| | |
|-----------------------|---|
| Subjects: | 28 (16 male, 12 female) |
| Underlying condition: | symptomatic medial knee osteoarthritis stage 1-3* |
| Previous orthosis: | none |
| Mean age: | 58.3 ± 8.0 years |
| Mean height: | 173 ± 10 cm |
| Mean weight: | 90.4 ± 14.0 kg |

*According to Kellgren and Lawrence: classification of osteoarthritis from 0 to 4 with 0: none, 1: doubtful, 2: minimal, 3: moderate, and 4: severe osteoarthritis

Study Design

Secondary¹ analysis of a randomized, monocentric, clinically prospective cross-over study:

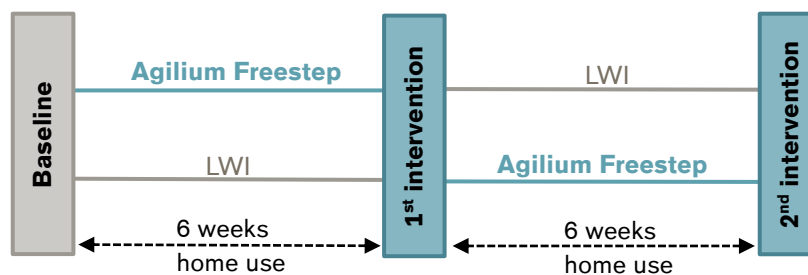

In the original study patients were randomly divided into two groups. Each patient received either a lateral wedged insole (LWI) or an ankle-foot orthosis (AFO; Agilium Freestep). After the first six-week phase patients switched to the alternate intervention for another six weeks. The patients underwent three assessments: at baseline, after six weeks (1st intervention) and after twelve weeks (2nd intervention).

This is a secondary analysis of the study specifically investigates the influence of limb alignment (mechanical axis deviation, MAD) on pain reduction - a relationship not previously explored in the primary study. It uses radiographic data (MAD) to correlate with pain outcomes from a questionnaire.

¹ Schwarze M, Bartsch LP, Block J, Alimusaj M, Jaber A, Schiltewolf M, et al. A comparison between laterally wedged insoles and ankle-foot orthoses for the treatment of medial osteoarthritis of the knee: a randomized cross-over trial. Clin Rehabil 2021; 10.1177/0269215521993636:269215521993636

Results

| Functions and Activities | | | | | | Participation | Environment |
|-----------------------------------|------------------------------|--------|-----|------------------|------------------|---------------|------------------|
| Biomechanics – Static Measurement | Biomechanics – Gait analysis | X-Rays | EMG | Functional tests | Clinical effects | Satisfaction | Health Economics |

| Category | Outcomes | Results for Agilium Freestep vs. LWI | | Sig.* |
|------------------|---|--|--------------------------------|-------|
| Clinical effects | Knee pain at examination dates [0 – no pain, 10 – worst pain] | Knee pain was reduced after both interventions. | | n.a. |
| | | Examination date | Knee pain [Median (quartiles)] | |
| | | Baseline | 5 (3; 7) | |
| | | After LWI** use | 3.5 (1; 7) | |
| | | After Agilium Freestep** use | 3 (2; 5) | |
| | Change of knee pain (Δ pain) | Knee pain is significantly reduced by both aids, but no significant difference between aids was found. | | |
| | | Examination dates | Knee pain [Median (quartiles)] | |
| | | LWI – baseline | -0.5 (-3; +1) | ++ |
| | | Agilium Freestep – baseline | -1.5 (-3; +1) | ++ |
| | | LWI vs Agilium Freestep | n.a. | 0 |

| Category | Outcomes | Results for Agilium Freestep vs. LWI | Sig.* | | | | | | | | | | | | | | | | | |
|--|--|--------------------------------------|-------|------------------|----------------|----|----|--------------------|----|----|------------------|-------|-------|-----------------|------|------|---------------------|------|------|--|
| Correlations | Mechanical axis deviation vs. | Tibiofemoral angle | ++ | | | | | | | | | | | | | | | | | |
| | | Knee pain after LWI | 0 | | | | | | | | | | | | | | | | | |
| | | Knee pain after AFO | ++ | | | | | | | | | | | | | | | | | |
| | | Knee pain change LWI | ++ | | | | | | | | | | | | | | | | | |
| | | Knee pain change AFO | ++ | | | | | | | | | | | | | | | | | |
| | | Knee pain baseline | 0 | | | | | | | | | | | | | | | | | |
| | Tibiofemoral angle vs. | Knee pain after LWI | 0 | | | | | | | | | | | | | | | | | |
| | | Knee pain after AFO | ++ | | | | | | | | | | | | | | | | | |
| | | Knee pain change LWI | ++ | | | | | | | | | | | | | | | | | |
| | | Knee pain change AFO | ++ | | | | | | | | | | | | | | | | | |
| | | Knee pain baseline | 0 | | | | | | | | | | | | | | | | | |
| | Knee pain after LWI vs. | Knee pain baseline | ++ | | | | | | | | | | | | | | | | | |
| Knee pain after AFO vs. | Knee pain baseline | ++ | | | | | | | | | | | | | | | | | | |
| Responders and non-responders to LWI and Agilium Freestep | Responders and non-responders to LWI and Agilium Freestep with calculated cut-off values for the mechanical axis deviation (MAD) | n.a. | | | | | | | | | | | | | | | | | | |
| | <table border="1"> <thead> <tr> <th></th> <th>LWI</th> <th>Agilium Freestep</th> </tr> </thead> <tbody> <tr> <td>Responders (n)</td> <td>14</td> <td>16</td> </tr> <tr> <td>Non-responders (n)</td> <td>14</td> <td>12</td> </tr> <tr> <td>Cut-off MAD (mm)</td> <td>14.25</td> <td>14.75</td> </tr> <tr> <td>Sensitivity (%)</td> <td>85.7</td> <td>87.5</td> </tr> <tr> <td>1 – specificity (%)</td> <td>28.6</td> <td>25.0</td> </tr> </tbody> </table> | | LWI | Agilium Freestep | Responders (n) | 14 | 16 | Non-responders (n) | 14 | 12 | Cut-off MAD (mm) | 14.25 | 14.75 | Sensitivity (%) | 85.7 | 87.5 | 1 – specificity (%) | 28.6 | 25.0 | |
| | LWI | Agilium Freestep | | | | | | | | | | | | | | | | | | |
| Responders (n) | 14 | 16 | | | | | | | | | | | | | | | | | | |
| Non-responders (n) | 14 | 12 | | | | | | | | | | | | | | | | | | |
| Cut-off MAD (mm) | 14.25 | 14.75 | | | | | | | | | | | | | | | | | | |
| Sensitivity (%) | 85.7 | 87.5 | | | | | | | | | | | | | | | | | | |
| 1 – specificity (%) | 28.6 | 25.0 | | | | | | | | | | | | | | | | | | |

*no difference (0), positive trend (+), negative trend (–), significant (++/--), not applicable (n.a.)
significance set at $p < 0.05$; trends set at $0.1 > p > 0.05$

**LWI- laterally wedged insole, Agilium Freestep: ankle-foot orthosis (AFO)

Author's Conclusion

"Both laterally wedged insoles and ankle-foot orthoses can be used successfully to reduce pain in medial osteoarthritis of the knee. The success of the therapy can be predicted with a sensitivity >80% by the mechanical axis deviation on full weight-bearing long-standing radiographs." (Bartsch et al., 2022)

Author's Affiliation

¹Clinic for Orthopedics, Heidelberg University Hospital, Heidelberg, Germany

© 2025, Otto Bock HealthCare Products GmbH ("Otto Bock"), All Rights Reserved. This article contains copyrighted material. Wherever possible we give full recognition to the authors. We believe this constitutes a 'fair use' of any such copyrighted material according to Title 17 U.S.C. Section 107 of US Copyright Law. If you wish to use copyrighted material from this site for purposes of your own that go beyond 'fair use', you must obtain permission from the copyright owner. All trademarks, copyrights, or other intellectual property used or referenced herein are the property of their respective owners. The information presented here is in summary form only and intended to provide broad knowledge of products offered. You should consult your physician before purchasing any product(s). Otto Bock disclaims any liability related from medical decisions made based on this article summary.