

Opalescence™

whitening toothpaste

ZH - 说明: 有助于预防蛀牙。

EN - Cool mint with fluoride ZH - 说明: 有助于预防蛀牙。

EN - Indications: Aids in the prevention of dental decay ZH - 说明: 成人和两岁或两岁以上幼儿使用方法

EN - Directions: Adults and children over 6 years of age: Brush teeth thoroughly, preferably after each meal or at least twice a day, or as directed by a dentist or physician. Children 6 years and younger: Use a pea-sized amount for supervised brushing to minimize swallowing. In case of intake of fluoride from other sources, consult a dentist or physician. Children under 2 years of age: Consult a dentist or physician.

ZH - 用法说明: 成人和6岁以上的儿童: 彻底刷牙, 最好每顿饭饭后或至少一天两次, 或遵照医嘱。6岁和6岁以下的儿童: 在家长监督下用豌豆大小的量刷牙, 以尽量减少吞咽。如果不慎通过其它途径吞服氟化物, 请教牙医或医生。2岁以下儿童: 请教牙医或医生。

EN - Warning: As with all fluoride toothpastes, keep out of the reach of children under 6 years of age. If you accidentally swallow more than used for brushing, seek professional assistance or contact a Poison Control Center immediately. Regulations on administering fluoride containing products may differ from country to country.

ZH - 警告: 不应使六岁以下儿童自己接触所有的含有氟物质的牙膏。如果你偶然吞进多于一次刷牙用的牙膏, 可寻求专业咨询或者立即同毒品控制中心联系。各国对控制产品中的氟物质可能会有不同规定。

Toothpaste Ingredients: Contains Sodium Fluoride 0.25% w/w, Glycerin, Water (Aqua), Silica, Sorbitol, Xylitol, Flavor (Aroma), Poloxamer, Sodium Lauryl Sulfate, Carbomer, FD&C Blue#1 (CI 42090), FD&C Yellow#5 (CI 19140), Sodium Benzoate, Sodium Hydroxide, Sparkle (CI 77019, CI 77891), Sucralose and Xanthan Gum.

Opalescence PF 35% Regular:
Ingredients: Glycerin, Water (Aqua), Carbamide (Urea) Peroxide, Carbomer, Hydrogen Peroxide, PEG-300, Xylitol, Sodium Hydroxide, EDTA, Potassium Nitrate, Sodium Fluoride, Sucralose

Opalescence PF 35% Mint:
Ingredients: Glycerin, Water (Aqua), Carbamide (Urea) Peroxide, Carbomer, Hydrogen Peroxide, PEG-300, Xylitol, Sodium Hydroxide, EDTA, Flavor (Aroma), Potassium Nitrate, Sodium Fluoride, Sucralose

Opalescence PF 35% Melon:
Ingredients: Glycerin, Water (Aqua), Carbamide (Urea) Peroxide, Carbomer, Hydrogen Peroxide, PEG-300, Xylitol, Sodium Hydroxide, Flavor (Aroma), EDTA, Potassium Nitrate, Sodium Fluoride, Sucralose

Opalescence PF 20% Regular:
Ingredients: Glycerin, Water (Aqua), Carbamide (Urea) Peroxide, Xylitol, Carbomer, PEG-300, Sodium Hydroxide, EDTA, Potassium Nitrate, Sodium Fluoride

Opalescence PF 20% Mint:
Ingredients: Glycerin, Water (Aqua), Carbamide (Urea) Peroxide, Xylitol, Carbomer, PEG-300, Sodium Hydroxide, EDTA, Flavor (Aroma), Potassium Nitrate, Sodium Fluoride

Opalescence PF 20% Melon:
Ingredients: Glycerin, Water (Aqua), Carbamide (Urea) Peroxide, Xylitol, Carbomer, PEG-300, Sodium Hydroxide, Flavor (Aroma), EDTA, Potassium Nitrate, Sodium Fluoride

Opalescence PF 15% Regular:
Ingredients: Glycerin, Water (Aqua), Carbamide (Urea) Peroxide, Xylitol, Carbomer, PEG-300, Sodium Hydroxide, EDTA, Potassium Nitrate, Sodium Fluoride

Opalescence PF 15% Mint:
Ingredients: Glycerin, Water (Aqua), Carbamide (Urea) Peroxide, Xylitol, Carbomer, PEG-300, Sodium Hydroxide, EDTA, Flavor (Aroma), Potassium Nitrate, Sodium Fluoride

Opalescence PF 15% Melon:
Ingredients: Glycerin, Water (Aqua), Carbamide (Urea) Peroxide, Xylitol, Carbomer, PEG-300, Sodium Hydroxide, Flavor (Aroma), EDTA, Potassium Nitrate, Sodium Fluoride

Opalescence PF 10% Regular:
Ingredients: Glycerin, Water (Aqua), Xylitol, Carbamide (Urea) Peroxide, Carbomer, PEG-300, Sodium Hydroxide, EDTA, Potassium Nitrate, Sodium Fluoride

Opalescence PF 10% Mint:
Ingredients: Glycerin, Water (Aqua), Xylitol, Carbamide (Urea) Peroxide, Carbomer, PEG-300, Sodium Hydroxide, EDTA, Flavor (Aroma), Potassium Nitrate, Sodium Fluoride

Opalescence PF 10% Melon:
Ingredients: Glycerin, Water (Aqua), Xylitol, Carbamide (Urea) Peroxide, Carbomer, PEG-300, Sodium Hydroxide, Flavor (Aroma), EDTA, Potassium Nitrate, Sodium Fluoride



EN- Health hazard
ZH- 健康危害



EN- See Instructions
ZH- 参阅说明



EN- Keep out of reach of children
ZH- 勿让儿童接触



EN- For Professional use only
ZH- 仅用于专业用途



EN- Lot Number
ZH- 批号



EN- Catalog number
ZH- 目录编号



EN- Recycle
ZH- 回收使用



EN- Use by date
ZH- 保质期



EN - Required storage condition
ZH - 所需的储存条件



EN- Keep products out of heat/sunlight
ZH- 产品的保存须避热/避阳光。



ZH- 合格标识

All Opalescence products are gluten-free.
© Copyright 2019 Ultradent Products, Inc. All Rights Reserved.
For Professional Use Only. Made in USA.

1008220.01 022520

Opalescence™

tooth whitening systems

EN-Tooth Whitening System
ZH- 白齿系统

Manufactured by
ULTRADENT
PRODUCTS, INC.
505 West 10200 South
South Jordan, UT 84095, USA
www.ultradent.com


EN-PATIENT INSTRUCTIONS



Follow your dentist's instructions on how to load gel into your custom bleaching tray. Use no more than 1/3 to 1/2 of syringe.




Brush teeth before inserting tray. Lightly tap tray to adapt tray sides to teeth. Wear Opalescence 10% for 8-10 hours or overnight, Opalescence 15% for 4-6 hours, Opalescence 20% for 2-4 hours, and Opalescence 35% for 30 minutes.

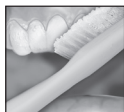

Remove excess gel with clean finger or soft toothbrush. Rinse twice; do not swallow rinsed gel.



If significant sensitivity occurs, stop treatment and consult dentist.

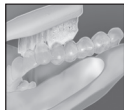

Clean tray with soft brush and cool tap water. Store tray in case provided.



Nonvital Patient Applied Bleaching: Irrigate opening as instructed by dentist. Place gel into lingual access and tray.

Bleaching Precautions:


1. Do not use household bleach to whiten teeth.
2. Pregnant women should not bleach.
3. If you have any questions regarding the appropriate use of this product, including how long it will take to bleach your teeth, please consult our dentist as soon as possible.
4. Do not use tobacco products or eat while bleaching.
5. White spots on the enamel or that appear during bleaching may blend during continued bleaching process.
6. Teeth are naturally darker along the gumline. These areas may require more time to lighten than the rest of the tooth surface and usually remain slightly darker.

General Precautions:

1. A small percentage of patients experience sensitivity with bleaching. Should this occur, remove the tray and contact your dentist.
2. Foods and juices high in citric acid can cause sensitivity to the teeth.
3. Some patients have noticed temporary discomfort of the gums, lips, throat, or tongue. Should any of these symptoms persist more than two days or progressively worsen, call your dentist. These side effects will usually subside within 1-3 days after treatment is discontinued.
4. Coffee, tobacco, and other products can re-stain your teeth over time. Should this occur, the teeth can be re-whitened within a few nights using Opalescence.
5. Regular dental checkups and cleanings are important before and after bleaching to maintain a healthy smile.
6. Some old amalgam or "silver" fillings may leave a dark purple color in your bleaching tray; this is normal.
7. Crowns, bridges, partial dentures, veneers, and composite fillings will not bleach.
8. Store bleach out of the sun and heat. Refrigeration recommended. Do not freeze.
9. Discard any unused bleaching gel after treatment is completed.
10. Keep Opalescence out of reach of small children.
11. Do not swallow gel or rinsed gel. Product contains peroxide and may contain fluoride; swallowing large amounts can be harmful.



按照牙医的指导将凝胶装入定制的漂白牙托。凝胶用量不超过1/3到1/2针管。

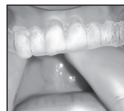


放入牙托前先行刷牙。轻轻敲击牙托，让牙托与牙齿侧面尽量贴合。10%浓度Opalescence凝胶需佩戴8-10小时，或整夜使用。15%的Opalescence凝胶需戴4-6小时，20%浓度的2-4小时，35%浓度的只需30分钟。



用干净手指或软毛刷去除多余的凝胶。冲洗两次。切勿吞咽冲洗下来的凝胶。



如果发生严重过敏，请停止治疗并咨询牙医。

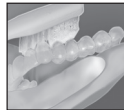

用软毛刷和冷水清洗牙托。将牙托放进专用盒。



死髓牙专用漂白剂: 按医嘱冲洗开口。将凝胶挤入舌侧开口和牙托。

ZH-患者指导

漂白警告:

1. 切勿使用家用漂白剂漂白牙齿。
2. 孕妇切勿漂白。
3. 如果有关于产品用法的任何问题，包括漂白所需的时间长度，请尽快向牙医咨询。
4. 漂白时请勿使用烟草产品或进食。
5. 漂白过程中在牙釉质上出现的白斑会在进一步的漂白过程中逐渐消失。
6. 沿牙龈线处牙齿颜色偏暗。漂白上述区域可能比牙齿表面的其他区域需要更多时间，而且一般在漂白完成后颜色仍然偏暗。

一般警告:

1. 小部分患者在漂白时可能出现过敏反应。如果出现过敏症状，请取下牙托，与牙医联系。
2. 柠檬酸含量较高的食品和饮料可能诱发牙齿过敏。
3. 有些患者会出现牙龈、嘴唇、喉咙或舌头暂时性的不适症状。如果上述症状持续超过两天或有加重趋势，请致电牙医咨询。上述副作用一般会在停止治疗后的1-3天内自动缓解。
4. 咖啡、烟草和其他产品过一段时间可以令牙齿重新着色。如果发生这种情况，可以用Opalescence凝胶重新对牙齿进行漂白。为保持健康笑容，漂白前后进行常规牙科检查和清洁至关重要。
5. 某些老的汞合金或“银”填料可能会在漂白牙托上留下暗紫的痕迹。这属于正常情况。
6. 漂白凝胶对牙冠、牙桥、部分假牙、饰面以及合成填料无效。
7. 将漂白凝胶保存在阴凉干燥处。建议进行冷藏。切勿进行冷冻。
8. 治疗结束后丢弃所有未用完的漂白凝胶。
9. 切勿让儿童接触Opalescence漂白凝胶。
10. 切勿吞嚼或冲洗凝胶。本品含有过氧化氢成分，可能含有氟化物；大量吞嚼可能对身体有害。

电话: 888.863.5883

传真: 801.553.4600

中国区代理人及售后服务机构

名称: 广州市皓齿登医疗器械有限公司

地址: 广州市海珠区工业大道北路67号34号楼首层自编102房

电话: 020-84156506

传真: 020-84156535

产品名称: 牙齿美白胶

规格型号: Opalescence PF 10%, Opalescence PF 15%, Opalescence PF 20%, 1.2ml/支

医疗器械注册证编号/产品技术要求编号: 国械注进20183172676