Opalescence Boost® 40% Tooth Whitening System
Product Description: Opalescence® Boost® 40% hydrogen peroxide whitening gel is for in-office use only. The 2-barrel 100 mL syringe provides strength for both clinical application and precise dosing of the activator. One barrel contains a unique chemical activator, which is combined with a second barrel containing hydrogen peroxide.

For all products described, carefully read and understand all instructions and SDS information prior to use.

Delivery Form(s):
• 12 mL Delivery Syringe
• Recommended delivery Tip
• Black Mini™ Tip

Indications for Intended Purpose
Opalescence Boost 40% whitening is for in-office use only. It is applied by the dental professional for whitening one or more teeth. Opalescence Boost in-office whitening is also used on non-vital teeth, including in-office intraorally whitening. Opalescence Boost in-office whitening is an alternative, conservative method (compared to crowns, veneers, etc.) for whitening dark, internally discolored teeth caused by predisposing factors such as disease, traumatic or iatrogenic injury.

Contraindications
• Patients with severe allergic reactions to chemical ingredients can include but are not limited to elevated bilirubin levels, tetracycline and adult minocycline stains, porphyria, erythroblastosis fetalis and high fluoride intake during tooth development.
• Pregnant or nursing women should not use teeth whitening products.
• For patients or users with allergy concerns, refer to product allergen document available at www.ultradent.com. If allergic reaction is observed, rinse exposed area thoroughly with tap water and have the patient consult their physician

Warnings and Precautions

• Use only recommended tips.
• Avoid direct contact of the surface of the tooth whitening product with the gums and/or salivary flow.
• For patients with known tooth sensitivity, pre-applied tooth treatment with Ultra™ Sensitizing gel, Opalescence® Whitening Toothpaste with Sensitivity Relief, and/or Enamelast® fluoride varnish is recommended.
• Some patients may experience temporary discomfort of the gums, lips, throat, tongue, or teeth. These symptoms persist for a few minutes after discontinuing use, or in cases of severe discomfort, call your dental professional.
• If the product is ingested, drink large amounts of water and contact a physician. If the product is aspirated, seek immediate medical attention.
• Inform patients restorative materials will not whiten, and we recommend whitening dark teeth before restorative placement.

• Wait at least 7–10 days after whitening procedures before placing aesthetic restorations.
• Patients with serious health concerns should consult their primary care provider prior to treatment.
• Do not place the product on teeth immediately if product comes into contact with teeth.
• To prevent additional irritation, gingival and general oral health should be evaluated prior to initiating treatment.
• Etching enamel or prophyl paste polishing before any whitening procedure is not recommended and may increase the whitening process but can cause sensitivity.
• Restorations should be adequately sealed, and all exposed sensitive dentin should be covered. A history of sensitivity exists, treat with appropriate restoration or blocking agent such as dentin bonding or temporarily with dentin sealant agent or fluoride varnish.
• Monitor patients' whitening progress to prevent over-bleaching the desired level of whitening or degree of translucency.

• If additional whitening is desired, re-schedule and repeat the Opalescence Boost 40% in-office procedure about 2 weeks after use. Opalescence® Endo whitening.
• Regression is to be expected. Follow-up whitening treatment every 3–5 years is recommended.
• If you have any questions regarding this product, please find additional information at <http://www.ultradent.com>.
• For more information, visit www.ultradent.com/categories/whitening/in-office/opalescence-boost

Processing
• To avoid cross-contamination, wipe syringe with an intermediate level disinfectant between each use, using a disposable barrier sleeve. If a barrier sleeve is not used, syringe should be treated as single-use.

Storage and Disposal

Store according to outer package label.
• Dispose of waste according to local rules, guidelines, and regulations.

• Before disposal, the syringes aspirate water into the syringe and expurge liquid down the drain. Repeat a couple of times before disposing of the syringe. Make sure any gauges used are rinsed with water.
• Press the plunger into the red syringe, pushing all contents into the clear syringe.
• Press the contents of the clear plunger completely back into the red syringe.
• To thoroughly mix activator with whitening gel, push stems back and forth continually and mix 25 times each side.

6. Press all mixed gel into red syringe.

7. Continue the two syringes and securely attach the recommended tip (Black Mini Tip) onto the red syringe.

8. To thoroughly mix activator with whitening gel, push stems back and forth continually and mix 25 times each side.

9. Press all mixed gel into red syringe.

10. Continue the two syringes and securely attach the recommended tip (Black Mini Tip) onto the red syringe.

11. Verify flow, consistency, and that the color is red before applying intraorally. If resistance is met, replace tip and re-check. If gel is not red, do not use and refer to Step 10.
• Mixing Opalescence Boost In-Office Whitening® section.
NOTE:

• Opalescence Boost in-office whitening should be used within 10 days of mixing. Keep refrigerated between uses.

• Refer to mixing card included.

Isolation – OpalDam® Resin Barrier (Recommended Isolation)

Remove Luer lock cap from OpalDam® resin barrier and securely attach recommended delivery tip. Refer to Warnings and Precautions.

2. Verify flow prior to applying intraorally.
3. Rinse and dry teeth and gums thoroughly.
Place UltraTrend IsoBlock™ bite block and die supporting lip cheekpiece over UltraTrend Umbrella™ retractor to avoid Opalescence Boost whitening gel from contacting any soft tissue.

4. Apply OpalDam resin barrier along the gingival margin, overlapping approximately 2–3 mm into the enamel. Blend and finish one tooth beyond the most distal tooth that is being whitened. Use barrier to isolate tooth/teeth to be treated.

5. Express resin through any open embrasures onto the lingual tissue to completely seal each exposed pulp space.

Use – Applying Opalescence Boost 40% In-Office Whitening

1. Apply 0.5–1.0 mm thick layer of gel to the labial surface of the tooth.
2. Allow gel to remain on the teeth 20 minutes per application, periodically checking on patient comfort and suctioning of saliva

NOTE: Avoid contact with soft tissue or salivary flow. If chemical irritation occurs due to the intraoral soft tissue coming into contact with the peroxide gel, immediately wipe the gel from the tissue using a wet gauze and rinse the area well. Tissue should return to normal within 20–30 minutes. Refer to Warnings and Precautions.

3. Suction gel from teeth using the Luer Vac and SST™ (Surgical Suction Tip) or a surgical suction tip. To avoid gel splatter, do not use water while suctioning gel. When no gel is visible, lightly rinse and dry. Use caudal flow to dislodge the isolation barrier or rubber dam seal.

4. Evaluate shade change after each application and monitor patient for sensitivity and irritation. If persistent sensitivity occurs, discontinue use.

5. Repeat steps under section "Applying Opalescence Boost 40% In-Office Whitening" for no more than 3 applications if desired results are not been achieved. Do not exceed 3 applications per visit.

6. Retreat as necessary using professional judgment.

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9. Vérifier le débit, la consistance et que la couleur est claire, thereby becoming more obvious during treatment. Continue whitening treatment on the unaffected tooth surface blends. Re-evaluate two weeks after whitening treatment when tooth color has stabilized.

10. For a small percentage of the population, their teeth will not appreciably whiten.

• Some amalgam fillings may leave a dark purple color in your whitening tray, this is normal.

• Teeth can re-stain over time, so do touch up teeth whitening sessions as necessary.

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2. Verify flow prior to applying intraorally.

3. Rinse and dry teeth and gums thoroughly.

Place UltraTrend IsoBlock™ bite block and die supporting lip cheekpiece over UltraTrend Umbrella™ retractor to avoid Opalescence Boost whitening gel from contacting any soft tissue.

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