LC Block-Out Resin	EN
<div>Product Description</div> <p>Ultradent LC Block-Out Resin is a viscous light-cured, methacrylate-based, blue pigmented utility resin which is syringe delivered for precise delivery. For all products described, carefully read and understand all instructions and SDS information prior to use.</p> <div>Delivery Form(s)</div> <ul style="list-style-type: none">1. 2 mL Delivery Syringe • Delivery Tip: <ul style="list-style-type: none">Black Mini™ Tip <div>Indications for Use/ Intended Purpose</div> <p>Ultradent LC Block-Out Resin is designed for laboratory use such as for bleaching reservoirs on plaster models, for lab procedures requiring spacing or undercut block out, and to repair small fractures or voids in laboratory models.</p> <div>Warnings and Precautions</div> <ul style="list-style-type: none">• LC Block-Out Resin is not intended for intraoral use. • Keep product out of direct sun and heat. <div>Stepwise Instructions</div> <div>Preparation</div> <p>NOTE: Stone/plaster models must be clean and dry before applying resin.</p> <ol style="list-style-type: none">1) Remove Luer cap. 2) Securely attach recommended tip (Black Mini™ Tip).	<div>3) Verify resin flow prior to applying. If resistance is met, replace tip and re-check. Use only recommended tips.</div> <div>Use – Model Repair or Block-Out</div> <ol style="list-style-type: none">1) Fill area no more than 3mm thick at a time. 2) Cure thoroughly. 3) Add additional resin and cure as necessary. 4) Wipe off the oxygen inhibition layer. <div>Use – Custom Whitening Tray Reservoirs</div> <ol style="list-style-type: none">1) Keep resin approximately 1.5mm from the gingival margin. It is recommended that LC Block-Out Resin be kept off the biting surfaces. 2) Apply 0.5mm thick of LC Block-Out Resin to the desired surfaces. 3) Using VALO™ curing light, cure 5 seconds per tooth. 4) Wipe off the oxygen inhibition layer. 5) Fabricate custom tray. <div>Storage and Disposal</div> <p>Dispose of waste according to local rules, guidelines, and regulations.</p> <div>Miscellaneous Information</div> <p>Report any serious incident to the manufacturer and the competent authority. <p>www.ultradent.com /1-800-552-5512 / 801-572-4200</p></p>

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