File-Eze™ EDTA Chelating & Filing Lubricant

Description & Indications:
File-Eze™ lubricant is a 19% EDTA dentin chelating agent in an aqueous, water-soluble solution with a lubricating base.

Indications:
Use File-Eze lubricant to aid in instrumentation and debridement of root canals.

Procedure:

Preliminary Steps:

A. IndiSpense™ system delivery:

1. The pre-labeled unit dose File-Eze lubricant syringe is attached to the end of the large, no-waste IndiSpense syringe by turning the Luer Lock of the unit dose syringe snugly onto the male thread of the IndiSpense syringe.

2. Depress plunger of IndiSpense syringe with palm of hand while stabilizing the unit dose syringe at the plunger with the non-dominant hand. Slightly pull on plunger of unit dose syringe to facilitate loading. Do not express contents of unit dose syringe back into IndiSpense syringe. To prevent cross-contamination, a used syringe should not be reattached to the IndiSpense syringe.

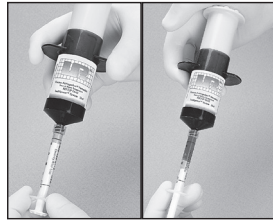
3. Place cap on unit dose syringe if not using immediately, or attach a Mini tip and cover with Ultradent's TipSoc™ covers.

4. When ready for use, continue with procedure steps 1-4.

B. Preloaded 1.2ml syringe delivery:

1. Remove Luer Lock cap from the syringe.

2. Twist a NaviTip™ tip firmly onto the syringe. For optimal control, hold syringe with plunger in palm of hand, instead of in conventional manner with thumb on plunger.

Figs. 1&2 A 1.2ml unit dose syringe is twisted firmly onto the IndiSpense syringe. While supporting plunger of unit dose syringe with the non-dominant hand, depress plunger of IndiSpense syringe and fill unit dose syringe to desired level.

Procedure:

1. The NaviTip™ tip can be used to deliver File-Eze lubricant directly into the canal opening, as shown in Fig. 3. When the canal is difficult to find, start by placing point of file into canal opening prior to introducing File-Eze lubricant to pulp chamber (or onto file.) Visibility is not impaired.

2. Once canal opening is engaged, express File-Eze lubricant through tip into canal to no more than half the working length. As an alternative, a small amount of File-Eze lubricant can be placed directly onto file as seen in Fig. 4.

3. File-Eze lubricant is only used with the first two or three instruments. A thorough amount of sodium hypochlorite is used during entire instrumentation and cleansing procedure, alternating with liquid EDTA between each file.

4. File-Eze lubricant is irrigated and rinsed from canal, similar to other root canal preparations; ChlorCid™ sodium hypochlorite solution is recommended. A slight effervescent cleansing action occurs when ChlorCid solution contacts residual File-Eze lubricant, which helps remove extraneous material from canal.

Precautions:

1. Deliver only to canal orifice. Do not express material with pressure up canal past apical foreamen.
2. Tips are disposed after use.
3. Clean and disinfect syringes between patients.
4. NaviTip tips do not have a side port hole. Do not express past apex.

EN Filing and Chelating Agent
DE EDTA-Gel zur Wurzelkanalaufbereitung
FR Agent Chélatant
NL Chelaterende gel
IT Agente Chelante Endodontico
ES Gel Quelante
PT Preparação do Canal com Quelante EDTA
SV Upplösande, Smörjande Dentin Behandlare
DA Smøre- og chelateringsmiddel
FI Pehmentävä ja Voiteluva Dentiinin Käsittelyaine

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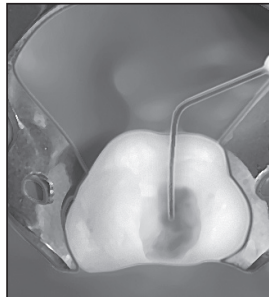
Fig. 3 File-Eze lubricant is placed after canal orifice is engaged. Visibility is not impaired.


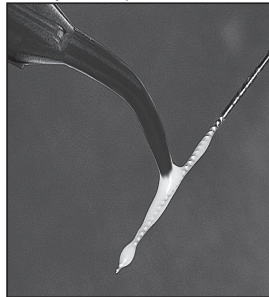
Fig. 4 File-Eze lubricant may be applied directly on file if desired.

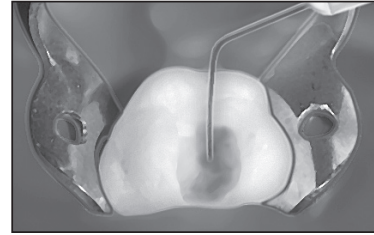File-Eze™ EDTA Chelating & Filing Lubricant

EN Place File-Eze solution into canal from bottom up. Following instrumentation, irrigate with sodium hypochlorite.

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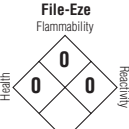
Hazard Rating
4 = Severe
3 = Serious
2 = Moderate
1 = Slight
0 = Minimal

Ultra-dent syringes have an expiration date stamped on the side of the syringe consisting of one letter and three numbers. The letter is a lot number used for manufacturing purposes and the three numbers are the expiration date. The first two numbers are the month, and the third number is the last number of the year.

U.S. federal law restricts the sale of this device by or on the order of a state licensed dental professional.

Keep out of reach of children.

For immediate reader and/or complete descriptions of Ultra-dent's product line, refer to Ultra-dent's catalog call Toll Free 1-800-552-5512. Outside U.S. call (801) 572-4200 or visit www.ultra-dent.com.

File-Eze™
EDTA Chelating & Filing Lubricant

Manufactured by:
Ultra-dent Products, Inc.
505 West 10200 South
South Jordan, Utah 84095 USA
Made in the USA from U.S. and globally sourced materials

For product SDS please see our website: www.ultra-dent.com
1-800-552-5512; 801-572-4200

ULTRA-DENT
PRODUCTS, INC.
© Copyright 2022 Ultra-dent Products, Inc. All Rights Reserved.
10187-UAR09 021122

EN

General Precautions:
1.For professional use only.
2.Review instructions, precautions, and MSDS before beginning treatment. Use only as directed.
3.Keep products out of heat/sunlight.
4.Avoid skin exposure to resins.
5.Isolate strong chemicals to area of treatment.
6.Confirm that patient has no known allergies to treatment materials.
7.Test flow of materials from syringe and tip before using intraorally.
8.Never force syringe plungers.
9.Use Ultra-dent™ syringe covers and/or clean and disinfect syringes between patients.

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