PermaFlo™ Purple	
Anatomical Indicating Composite	
Product Description	
PermaFlo™ Purple composite is a 68% filled, light-cured, radiopaque, thixotropic, methacrylate-based flowable composite with 1µ average particle size. Its color aids in identifying root canal orifices when restoring a tooth following an endodontic procedure. For all products described, carefully read and understand all instructions and SDS information prior to use.	
Delivery Form(s)	
<ul style="list-style-type: none">1,2 ml delivery syringeDelivery tip:<ul style="list-style-type: none">Micro 20 ga spids	
Indications for Use/Intended Purpose	
Restorative composites are used for direct and indirect restorations in both the anterior and posterior regions. PermaFlo Purple can be used for anterior and posterior restorations following endodontic procedures.	
Contraindications	
<ul style="list-style-type: none">Do not use on patients with known sensitivity to methacrylates or other resins. If dermatitis, rash, or other allergic reaction is observed, rinse exposed area thoroughly and have patient consult their physician.For patients or users with allergy concerns, refer to product allergen document available at www.ultradent.com. If allergic reaction is observed, rinse exposed area thoroughly with water and have the patient consult their physician.	
Warnings and Precautions	
<ul style="list-style-type: none">Resins can be sensitizing. Do not use on patients with a known sensitivity to acrylics or other resins. If allergic reaction, dermatitis, or rash develops, have patient consult their physician.To avoid cross-contamination, use clean gloves and a clean instrument.Redirect overhead light to prevent premature polymerization.To optimize bond to each succeeding	
layer, do not remove oxygen inhibition layer between incremental layers of composite.	
<ul style="list-style-type: none">Ensure composite is totally polymerized between layers.	
Stepwise Instructions	
Preparation	
<ol style="list-style-type: none">Complete endodontic obturation. Clean pulpfill floor with cotton and isopropyl alcohol and dry.Apply dentin bonding agent to entire preparation according to manufacturer's instructions.Remove Luer Lock cap and securely attach recommended delivery tip.Verify flow prior to applying intraorally.	
Use	
<ol style="list-style-type: none">Apply ≤ 1.0 mm layer of PermaFlo Purple composite over each canal orifice.Place curing lights with an intensity of ≥ 800 mW/cm2 in the wavelength region of 385–515 nm, cure each layer for 10 seconds.Place temporary or permanent restoration.Place curing lights with an intensity of ≥ 800 mW/cm2 in the wavelength region of 385–515 nm, cure each layer for 10 seconds.Place temporary or permanent restoration.Place curing lights with an intensity of ≥ 800 mW/cm2 in the wavelength region of 385–515 nm, cure each layer for 10 seconds.Place temporary or permanent restoration.Place curing lights with an intensity of ≥ 800 mW/cm2 in the wavelength region of 385–515 nm, cure each layer for 10 seconds.	
Processing	
To avoid cross-contamination, wipe syringe with an intermediate level disinfectant before using and use a disposable barrier sleeve. If a barrier sleeve is not used, syringe should be treated as single-use.	
Storage and Disposal	
Dispose of waste according to local rules, guidelines, and regulations.	
Miscellaneous Information	
Report any serious incident to the manufacturer and the competent authority. www.ultradent.com / 1-800-552-5512 / 801-572-4200	

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