## Opalustre™ Enamel Micro-AbrasionSlurry EN

**Product Description**
Opalustre is a chemical and mechanical abrasion slurry containing 6.6% hydrochloric acid and silicon carbide micro particles that can be rinsed off with water. For all instructions, carefully read and understand all described, SADS information prior to use.

### Delivery Forms)

- 1.2 ml syringe
- Delivery tip
- White Mac™ Tip
- OpalCups™
- Bristle
- Ersterhandling

**Use**
1) Apply Opalustre layer over the area to be treated.
2) Using an OpalCups Bristle at a slow RPM (approximately 500 RPM), apply intermittent medium to heavy pressure for approximately 60 seconds per application.
3) Remove the paste with suction then rinse, evaluate and repeat. (2.2mm has been removed, and repeat as necessary.

**Cleanup**
1) After final rinse, remove rubber dam, or OpalDam/ OpalDam Green, thoroughly.

**Contraindications**
• Do not use on severe fluorosis.
• Do not use on thin enamel.
• For patients or users with allergy concerns, refer to product allergen document available at www.ultradent.com. If allergic reaction is observed, rinse exposed area thoroughly with water and have the patient consult their physician.

**Warnings and Precautions**
• Because of the hydrochloric acid, metuliculous isolation is extremely important when using this product.
• Avoid contact with soft tissue. If product comes in contact with soft tissue, immediately rinse area thoroughly with water.
• Patient, clinician, and assistant must wear protective eyewear.

• Use only recommended tips.
• Keep away from heat and direct sunlight.
• If rubber dam is not used, follow instructions for OpalDam application.
• If allergic reaction is observed, rinse exposed area thoroughly and have the patient consult their physician.

**Stepwise Instructions**
**Preparation**
1) It is recommended to take initial photographs for reference.
2) Using rubber dam and clamps, or following instructions for OpalDam/OpalDam Green application, isolate tooth or teeth to be treated along with one tooth adjacent on each side.
3) Remove Luer lock cap from Opalustre syringe and securely attach White Mac™ tip.
4) Verify flow prior to applying intraoral. If resistance is met, replace tip and re-check.

**Process**
1) Apply Opalustre layer over the area to be treated.
2) Using an OpalCups Bristle at a slow RPM (approximately 500 RPM), apply intermittent medium to heavy pressure for approximately 60 seconds per application.
3) Remove the paste with suction then rinse, evaluate and repeat. (2.2mm has been removed, and repeat as necessary.

**Cleanup**
1) After final rinse, remove rubber dam, or OpalDam/ OpalDam Green, thoroughly.
2) Use the OpalCups Finishing for surface polishing. A fluoride treatment is also recommended.
3) Take final photos for reference.

**Use**
1) For post-operative sensitivity, potassium nitrate or fluoride varnish products for 30 minutes to 8 hours per day are recommended. Areas of deep hypoplasia and erosion should be observed with respect to treatment and may require esthetic bonding restorations. We recommend whitening products to be used prior to Opalustre treatments as this procedure can compromise the aesthetic appearance. If used, wait 7-10 days before any esthetic bonding restorations are placed.

**Storage and Disposal**
Dispose of waste according to local rules, guidelines, and regulations.

**Miscellaneous Information**
Report any serious incident to the manufacturer and the competent authority:
www.ultradent.com / 1-800-552-5512 / 801-572-4200

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**Preparation**
1) It is recommended to take photographs in initial as reference.
2) Using dental dam and clamps, or following instructions for OpalDam/OpalDam Green to isolate the area to be treated along with one tooth adjacent on each side.
3) Remove Luer lock cap from Opalustre syringe and securely attach White Mac™ tip.
4) Verify flow prior to applying intraorally. If resistance is met, replace tip and re-check.

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