## Consepsis™ Scrub

### Chlorhexidine Antibacterial Slurry

Descrizione: Consepsis™ Scrub is a lightly flavored 2.0% (relative to liquid component) chlorhexidine gluconate disinfecting scrub.

Indicazioni for Use: Consepsis Scrub is used for removal of residual temporary cement prior to permanent cementation and/or for cleaning debris while disinfecting.

- Directions for Use:
- For 1,2 pre-filled syringes:
    - Remove Luer Lock cap.
    - Securely attach working tip of choice to the syringe (White Mac™ Tip is recommended).
    - Verify the color and consistency of the product flow before applying by expressing a small amount onto a mixing pad or gauze. Discard any clear liquid that is expressed.
    - Replace the tip with the Luer cap after use.
  - For 30ml IndSpense™ syringe:
    - Remove Luer Lock cap from IndSpense syringe.
    - Attach a 1,2 syringe to the male threads of the IndSpense™ syringe.
    - Depress the IndSpense plunger while guiding the 1,2ml syringe plunger outward until the desired amount has been dispensed.
    - Separate syringes and re-cap IndSpense syringe with Luer cap.
  - Securely attach working tip (White Mac™ Tip is recommended) to the 1,2ml syringe.
  - Direct placement instructions:
    - Express Consepsis Scrub directly to the tooth being treated or into rubber cup.**NOTE:** If tooth is not anesthetized, warm the syringe under hot tap water for approximately 10 seconds before applying.
  - Scrub with the rubber cup or StarBrush™ at a slow speed until preparation is clean.
  - Rinse and dry prepared area.
  - Proceed with restorative placement taking care to avoid saliva contamination.

- Precautions and Warnings:
- For Professional use only. Use only as directed.
  - Read carefully and understand all instructions before using.
  - Keep out of reach of children.
  - Confirm that patient has no known allergies to product ingredients.
  - Re-cap with the Luer Lock cap. To avoid cross contamination, disinfect and wipe syringe with an intermediate level disinfectant between uses or use a disposable syringe cover.
  - Dispose of tip and empty syringe properly. To avoid cross-contamination, tips should not be re-used.
  - Verify flow of all syringes prior to applying intraorally. If resistance is met, replace tip and re-check. Use only recommended tips. To avoid cross contamination, do not re-use tips.
  - Verify the color and consistency of the product flow before applying by expressing a small amount onto a mixing pad or gauze. Discard any clear liquid that is expressed.
  - Store at room temperature.
  - Do not use after expiration date noted on syringe.
- NOTE:** For SDS and additional information about using our products, please go to [www.ultradent.com](http://www.ultradent.com)

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- Precautions and warnings:
- For Professional use only.
  - Read carefully and understand all instructions before using.
  - Keep out of reach of children.
  - Confirm that patient has no known allergies to product ingredients.
  - Re-cap with the Luer Lock cap. To avoid cross contamination, disinfect and wipe syringe with an intermediate level disinfectant between uses or use a disposable syringe cover.
  - Dispose of tip and empty syringe properly. To avoid cross-contamination, tips should not be re-used.
  - Verify flow of all syringes prior to applying intraorally. If resistance is met, replace tip and re-check. Use only recommended tips. To avoid cross contamination, do not re-use tips.
  - Verify the color and consistency of the product flow before applying by expressing a small amount onto a mixing pad or gauze. Discard any clear liquid that is expressed.
  - Store at room temperature.
  - Do not use after expiration date noted on syringe.
- NOTE:** For SDS and additional information about using our products, please go to [www.ultradent.com](http://www.ultradent.com)

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U.S. federal law restricts the sale of this device by or on the order of a dentist.

For immediate reorder and/or complete descriptions of Ultradent's product line, refer to Ultradent's catalog or call Toll Free 1-800-552-5512. Outside U.S. call (801) 572-4200 or visit [www.ultradent.com](http://www.ultradent.com).

## Consepsis™ Scrub

### Chlorhexidine Antibacterial Slurry

Manufactured by:  
Ultradent Products Inc.  
505 West Ultradent Drive (10200 South)  
South Jordan, Utah 84095 USA

For product SDS please  
visit our website:  
[www.ultradent.com](http://www.ultradent.com)

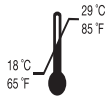
**ULTRADENT**  
PRODUCTS, INC.  
© Copyright 2022 Ultradent Products, Inc. All Rights Reserved. 23989A/11 102422

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