Consepsis™ Scrub

Chlorhexidine Antibacterial Slurry

Descrizione: Consepsis™ Scrub is a lightly flavored 2.0% (relative to liquid component) chlorhexidine gluconate disinfecting scrub.

Indicazioni di uso: Consepsis Scrub is used for removal of residual temporary cement prior to permanent cementation and/or for cleaning debris while disinfecting.

- Directions for Use:
- For 1,2 pre-filled syringes:
 - Remove Luer Lock cap.
 - Securely attach working tip of choice to the syringe (White Mac™ Tip is recommended).
 - Verify the color and consistency of the product flow before applying by expressing a small amount onto a mixing pad or gauze. Discard any clear liquid that is expressed.
 - Replace the tip with the Luer cap after use.

- For 30ml IndSpense syringe:
 - Remove Luer Lock cap from IndSpense syringe.
 - Attach a 1,2 syringe to the male threads of the IndSpense syringe.
 - Depress the IndSpense plunger while guiding the 1,2ml syringe outward until the desired amount has been dispensed.
 - Separate syringes and re-cap IndSpense syringe with Luer cap.
- Securely attach working tip (White Mac™ Tip is recommended) to the 1,2ml syringe.
- Direct placement instructions:
 - Express Consepsis Scrub directly to the tooth being treated or into rubber cup.**NOTE:** If tooth is not anesthetized, warm the syringe under hot tap water for approximately 10 seconds before applying.
- Scrub with the rubber cup or StarBrush™ at a slow speed until preparation is clean.
- Rinse and dry prepared area.
- Proceed with restorative placement taking care to avoid saliva contamination.

- Precautions and Warnings:
- For Professional use only. Use only as directed.
 - Read carefully and understand all instructions before using.
 - Keep out of reach of children.
 - Confirm that patient has no known allergies to product ingredients.
 - Re-cap with the Luer Lock cap. To avoid cross contamination, disinfect and wipe syringe with an intermediate level disinfectant between uses or use a disposable syringe cover.
 - Dispose of tip and empty syringe properly. To avoid cross-contamination, tips should not be re-used.
 - Verify flow of all syringes prior to applying intraorally. If resistance is met, replace tip and re-check. Use only recommended tips. To avoid cross contamination, do not re-use tips.
 - Verify the color and consistency of the product flow before applying by expressing a small amount onto a mixing pad or gauze. Discard any clear liquid that is expressed.
 - Store at room temperature.
 - Do not use after expiration date noted on syringe.
 - NOTE: For SDS and additional information about using our products, please go to www.ultradent.com

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Ultradent syringes have an expiration date stamped on the side of the syringe consisting of one letter and three numbers. The letter is a lot number used for manufacturing purposes and the three numbers are the expiration date. The first two numbers are the month, and the third number is the last number of the year.

U.S. federal law restricts the sale of this device by or on the order of a dentist.

For immediate reorder and/or complete descriptions of Ultradent's product line, refer to Ultradent's catalog or call Toll Free 1-800-552-5512. Outside U.S. call (801) 572-4200 or visit www.ultradent.com.

Consepsis™ Scrub

Chlorhexidine Antibacterial Slurry

Manufactured by:
Ultradent Products Inc.
505 West Ultradent Drive (10200 South)
South Jordan, Utah 84095 USA

For product SDS please
visit our website:
www.ultradent.com

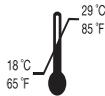
ULTRADENT
PRODUCTS, INC.
© Copyright 2022 Ultradent Products, Inc. All Rights Reserved. 23989A/10 021122

EN- Catalog Number
DE- Katalognummer
FR- Numéro de référence
NL- Catalogusnummer
IT- Numero di catalogo
ES- Número de catálogo
PT- Número do catálogo
SV- Katalognummer
DA- Katalognummer
FI- Luettelonumero
EL- Αριθμός καταλόγου

EN- Dentist Dispensed.
DE- Nach Gebrauch abzugeben.
FR- Distribué par le chirurgien dentiste.
NL- Door de tandarts verstrekt.
IT- Prescritto dal dentista.
ES- Administrado por el Dentista.
PT- Administrado pelo dentista.
SV- Appliceras av tandläkare.
DA- Administreret af tandlæge.
FI- Hammaslääkäarin käyttöön.
EL- Διανέμεται από τον οδοντίατρο.

EN- Destroy after use.
DE- Nach Gebrauch vernichten.
FR- Détruire après utilisation.
NL- Na gebruik vernietigen.
IT- Distruggere dopo l'uso.
ES- Destruir/lo después de usarlo.
PT- Destruir após usar.
SV- Förstöras efter användning.
DA- Destruer efter brug.
FI- Hävittää käytön jälkeen.
EL- Καταστρέψτε το μετά τη χρήση.

EN- Store at room temperature
DE- Bei Raumtemperatur lagern
FR- Entreposer à température ambiante
NL- Bij kamertemperatuur bewaren
IT- Conservare a temperatura ambiente
ES- Almacenar a temperatura ambiente
PT- Armazenar em temperatura ambiente
SV- Förvaras i rumstemperatur
DA- Opbevares ved stuetemperatur
FI- Säilytä huoneenlämmössä
EL- Φυλάσσεται σε θερμοκρασία δωματίου

EN- Use by date
DE- Verfallsdatum
FR- Date de péremption
NL- Uiterste gebruiksdatum
IT- Utilizzare entro
ES- Usar antes de
PT- Usar até a data
SV- Använd före datum
DA- Udløbsdato
FI- Viimeinen käyttöajankohta
EL- Ημερομηνία λήξης

EN- For professional use only
DE- Nur zur Verwendung durch Sachkundige
FR- Destiné à un usage professionnel uniquement
NL- Alleen voor professioneel gebruik
IT- Esclusivamente ad uso professionale
ES- Sólo para uso profesional
PT- Apenas para uso profissional
SV- Enbart för yrkesmässig användning
DA- Kun til professionel brug
FI- Vain ammattikäyttöön
EL- Για επαγγελματική χρήση μόνο

EN- Recycle
DE- Recyceln
FR- Recycler
NL- Recyclen
IT- Riciclare
ES- Reciclar
PT- Reciclar
SV- Återvinna
DA- Genbrug
FI- Kiertää
EL- Ανακυκλώστε το

EN- Corrosive
DE- Ätzend
FR- Corrosif
NL- Corrosief
IT- Corrosivo
ES- Corrosivo
PT- Corrosivo
SV- Svåraretsande
DA- Ætsende
FI- Syövyttävä
EL- Διαβρωτικό

EN- Hazardous to the environment
DE- Umweltgefährlich
FR- Dangereux pour l'environnement
NL- Gevaarlijk voor het milieu
IT- Pericoloso per l'ambiente
ES- Peligroso para el medio ambiente
PT- Riscos ao meio ambiente
SV- Miljöfarlig
DA- Miljøfarlig
FI- Ympäristövaara
EL- Επιβιότονο για το περιβάλλον

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