## Consepsis™ Scrub

### Chlorhexidine Antibacterial Slurry

Description:

Consepsis™ Scrub is a lightly flavored 2.0% (relative to liquid component) chlorhexidine gluconate disinfecting scrub.

Indications for Use:

Consepsis Scrub is used for removal of residual temporary cement prior to permanent cementation and/or for cleaning debris while disinfecting.

Directions for Use:

1. For 1.2 pre-filled syringes:

- Remove Luer Lock cap.
- Securely attach working tip of choice to the syringe (White Mac™ Tip is recommended). Verify the color and consistency of the product flow before applying by expressing a small amount onto a mixing pad or gauze. Discard any clear liquid that is expressed.
- Replace the tip with the Luer cap after use.

2. For 30mL IndiSense syringe:

- Remove Luer Lock cap from IndiSense syringe.
- Attach a 1.2 syringe to the male threads of the IndiSense syringe.
- Depress the IndiSense plunger while guiding the 1.2ml syringe plunger outward until the desired amount has been dispensed.
- Separate syringes and re-cap IndiSense syringe with Luer cap.
- Securely attach the working tip (White Mac™ Tip is recommended) to the 1.2ml syringe.

3. Direct placement instructions:

- Express Consepsis Scrub directly to the tooth being into or rubber cup.
- NOTE:** If tooth is not anestheticized, warm the syringe under hot tap water for approximately 10 seconds before applying.

- Proceed with the rubber cup or StarBrush at a slow speed until preparation is clean.
- Rinse and dry prepared area.

4. Proceed with restorative placement taking care to avoid saliva contamination.

Precautions and Warnings:

- For Professional use only. Use only as directed.
- Carefully read and understand all instructions before using.
- Keep out of reach of children.
- Confirm that patient has no known allergies to product ingredients.
- Re-cap with the Luer Lock cap. To avoid cross contamination, disinfect and wipe syringe with an intermediate level disinfectant between uses or use a disposable syringe cover.
- Discard any tip or unused amount of product. Disinfection, tips should not be re-used.
- Verify flow of all syringes prior to applying intraligament. If resistance is met, replace tip and re-check. Use only recommended tips. To avoid cross contamination, do not re-use tips.
- Verify the color and consistency of the product flow before applying by expressing a small amount onto a mixing pad or gauze. Discard any clear liquid that is expressed.
- Store at room temperature.
- Do not use after expiration date noted on syringe.

**NOTE:** For SDS and additional information about using our products, please go to [www.ultradent.com](#).

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Ultradent syringes have an expiration date stamped on the side of the syringe consisting of one letter and three numbers. The letter is a lot number used for manufacturing purposes and the three numbers are the expiration date. The first two numbers are the month, and the third number is the last number of the year.

U.S. federal law restricts the sale of this device by or on the order of a dentist.

For immediate reorder and/or complete descriptions of Ultradent's product line, refer to Ultradent's catalog or call Toll Free 1-800-552-5512. Outside U.S. call (801) 572-4200 or visit [www.ultradent.com](http://www.ultradent.com).

## Consepsis™ Scrub

### Chlorhexidine Antibacterial Slurry

Manufactured by:  
Ultradent Products Inc.  
505 West Ultradent Drive (10200 South)  
South Jordan, Utah 84095 USA

Made in USA

For product SDS please  
visit our website:  
[www.ultradent.com](http://www.ultradent.com)

© Copyright 2020 Ultradent Products, Inc. All Rights Reserved. 23889-9 082420

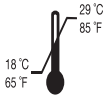
EN- Catalog Number  
DE- Katalognummer  
FR- Numéro de référence  
NL- Catalogusnummer  
IT- Numero di catalogo  
ES- Número de catálogo  
PT- Número do catálogo  
SV- Katalognummer  
DA- Katalognummer  
FI- Luettelo numero  
EL- Αριθμός καταλόγου

EN- Dentist Dispensed.  
DE- Vom Zahnarzt abzugeben.  
FR- Distribué par le chirurgien dentiste.  
NL- Door de tandarts verstrekt.  
IT- Prescritto dal dentista.  
ES- Administrado por el Dentista.  
PT- Administrado pelo dentista.  
SV- Appliceras av tandläkare.  
DA- Administreret af tandlæge.  
FI- Hammassäläkärin käyttöön.  
EL- Διανέμεται από τον οδοντίατρο.

EN- Destroy after use.  
DE- Nach Gebrauch vernichten.  
FR- Détruire après utilisation.  
NL- Na gebruik vernietigen.  
IT- Distruggere dopo l'uso.  
ES- Destruyalo después de usarlo.  
PT- Destruir após usar.  
SV- Förstöras efter användning.  
DA- Destruer efter brug.  
FI- Hävittää käytön jälkeen.  
EL- Καταστρέψτε το μετά τη χρήση.

EN- For professional use only  
DE- Nur zur Verwendung durch Sachkundige  
FR- Destiné à un usage professionnel uniquement  
NL- Alleen voor professioneel gebruik  
IT- Esclusivamente ad uso professionale  
ES- Sólo para uso profesional  
PT- Apenas para uso profissional  
SV- Enbart för yrkesmässig användning  
DA- Kun til professionel brug  
FI- Vain ammattikäyttöön  
EL- Για επαγγελματική χρήση μόνο

EN- Store at room temperature  
DE- Bei Raumtemperatur lagern  
FR- Entreposer à température ambiante  
NL- Bij kamertemperatuur bewaren  
IT- Conservare a temperatura ambiente  
ES- Almacene a temperatura ambiente  
PT- Armazene em temperatura ambiente  
SV- Förvaras i rumstemperatur  
DA- Opbevares ved stuetemperatur  
FI- Säilytä huoneenlämmössä  
EL- Φυλάσσεται σε θερμοκρασία δωματίου

EN- Use by date  
DE- Verfallsdatum  
FR- Date de péremption  
NL- Uiterste gebruiksdatum  
IT- Utilizzare entro  
ES- Usar antes de  
PT- Usar até a data  
SV- Använd före datum  
DA- Udløbsdato  
FI- Viimeinen käyttöpäivä  
EL- Ημερομηνία λήξης

EN- Hazardous to the environment  
DE- Umweltgefährlich  
FR- Dangereux pour l'environnement  
NL- Gevaarlijk voor het milieu  
IT- Pericoloso per l'ambiente  
ES- Peligroso para el medio ambiente  
PT- Riscos ao meio ambiente  
SV- Miljöfarlig  
DA- Miljøfarlig  
FI- Ympäristövaara  
EL- Επικίνδυνο για το περιβάλλον

EN- Recycle  
DE- Recyceln  
FR- Recycler  
NL- Recyclen  
IT- Riciclare  
ES- Reciclar  
PT- Reciclar  
SV- Återvinna  
DA- Genbrug  
FI- Kierrättää  
EL- Ανακυκλώστε το

EN- Corrosive  
FR- Corrosif  
NL- Corrosief  
IT- Corrosivo  
ES- Corrosivo  
PT- Corrosivo  
SV- Frätande  
DA- Ætsende  
FI- Syövyttävä  
EL- Διαβρωτικό

EN- Hazardous to the environment  
DE- Umweltgefährlich  
FR- Dangereux pour l'environnement  
NL- Gevaarlijk voor het milieu  
IT- Pericoloso per l'ambiente  
ES- Peligroso para el medio ambiente  
PT- Riscos ao meio ambiente  
SV- Miljöfarlig  
DA- Miljøfarlig  
FI- Ympäristövaara  
EL- Επικίνδυνο για το περιβάλλον

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