## Cleaning & Sterilization Processing Instructions EN

Initial treatment at the time of use	No preparation or cleaning is required for initial use out of the package. Sterilization is required before every use.								
Cleaning	Place device in an ultrasonic cleaner with the detergent listed below: <table><tbody><tr><td><b>Detergent</b></td><td><b>Cleaning Time</b></td></tr><tr><td>Henry Schein General Purpose Cleaner or equivalent product.</td><td>10 mins</td></tr></tbody></table>	<b>Detergent</b>	<b>Cleaning Time</b>	Henry Schein General Purpose Cleaner or equivalent product.	10 mins				
<b>Detergent</b>	<b>Cleaning Time</b>								
Henry Schein General Purpose Cleaner or equivalent product.	10 mins								
Rinsing	Rinse device with warm water to remove detergent for 2 minutes.								
Drying	Dry with gauze. Air dry 30 mins.								
Maintenance, inspection and testing	Visually inspect that the device is not bent or damaged and free of debris. If debris is present repeat cleaning process.								
Packaging	Place selected device in approved sterilization packaging (ISO11407 & ISO11607).								
Sterilization	Place sterilization package in autoclave using the following settings: <table><tbody><tr><th>Cycle Type</th><th>Minimum Sterilization Exposure Time</th><th>Minimum Sterilization Exposure Temperature</th><th>Minimum Dry Time</th></tr><tr><td>Pre-vacuum</td><td>3 mins</td><td>132°C /270°F</td><td>20 mins</td></tr></tbody></table> <p>Do not overload the sterilizer.</p>	Cycle Type	Minimum Sterilization Exposure Time	Minimum Sterilization Exposure Temperature	Minimum Dry Time	Pre-vacuum	3 mins	132°C /270°F	20 mins
Cycle Type	Minimum Sterilization Exposure Time	Minimum Sterilization Exposure Temperature	Minimum Dry Time						
Pre-vacuum	3 mins	132°C /270°F	20 mins						
Storage	Keep device in sterilized packaging until ready for use. Keep out of sunlight and moisture.								
Additional information	If sterilization packaging is compromised, repeat sterilization process. This procedure was validated by an independent and accredited laboratory.								

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© 2020 Ultradent Products, Inc. All Rights Reserved. 19887777 05/20
Ultradent Products, Inc.
Ultradent Australia Pty Ltd