

Colour determination sheet. For silicone products.

This Colour Determination Sheet offers a standardized approach for assessing the patient's skin colour. On the coloured background provided, enter the personal data, take digital photos of the hands and feet (palmar/plantar and dorsal views), and email them to Ottobock. We will take care of the rest.

Preparation

Guidelines

- A place with natural light and at room temperature
- Water-based cream (Ottobock's Procomfort Gel 633S2 recommended) to relieve dry skin
- Patient should have clean nails (no polish) filed to the desired length
- Use a neutral grey background (on the reverse of this page)
- No flash and no zoom
- Clear and close-up images from various angles (including close-up images of the nails)

Patient

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
- No black or white clothing should be worn
- Patient should be warm, with a stable heart rate
- Patient should sit facing the window (or other light source)
- Avoid shadows on hands/feet
- Avoid exposing the hands/feet to direct sunlight
- Position the hands/feet on the background provided
- Ensure that at least one colour scale remains visible and free of shadow along its whole length
- Place the hands/feet in relaxed position on the photo background

For Best Results

- Please enter the patient data in the fields provided
- Use colour swatches if available

Email

- Import the photos into the computer and check for accuracy
- Email photos to iFabBUR@ottobock.com



Patient name: _____

Customer no.: _____

Company: _____

Contact Person: _____

Date: _____

Email photos to iFabBUR@ottobock.com

ottobock.