

Skeo Up Daily Assist. 14Y4=*.

The *Skeo Up Daily Assist* 14Y4 is a passive waterproof functional device made of silicone. It is worn like a liner and allows the user to hold objects. The silicone contour corresponds to the four fingers, while the inserted plastic *clamping adapter* 9S405=1 (always included in 14Y4=*) allows objects up to 4 cm thick (e.g., toothbrushes, razors) to be clamped. The functional aid is particularly suitable for use in wet areas, such as for personal hygiene.

Quantity	Naming	Article number
1	IFU	-
1	Skeo Up Daily Assist	-
1	Clamping tool/adapter	9S405=1

Available Sizes

The product is available in the following sizes (residual limb circumference in mm):
140, 160, 180, 200, 220

Indications

- Recommended for residual limb lengths of longer than 10 cm
- Transradial amputation or dysmelia
- Unilateral or bilateral amputation

Contraindications

- Osseointegration
- Skin diseases, skin injuries, inflammation, raised scars with swelling, redness and overheating in the treated area of the body


Important Instructions For Use

This product is intended for use in wet areas for a daily usage period of 2 hours.

Please note that this product is not intended for use in compression therapy.

First Use

The product should be worn for a maximum of 30 minutes. Following this, the residual limb should be inspected for any constrictions or marks. If no issues are identified, the wearing time can be increased with each subsequent use.

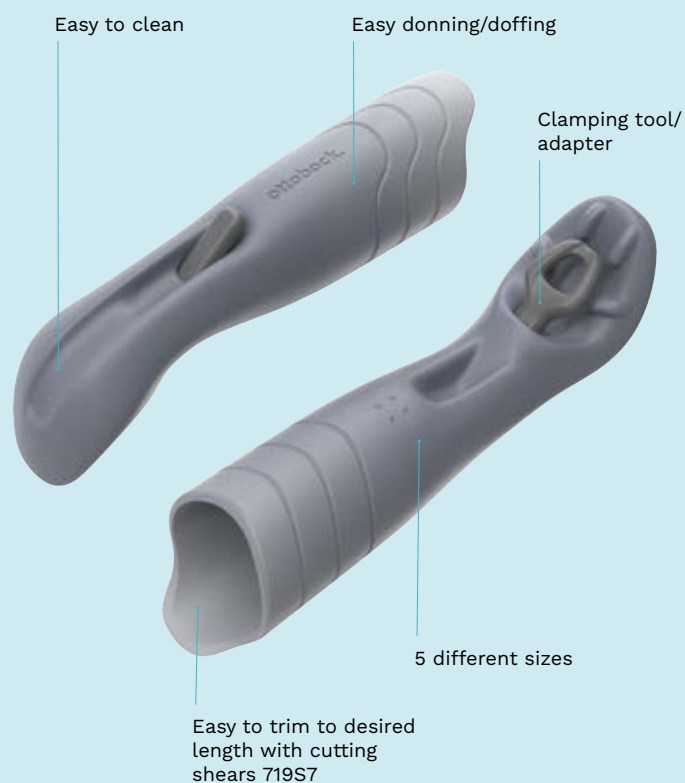

Skeo Up Daily Assist.

14Y4=*



Put on

Turn the silicone material inside out or roll it up, place it on the end of the residual limb and align it. Roll over the residual limb without soft tissue displacement or air pockets. Briefly lift the proximal edge to relieve tension in the skin.



Take Off

Turn the silicone material inside out and remove the functional aid from the residual limb. To do this, pull carefully on the flap.



Waterproof and easy to clean

The product is particularly suitable for use in wet areas, such as for personal hygiene. To clean the product, we recommend washing the inside and outside by hand with warm water and a neutral soap. Any soap residue should then be rinsed off thoroughly with clean water. A towel can be placed inside the functional aid to dry it, which should then be laid flat to dry on a clothes horse.



Clamping tool/adapter

The clamping tool is designed with a distinctive shape and is sufficiently attached to the silicone of the product. It can be easily removed for cleaning or replacement.



Trimming to Length

To trim the product to the desired length, use the 719S7 cutting shears. The product's lines can be used as a contour. Ensure that the cut edge is smooth to prevent tearing.

Examples of use:



Clamp objects in the hand
e.g. toothbrush, razor



Place object in hand
e.g. cream jar



Support with bilateral
activities e.g. clamping
a magazine