

Evanto Cleaning Instructions.

Common Cause of Noise: Sand/Dust

To reduce or eliminate noise:

Shell & Sock

- Remove the shell, rinse thoroughly with water, and dry.
- Replace the Spectra sock (2U3=2-7).
- If reusing the old sock: machine wash and let it dry completely.

Strap

- Fold up and rinse under running water.
- Squeeze, fold, and rub to remove dust from textile fibers.

Foam & Base Spring

- Clean the foam pocket and strap groove with a damp cloth.
- Wipe the lower foam with a damp cloth.

Lubrication (if noise persists)

- Use lubricants like:
 - Polylub (633F30)
 - Molykote (633F14)
 - 633G6 lubricant
- Apply to:
 - Lower edge of the lower foam
 - Strap groove and side of the strap
 - Spectra sock (when assembled) on the outside of the COP-K and strap side

Important Notes

- Use only water and soap for cleaning.
- Avoid acetone, talcum powder, and silicone spray:
 - Talcum may worsen noise around the bearing and ball of the foot.
 - Silicone spray has been linked to increased noise.
- Use minimal lubricant to avoid attracting more dust.

