


iFab custom silicone photo taking instructions.

Guidelines


- A place with natural light and at room temperature
- Water-based cream (Ottobock's Procomfort Gel 633S2 recommended) to relieve dry skin
- Patient should have clean nails (no polish) filed to the desired length
- Hands/feet should be in a relaxed position
- Use a neutral grey background (if available, use the grey background in the Colour Determination Loaner Kit)
- No flash and no zoom
- Clear and close-up images from various angles (listed below)

Required photos for initial quoting and casting stages:


Dorsal

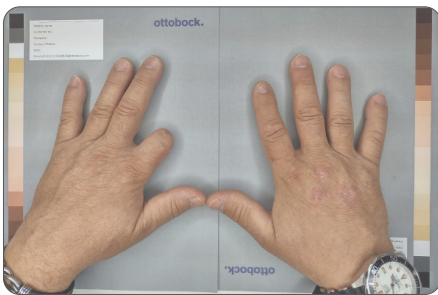

Palmar

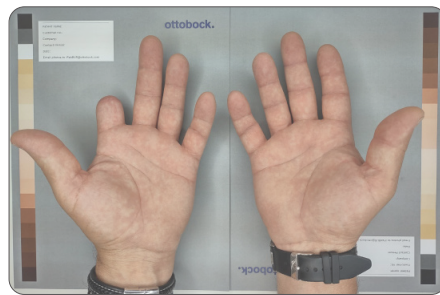

Medial

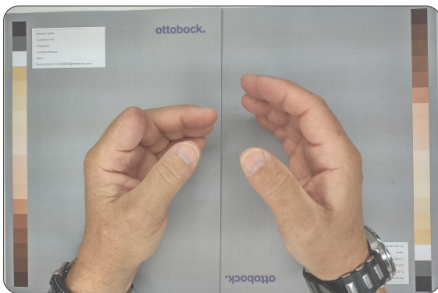

Lateral



Close Up:



Nails without polish, filed to desired length.

Photo & Testimonial Release

Scan the QR code to complete the Photo & Testimonial Release




Patient's consent to send and use photos must be provided.

Send photos to iFabBUR@ottobock.com

During reassessment take photos with chosen colour swatches (layering for colour accuracy) and also with the trial device:


Base

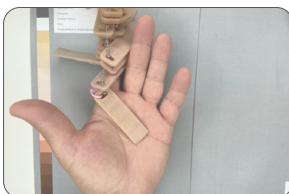

Fingertip



Palmar

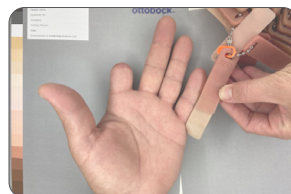

Trial



Tan



MCP/PIP/DIP



Nail

