

# Academy Tech Tips

## Power flush for P4 Harmony modular pumps 4R182/4R183

The Tech Bulletin series is designed to quickly and easily provide useful information regarding Ottobock products and services. In this issue, we will focus on Harmony 4R182/4R183.



When doing a power flush please follow the instructions below:

### **Needed Flexible 1/8 ID Tubing**

Step 1. Remove socket and foot from pump to prevent possible damage to foot . Step 2. Insert tubing to barb on the pump Fig 1. The tubing should be dropped into a hot cup of water Fig 2. Step 3. Place pump on towel. Step 4. Using the palm of your hand, compress the pump up and down until water is completely flushed though the pump Fig 3. Check vacuum with gauge, repeat flush if necessary. (vacuum gauge kit 4R131)



Fig 1.



Fig 2.



Fig 3.

Last step Install the foot and socket as described in the instructions for use.

If you have any further questions, please contact Ottobock at 800 328 4058 and ask to speak with a member of our Professional and Clinical Services department.