

Care and cleaning instructions. Custom silicone covers.

Daily cleaning

- Silicone hands and fingers must be cleaned daily to avoid skin irritation and to prolong the life of the glove
- Use only a mild soap such as regular dishwashing soap
- NO antibacterial soap should be used
- Using a soft bristled brush, thoroughly wash and rinse both the inside and outside of the prosthesis
- Let air dry or use a lint free towel

Thorough cleaning

- The glove should be thoroughly cleaned once every month or immediately if the prosthesis has been in contact with a contaminant such as chlorine or petroleum-based products
- To thoroughly clean, apply some Procomfort Gel (633S2) on the widest portion of the hand and then reverse the glove to remove it from the prosthesis. Using a soft bristled toothbrush, thoroughly wash and rinse the inside of the silicone glove. It could also be boiled with a few drops of soap in a pot with a face cloth between the pot and the glove
- If using other creams, ensure they are water-based creams and there is very little alcohol and no petroleum products (i.e. Vaseline) present as they can cause permanent stretching and degradation of the silicone

Stain prevention

- Avoid contact with permanent inks
- If a mark does occur, wash immediately with ONLY soap and water and a soft bristled brush (this does not guarantee removal of the stain)
- **WARNING: The dye may rub off of dark coloured clothing and stain the silicone, please ensure that the garment is colour-fast**

Nail polish

- Water-based polish can be used on the acrylic nails
- When removing the polish, use only acetone-free nail polish remover

If you require any additional information please contact us:

Toll Free: 1.800.665.3327

E-mail: iFabBUR@ottobock.com