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Cleaning and Care

Your prosthesis requires regular care.

For daily care of the inner surfaces, cleaning with water and soap is sufficient to reliably remove any sweat residue.

To clean the outer surfaces you may leave the prosthesis still donned while performing daily handwashing routine.

It is important to thoroughly dry all cleaned surfaces and to make sure that no soap residue remains on the product since this could cause skin irritation on your residual limb.

To remove tough stains and grease, we recommend boiling the prosthesis in pure tap water for 5 – 10 minutes once every 2 weeks.

If you have any questions on skin diseases and disinfection, please consult your physician or prosthetist.

⚠ Damage caused by improper cleaning agents.

To prevent material damage, do not use any solvents such as thinner, acetone, benzene, etc.

⚠ Damage due to inadequate cleaning.

Daily cleaning with a neutral soap will remove sweat components such as skin grease, amino acids and sugar from your silicone prosthesis. Without such daily cleaning, your silicone prosthesis might suffer long-term damage.

⚠ Damage due to substances with a strong dyeing effect.

Avoid any contact with substances that have a strong dyeing effect (ink, dyes, etc.). They can penetrate the silicone material and lead to permanent discoloration.

Your Feedback

Thank you for your trust and confidence in Ottobock. Your opinion is the basis for our quality improvement.

Contact us:

Our clinic:

Bangkok (Head Office)

6 Soi Khlang Montri, Chatuchark,
Chatuchark, Bangkok TH 10900
Tel. 02-587-8799

Suratthani

189/4-5 Moo 1, Tumbon
Wadprudu, Amphur Mueng,
Suratthani 84000
Tel. 077-945-185, 061-178-4666

Khonkaen

272/16 Moo 12, Tumbon Mueng
Kao, Amphur Mueng, Khonkaen
TH 40000
Tel. 043-472-509


2021 © Ottobock · SEA · EN · OK02801 · IFU

ottobock.

Instruction for Use

Partial Hand and Finger Silicone Prosthesis

Custom-made device

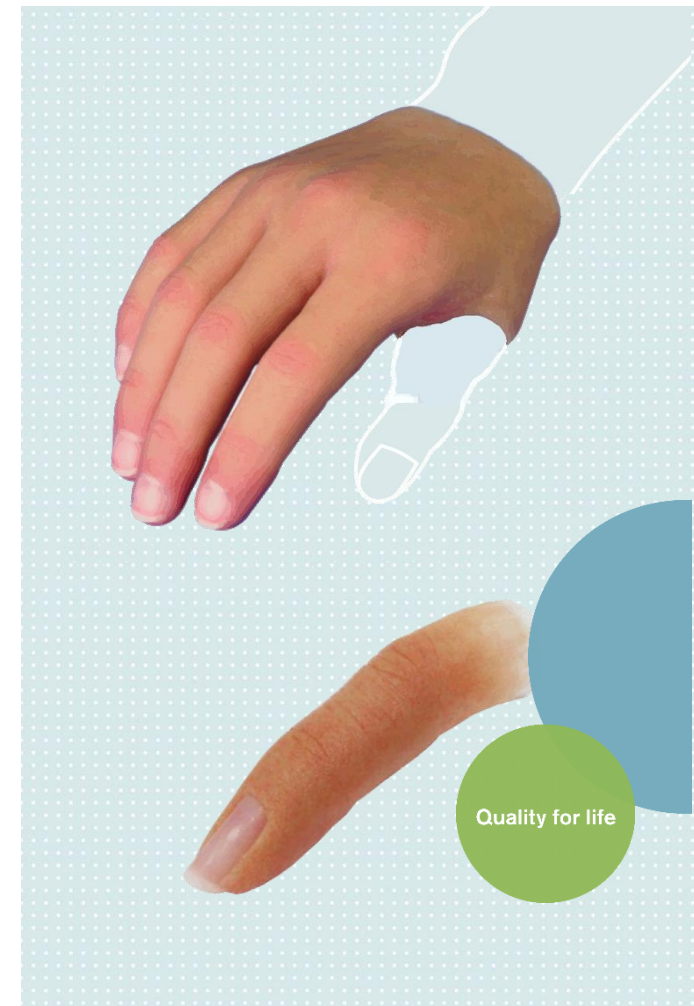You have received a custom made silicone prosthesis, which has been adapted to your personal needs. Your prosthetist has informed you about the proper use of your prosthesis. Please read the following instructions for use carefully. They contain significant additional information regarding the proper use of your prosthesis as well as important safety instructions.

Description of Design

The prosthesis consists of a socket to embed your residual limb in and a passively functional replacement compensating for the missing body area. The silicone material used is skin-friendly and physiologically safe.

Areas of application and Usage Restrictions

Your finger and partial hand prosthesis serves as a functional replacement for the affected body area and for restoring the outer appearance. It will help you to grasp and hold objects.

Your prosthesis should generally present no discomfort and should be worn during the day. Excessive pressure can lead to complications.

If you have any questions or concerns about your prosthesis, please do not hesitate to contact your certified prosthetist.

Volume Fluctuations

As a prosthesis wearer, you should pay special attention to your weight since any fluctuations will have a direct impact on the fit of your prosthesis. Minor volume fluctuations over the course of the day are normal. If you have additional questions regarding volume fluctuations, please consult your physician or certified prosthetist.

Guideline for first time user

Please verify that your prosthesis is undamaged and functional prior to each use.

First-time users in particular have to learn how to handle the prosthesis. Your physician or prosthetist will provide you with more detailed information. Special attention was paid to the materials used. They were tested for skin tolerance. Nevertheless, should skin changes develop on your residual limb, consult your physician immediately.

Risk of skin damage on the residual limb.

Sensitivity problems and skin damage require special attention in regard to the fit of the prosthesis. Examining your residual limb visually and by palpation every day is recommended. Please consult your physician or certified prosthetist in case of any signs of skin damage, even if they are only minor.

Donning the Prosthesis

- 1) Because of the high surface friction of the silicone material, it may be necessary to make the residual limb more slippery with a neutral, waterbased and perfume-free skin lotion. For this purpose, apply an additional bit of Procomfort Gel to the inner brim of the prosthesis.
- 2) Slide your residual limb into the access opening with only light pressure.
- 3) Slowly continue to slide your residual limb into the finger and partial hand prosthesis.
- 4) Verify the proper fit.
- 5) Press or massage out any air bubbles between the finger and partial hand prosthesis and residual limb.

Doffing the Prosthesis

To remove the finger and partial hand prosthesis, proceed in the reverse sequence. Let air between the silicone material and skin by forming an air channel. It will then be easy to pull off the prosthesis.

Notice:

Damage due to finger-nails. Sharp-edged finger nails can damage the thin brim of the prosthesis when donning and doffing it.

Risk of skin irritation due to air pockets.

Air bubbles between the silicone material and skin prevent breathability of the silicone. Evaporated sweat can penetrate the gas permeable silicone material only if the silicone lies directly on the skin. Otherwise, the sweat will condense, accumulate as liquid in the silicone product and can cause skin irritation.