



Care and cleaning instructions.

Custom silicone myoelectric.

Daily cleaning

- A silicone liner must be cleaned daily to avoid skin irritation and to prolong the life of the liner
- Avoid using alcohol or acetone based cleaners such as antibacterial soaps
- Use non-alcoholic wipes (i.e. baby wipes) to clean the liner
- Avoid contact with water while the electronic components are attached
- If surfaces get wet, use a lint free towel and dry immediately

Other important information

- For easier donning, a small amount of Procomfort Gel (633S2) can be applied to the limb. Procomfort Gel is a water-based Ottobock product that has been tested on silicone
- Use water-based creams and lotions on your skin only. Avoid using oil-based products such as Vaseline - such products can cause stretching and affect the fit and durability of the liner
- Avoid contact with sharp objects (i.e. fingernails) and abrasive surfaces
- If repairs or changes are required, contact your clinician
- Proper care and cleaning will extend the life of your custom silicone liner

If you require any additional information please contact us:

Toll Free: 1.800.665.3327

E-mail: iFabBUR@ottobock.com