



Custom silicone *Myoelectric* Liners.

Care and cleaning instructions.

Daily cleaning

- A silicone liner must be cleaned daily to avoid skin irritation and to prolong the life of the liner
- Avoid using alcohol or acetone based cleaners such as antibacterial soaps
- Use non-alcoholic wipes (i.e. baby wipes) to clean the liner
- Avoid contact with water while the electronic components are attached
- If surfaces get wet, use a lint free towel and dry immediately

Other important information

- For easier donning, a small amount of Procomfort Gel can be applied to the limb. Procomfort Gel is a water-based Ottobock product that has been tested on silicone.
- Use water-based creams and lotions on your skin only. Avoid using oil-based products such as Vaseline - such products can cause stretching and affect the fit and durability of the liner
- Avoid contact with sharp objects (i.e. fingernails) and abrasive surfaces
- If repairs or changes are required, contact your clinician
- Proper care and cleaning will extend the life of your custom silicone liner

If you require any additional information please contact us using the information at the bottom of the page.