

Maverick junior.

A smile with every step.

The F24 *Maverick junior* was designed for active children and adolescents who navigate varied indoor and outdoor environments and place a high value on flexibility and durability.

Designed for optimal outcomes

- The fibreglass design delivers a high degree of durability and flexibility with the expected energy return
- A split toe was deliberately omitted, the rounded toe element offers users increased robustness
- Low build height
- The connection to additional prosthetic components can be made via a pyramid adapter or a lamination adapter
- The integrated heel wedge with optional insert makes it possible to customise the shock absorption at heel strike
- The sole of the footshell features a smiley face as well as a texture for walking barefoot
- Easy cleaning
- Protected against fresh, salt and chlorinated water

More than a foot. A foundation.



Technical data

For	Children, adolescents and adults with small feet
Max. body weight	45 kg
Sizes	16 – 21 cm
Weight without footshell*	210 g
Footshell shape	Slim shape for a heel height of 5 mm
Footshell colour	beige (4), light brown (15)
Weight*	352 g
Build height*	47 mm

* Reference size is 21 cm.



Two different adapter options available

Made of fibreglass

Low build height

F24. Order form.

Customer

Customer no.	
Company	
Street	
Postal code	City
O&P professional	

Shipping address (if different from customer address)

Customer no.	
Company	
Street	
Postal code	City
Customer ref.	

Selection of the spring stiffness relative to body weight and foot size

Foot size \ Body weight	16 cm	17 cm	18 cm	19 cm	20 cm	21 cm
9 – 15 kg	1	1	1	1	–	–
16 – 24 kg	2	2	2	2	2	–
25 – 34 kg	–	3	3	3	3	3
35 – 45 kg	–	–	4*	4*	4*	4*

* Stiffness 4 is not available for the Maverick junior with lamination adapter.

Example for ordering F24=L-16-2-P/4N

Reference no.	Side	Size	Stiffness	Connection	Colour	Shape
F24	=	-	-	-	/	N

Side	Size (cm)	Stiffness	Connection	Colour
L Left	16, 17, ..., 21	1, 2, 3, 4	P Pyramid adapter	4 Beige
R Right			W Lamination adapter	15 Light brown



Scope of delivery

The scope of delivery includes the F24 foot module, the 2C24 footshell, the 2F14 heel wedge insert, the 2C101 footshell removal tool, a black Spectra sock as well as a quick guide for users and their families on how to remove the footshell for easy cleaning.

In case you ordered the F24 foot module with lamination adapter, the 2R89=1 lamination anchor and the 601T24=* cap screw are also included in the scope of delivery.

Recommended combinations **

TT	TF
<p>Children modular system 4R11=* Quickchange 2R41=1 4R66 4R110 4R60</p> <p>Adult modular system (waterproof) 4R11=* Quickchange 2R37 with 4X907 2R38 with 4X907</p>	<p>3R67 3R68 movido</p> <p>4R11=* Quickchange 2R41=1 4R66 4R110 4R60</p>

** All components are sold separately and are available Ottobock products that are compatible with the F24 *Maverick junior*, which helps ensure optimal performance. Practitioners need to select components based upon individual patient criteria.

The *Maverick junior* prosthetic foot is also approved for combination with *Genium X4*, *C-Leg* and *Kenevo*. The selection of the additional prosthetic parts must be matched to the knee joints with a kids to adult adapter by the technician.