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Blood Sample Collection Instructions (ADX-2 strip)

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The **Blood Sample Collection** should contain these items:

ADX blood collection card

Alcohol pads

Sterile gauze pads

Single use lancets

Adhesive bandages

Blood sample return pouch

Take the ADX blood collection card and **write your first and last name, date of birth, and collection date in MM/DD/YYYY format** at the bottom of the card where prompted. Then follow sample collection instructions below.

Collect your sample. Find instructional video at the QR code below:

Warm up, prep your test, and wash your hands

Get your blood flowing by drinking plenty of water beforehand! Do a set of jumping jacks, or swing your arms by your side. Place the collection card on a clean and dry surface, making sure to only touch the edges of the card. Then give your hands a good wash with warm water.


Clean your fingertip and prick with lancet

Clean your fingertip with the alcohol pad. Try using the middle or ring finger of your non-dominant hand. Then, take a lancet and twist the cap off. Press the small tip into the side of your finger near the top, until the needle ejects with a click. Keep your hand below your heart to help the blood flow.

Fill collection card past the first line

Wipe off the first drop of blood with a gauze pad. Place 4-7 large drops of blood in the top left square of the card, or as many drops as it takes for the blood to migrate past the first line. If the blood does not pass the first line, our lab will not be able to analyze your sample. If you weren't able to fill past the line with one prick, repeat the steps above with a new finger and extra lancet. Don't add additional blood to the square once they have dried--this will invalidate the test!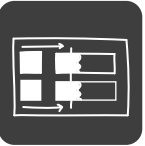

Repeat the steps above for the second strip. Once both strips have been filled, put a bandaid on your finger. Let the card air dry for one hour.

Package sample

Tear open the blood sample return pouch provided. Do not remove any of the contents inside. Place the dried card in the bag and seal. Congrats, you're done! Keep the sample out of extreme temperatures (above 100F or below 32F) before you ship.

Ship your sample and view results in your secure online portal.

Once you have completed collection, simply place the pouch with your sample back into the kit box. Then place the box back into the prepaid envelope and ship. See local regulations for instructions around proper disposal of lancets (or "sharps").