

Whitening gel that easily and efficiently  
removes stains.

Give yourself 16 minutes a day to relax and  
brighten your smile. No pain, no mess,  
no hassle.

ENJOYING BRIGHTBYTE PRO?  
SHOW US SOME LOVE

@BYTEOFFICIAL\_AU



QUESTIONS? CALL 1800 298 363 OR GO TO  
[PATIENT.BYTE.COM/SUPPORT](https://patient.byte.com/support)

# BrightByte PRO



AT-HOME WHITENING KIT WITH  
LED ACCELERATOR LIGHT

## HOW TO USE

### STEP 01

Rinse the BrightByte™ Pro mouth tray and your mouth before use. Dry your teeth to prevent saliva from diluting the gel adherence. Look at yourself in the mirror and compare your teeth to the shade guide (included), then record what stage you're on. Twist open the whitening pen until the gel flows onto the brush. Apply approx. 0.5ml to 0.7ml (about a pea-sized amount) evenly to all teeth.



### STEP 02

Connect the smart whitening device to your mobile phone, put the mouth tray into your mouth and gently bite down.



### STEP 03

Whiten for up to 16 minutes. When time is up, remove the mouth tray. Rinse the tray and your mouth with water. Compare your teeth with the shade guide again and record your results. Do not eat or drink anything, except for water, for 30 minutes after completing your whitening treatment.



## SAFETY DIRECTIONS

Avoid contact with eyes, skin and wash hands after use.

## CAUTION

- **IRRITANT.** Do not apply the gel directly on your gums or on your inside lips, as this may create discomfort and irritation in the applied area. If you experience discomfort, rinse with water.
- Not suitable for veneers, ceramic teeth, or other dental restorations.
- Not suitable for severe tetracycline, decayed teeth, defective enamel, exerted dentin, damaged teeth, or teeth discoloration caused by lesion or medication.
- Not suitable for children under 16 years old or anyone that is pregnant (or nursing).
- Shelf life is approximately 12 months. Do not use after expiration date.
- Place cap back after use. Avoid sunlight; store in a cool, dry place.

## FIRST AID

If in eyes, hold eyelids apart and flush the eye continuously with running water. Continue flushing until advised to stop by a Poisons Information Centre. If swallowed, do NOT induce vomiting. If skin or hair contact occurs, remove contaminated clothing and flush skin and hair with running water.

For advice, contact a Poisons Information Centre (e.g. phone Australia 13 11 26; New Zealand 0800 764 766) or a doctor (at once).

## TIPS FOR BEST RESULTS

- Run your mouthpiece under warm water to clean before and after use.
- If whitening gel gets on your clothes or face, wash it off immediately.
- The shade guide allows you to track your whitening progress. Use it to see how much your teeth have changed until you're happy with your whitening results!

## INGREDIENTS

### WHITENING GEL

UREA PEROXIDE, GLYCERIN, PROPYLENE GLYCOL, AQUA, CARBOMER, CELLULOSE GUM, PVP, MENTHOL, SODIUM HYDROXIDE.



PRT.BBPI.AU-001

DISTRIBUTED BY BYTE AU PTY LTD  
8/226 CONDOMINE STREET  
MANLY VALE NSW 2093