

BrightByte Pro brings dentist-quality whitening to your home. Our exclusive LED mouth tray combines cutting-edge cold light technology with FDA-approved whitening gel to easily and efficiently remove even your deepest stains.

Give yourself 10 minutes a day to relax and brighten your smile. No pain, no mess, no hassle.

ENJOYING BRIGHTBYTE PRO?
SHOW US SOME LOVE.

[@BYTEOFFICIAL](#)



BrightByte PRO



PROFESSIONAL AT-HOME WHITENING KIT WITH
LED ACCELERATOR LIGHT

HOW TO USE

STEP 01

Rinse mouthpiece before use. Dry your teeth. Twist whitening pen to release gel onto the brush and apply evenly to all teeth. Keep your gums and lips away from the gel.



STEP 02

Connect the mouthpiece to your cell phone and place into your mouth. When the light shines, you're ready to bite down tight.



STEP 03

Whiten for 10-16 minutes. When time is up, take out the mouthpiece, rinse your teeth off with warm water, and use your shade guide to see your whitening progress. While whitening, you may experience some sensitivity. If it's severe, reduce your whitening time or stop immediately and apply desensitizing gel.



STEP 04 (OPTIONAL)

If you're feeling any sensitivity on your teeth, disperse a small amount of desensitizing gel onto the brush top and apply a thin layer directly to any affected teeth. This gel is not only scientifically engineered to reduce tooth sensitivity, it also prevents re-staining!



CAUTION

- Not suitable for ceramic or false teeth.
- Not suitable for teeth discoloration caused by lesion or medication.
- Not suitable for severe tetracycline decayed teeth.
- Not suitable for children under 12 years or pregnant women.
- Shelf life is approximately 12 months. Do not use after a 12 month period.
- Secure the caps back onto the pens tightly after use so ingredients do not expire.
- Avoid sunlight; store in a cool, dry place.

INGREDIENTS

WHITENING GEL

Urea peroxide, Glycerin, Propylene Glycol, Aqua, Carbomer, Cellulose Gum, PVP, Menthol, Sodium Hydroxide.

DESENSITIZING GEL

Propylene Glycol, Glycerin, Aqua, Potassium Nitrate, Carbomer, Cellulose Gum, PVP, Menthol, CI 42090, Sodium Hydroxide.

TIPS FOR BEST RESULTS

- Run your mouthpiece under warm water to clean before and after use.
- Everyone's teeth are different. Some people may experience slight sensitivity - this is normal when you're improving your smile. Apply the desensitizing gel to put the brakes on any discomfort. If it feels severe, reduce or stop your treatment time.
- If whitening gel gets on your clothes or face, wash it off immediately.
- The shade guide allows you to track your whitening progress. Use it to see how much your teeth have changed until you're happy with your whitening results!

