

QUICK INSTRUCTIONS FOR LABORATORIES

Drawing AlloSure Specimens

1 COMPLETE LABELS

- A On the Tube & Accession Labels card, complete the two tube labels completely. First and last name, DOB, phlebotomist initials and draw date MUST be completed.

2 DRAW BLOOD

- A Check the expiration dates on both Streck tubes.
- B ONE Streck tube needs to be completely full. Fill the 2nd tube as much as possible. Follow the draw order shown at bottom of page.*
NOTE: for smaller pediatric patients only 5-6mL, or half of one tube, is needed.
- C Mix by gentle inversion.
- D Place a completed tube label lengthwise on each tube; do not cover expiration date.
NOTE: if using lab labels add small accession ID under lab label

3 PREPARE ORDER FORM





- A From the Tube & Accession Labels card, place the first white barcode label on the upper right corner of the order form.
- B Whatever type of order is used, please include:
 - patient's first and last name
 - date of birth
 - transplant center
 - draw date
 - phlebotomist initials

4 PACKING THE SPECIMENS

- A Place the tubes in the absorbent pouch and place pouch in the specimen bag and seal.
- B Wrap the specimen bag in the gel pack.
- C Place gel pack in foil pouch and seal.
- D Place pouch in kit box. Place the order form and Tube & Accession Labels card on top of pouch. Place List of Contents card on top.
- E Close the kit box.
- F Place the box inside the labeled shipping courier bag. Seal and send.
- G Specimens should be shipped same day. Specimen must arrive at CareDx within seven days of draw.

5 STORAGE & SHIPMENT

- A Store specimens at room temperature, between 6° and 37° C until shipping
- B Contact shipping courier for same day pickup or drop the specimens at the nearest shipping courier facility.

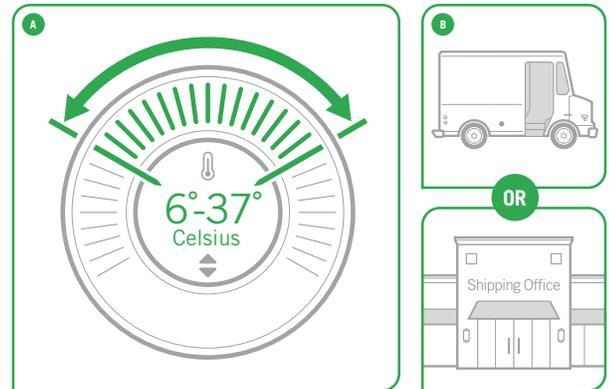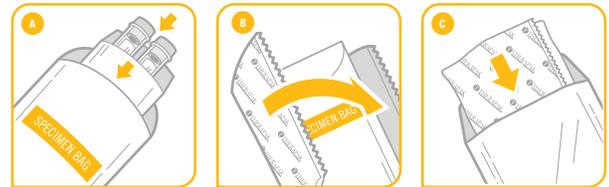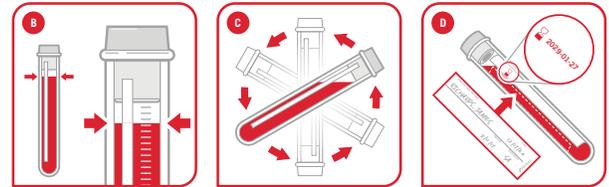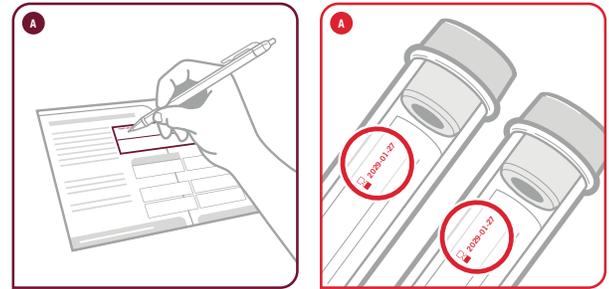

*DRAW ORDER

This is the draw order to be followed for AlloSure testing.





**AlloSure Draw
Instructions
for Adult and
Pediatric Patients**