

Fixodent

PROFESSIONAL


4 EASY STEPS



CLEAN & DRY

1

Clean and dry your denture before you apply the adhesive cream.

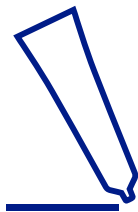

APPLY

2

Apply adhesive in thin strips as shown or less. You might need a few tries to find out the ideal amount for your individual denture.



Upper



Lower



Partial

Maximum size of each strip

4x16.5mm



INSERT & HOLD

3

Insert denture and hold it briefly in place. If product oozes off denture in mouth, you are using too much.



RINSE & REMOVE

4

To remove your denture, rinse your mouth with warm water and rock denture gently until loose.

ADHESIVE CREAM for full and partial dentures.

IMPORTANT:

- ✘ • DO NOT use more product than shown.
- ✘ • DO NOT use product more than once a day. With proper use, this tube should last at least 4 to 5 weeks.
- ✘ • DO NOT use excess product for poorly fitting dentures.
- ✘ • DO NOT use more than directed. Contains zinc, consult a doctor if using other products containing zinc.

⚠ WARNING: Keep out of reach of children. Do not use in the case of hypersensitivity to ingredients. In the unlikely event of a serious suspected problem with this product consult your dentist or doctor and report this to the manufacturer and national health authority.

Keep nozzle and cap dry to avoid clogging. If tip becomes clogged, use a pin to reopen.

DUAL POWER INGREDIENTS: Calcium/Zinc PVM/MA Copolymer (35%), Paraffinum Liquidum, Cellulose Gum (20%), Petrolatum, Silica, CI 45410.

Doesn't feel right? Please consult your dentist regularly to ensure you have properly fitting dentures. Poorly fitting dentures may impair your health.