

**Must be applied with care to the following patients:**

- Pregnant women in whom it risks inducing labor
- Patients with malignant tumors (when it risks aggravating symptoms).
- Patients with hemorrhagic disease.

**Follow the instructions below. Failure to do so may result in injury.**

1. Spinex needles are intended for use by medical practitioners only, including doctors and acupuncturists.
2. Spinex needles are sterile and disposable for one time use only.
3. Do not use if blister packaging is damaged, discolored or if the paper seal is broken. Individualized packaging assures sterile needles.
4. Always check needle expiration date. Do not use expired needles.
5. If the treatment area becomes red or itchy, or displays other abnormal reactions, advise the patient to remove the needle and consult the acupuncture practitioner.
6. Caution must be taken when using needles on patients with stainless steel allergies.
7. Do not use for electrical stimulation, as these are not electroacupuncture needles.
8. Advise your patients to refrain from intense movements during insertion of the acupuncture needles.
9. Advise your patients that treated body parts should not be affected by strong impacts or friction.
10. Advise patient to remove Spinex needles before bathing.
11. Store needles in a safe place away from children, chemicals, humidity and extreme temperatures.



Manufacturer :  
**SEIRIN CORPORATION**  
1007-1 Sodeshi-cho, Shimizu-ku,  
Shizuoka-shi, Shizuoka, Japan  
<http://www.seirin.jp>

Dist by : SEIRIN-America, Inc.  
230 Libbey Parkway Weymouth,  
MA 02189 U.S.A.  
<http://www.seirinamerica.com>



Emergo Europe B.V.  
Prinsessegracht 20,  
2514 AP The Hague,  
The Netherlands

**STERILE EO**