



Using the CARES Framework

A Clinical Guide to Advanced Diabetes
Devices and Closed-Loop Systems



What is CARES?

The **CARES framework** was designed by diabetes researchers to better understand the differences and benefits of the various automated insulin delivery (AID) systems. This five-step framework asks pertinent questions designed to compare, understand, and ultimately choose the right AID therapy for users.



Scan to learn more information

CUSTOMIZE, DON'T COMPROMISE

t:slim X2 Insulin Pump with Control-IQ technology vs. MiniMed 780G¹



Calculate

How does the algorithm calculate insulin delivery?

Adjust

How can insulin delivery be adjusted?

Revert

When will the system default to open loop (no automation)?

Educate

What are the critical education points?

Sensor/Share

What are the key aspects of the systems' sensor and sharing capabilities?

¹Panther™ Diabetes Technology Deciphered.™ 23 May 2023, <https://www.pantherprogram.org/device-comparison>

Calculate

How does the algorithm calculate insulin delivery?

BASAL AUTOMATION

BASAL AUTOMATION TREATMENT VALUES / TARGET GLUCOSE

BOLUS AUTOMATION

BOLUS CORRECTION TARGET

Tandem



Basal rate modulates when the 30-minute predicted glucose is outside of the treatment range and is calculated using a metabolic target

6.25-8.9 mmol/L
6.25-6.7 mmol/L
7.8-8.9 mmol/L



Automatic Correction Bolus
uses 30-minute predicted glucose value to deliver a 60% correction dose when expected to be ≥ 10.0 mmol/L[†]

6.1 mmol/L



Medtronic



Auto basal is calculated based on TDI delivery from past 2-6 days

5.5, 6.1 or 6.7 mmol/L



Auto-correction bolus
Uses current and past glucose trends to determine dose and delivers if glucose > 6.7 mmol/L and at maximum "auto basal" delivery

6.7 mmol/L

CALCULATE

Control-IQ Technology and How it Works

Control-IQ technology **predicts glucose levels 30 minutes ahead** and automatically adjusts insulin every 5 minutes.



Helps Prevent Lows

Decreases or stops basal insulin if sensor glucose is predicted to be low.




Helps Prevent Highs

Increases basal insulin and delivers automatic correction boluses* () if sensor glucose is predicted to be high.



Scan to see how Control-IQ technology works

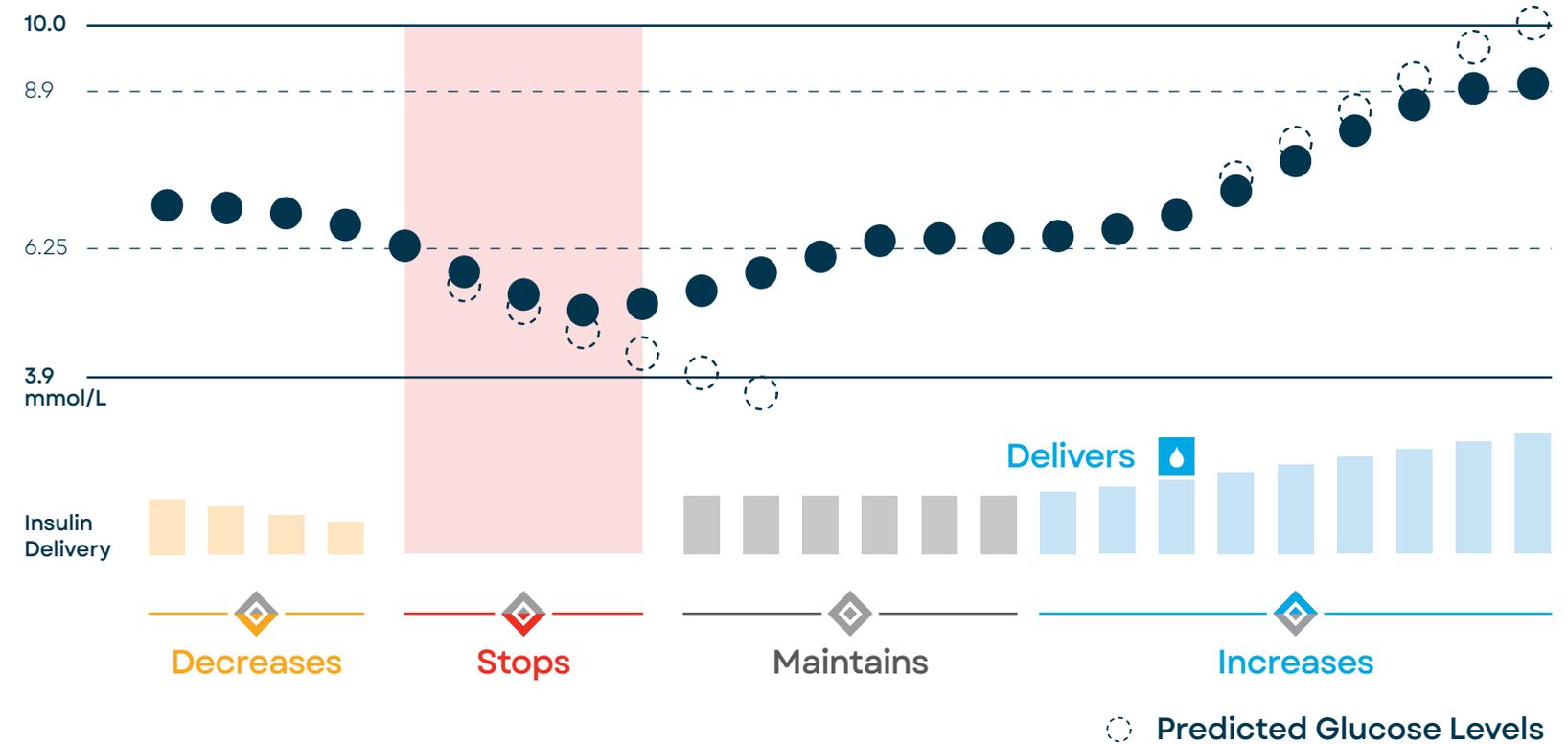

How Do You Treat High Glucose?

- Increase Basal Insulin
- Give a Correction Bolus
- Control-IQ technology does both!**

Adjust

How can insulin delivery be adjusted?

MODIFIABLE INSULIN SETTINGS THAT IMPACT AUTOMATION

Tandem

- Basal Rate
- Correction Factor
- Active Insulin Time
- Correction Target

EXTENDED BOLUS



EDIT CALCULATED BOLUS DOSE



SPECIAL FEATURES

Sleep Activity

Changes treatment values to 6.25-6.7 mmol/L

Exercise Activity

Changes treatment values to 7.8-8.9 mmol/L

Medtronic

- Basal Rate
- Correction Factor
- Active Insulin Time
- Correction Target



Temp Target

Changes algorithm target to 8.3 mmol/L



Mide
diagnosed 2012

ADJUST

Control-IQ Technology Aims to Increase the Time Spent in Range between 3.9-10.0 mmol/L

Sleep Activity

Sets a lower range of treatment values (6.25-6.7 mmol/L) without delivering automatic correction boluses.[†] This feature is designed to achieve glucose levels of approximately 6.1-6.7 mmol/L during a fasting state.

Who may benefit from Sleep Activity all day?

- + Those wanting basal modulation to occur with the tighter range of treatment values
- + People not wanting automatic correction boluses[†]
- + Users willing to give manual corrections as necessary



Sleep Schedule during the night

Sleep Schedule all day



HELPFUL TIP

Automatic correction boluses[†] may be more helpful in certain situations as opposed to basal modulation only.

Consider the following when setting a Sleep Schedule to optimize start and stop times:

- + Start Sleep 1-2 hours after the last snack or meal
- + Stop Sleep prior to waking
- + Turn Sleep off during times of high stress or growth spurts

ADJUST

Exercise Activity

Sets a higher range of treatment values (7.8-8.9 mmol/L) that are designed to accommodate the likely natural drop in glucose during and following exercise.


Alternative uses for additional hypoglycemia protection:

- + Intermittent fasting
- + NPO for medical procedure
- + Nausea or vomiting
- + Hypoglycemia unawareness
- + Fear of hypoglycemia
- + Those who are older or at high-risk
- + Alcohol consumption



HELPFUL TIP

Turn on Exercise Activity as a temporary reset for glycemic variability and ease off the rollercoaster

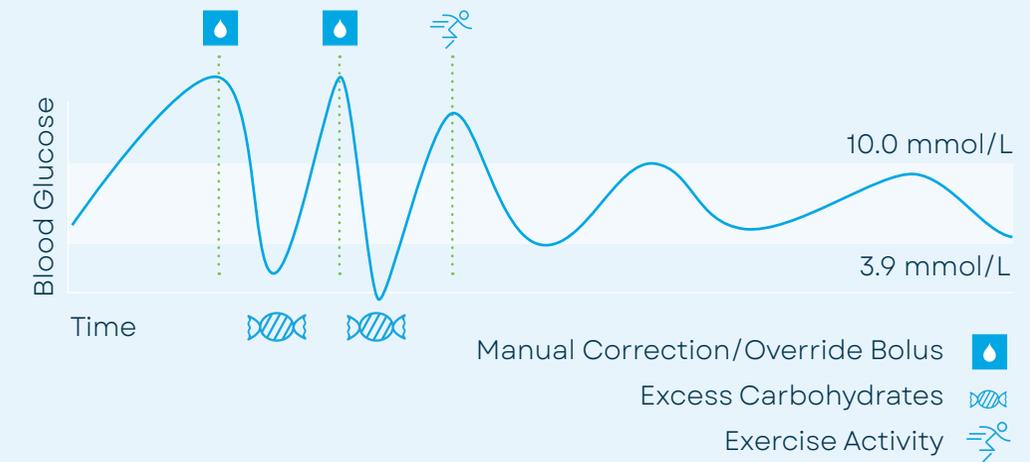

Revert

When will the system default to open loop (no automation)?

OUT OF FULL AUTOMATION

Tandem

No CGM data \geq 20 min

(Defaults to Personal Profile settings and Control-IQ technology automatically resumes when CGM⁺ connection is restored)



Flashing yellow light and **Out of Range** alert notify the user that safety conditions may need attention.

Medtronic

Safe-Basal

(static basal rate determined by the system without glucose-dependent adjustment)

Possible triggers include:

- + System concerns about sensor accuracy
- + Max/min delivery constraints
- + Loss of CGM data

Manual Mode

(no automation)

Possible triggers include:

- + Triggered after four hours in Safe-Basal mode

REVERT

Today's doses shouldn't be based on yesterday's activities.




Control-IQ technology

Customize insulin dosing using an alternate Personal Profile.

Up to six Personal Profiles can be created to personalize anticipated changes in insulin requirements.

- + Illness
- + Steroids
- + Menstruation
- + Vacation
- + Summer Camp

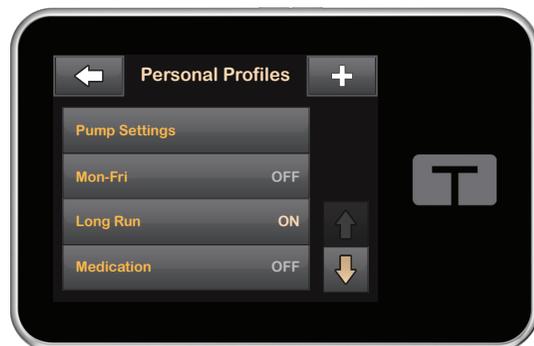

Today	
10 AM	Mon-Fri Personal Profile
11 AM	
12 PM	
1 PM	
2 PM	Long Run Personal Profile
3 PM	
4 PM	
5 PM	
6 PM	



Personal Profile	Mon-Fri	Long Run
Basal Rate	0.8 units	0.4 units
Carb Ratio	9 grams	9 grams
Correction Factor	2.2 mmol/L	4.4 mmol/L

Doses are for illustrative purposes only.

Educate

What are the critical education points?

MEALTIME

Tandem

For sensor glucose < 6.1 mmol/L, consider declining prompt:

*"Your BG is **Below** Target
Reduce Bolus Calculation?"*


OVERNIGHT

Sleep Activity Schedule

EXERCISE

Exercise Activity
and/or **Personal Profiles**

OTHER
CONSIDERATIONS

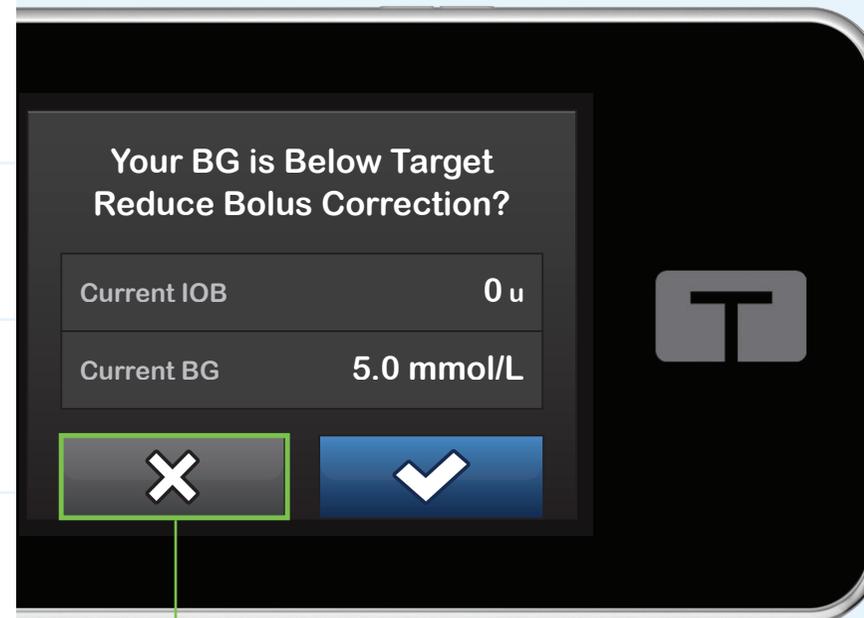

Consider declining prompt to reduce bolus

Medtronic

Potential need to strengthen carb ratio

For insulin sensitive users, consider setting a temp target to prevent hypoglycemia

Temp Target

- + **Unable to deliver** manual correction doses
- + Follow prompts for BG Required to stay in SmartGuard

How to Prevent and Treat Hypoglycemia



BOLUSING TIPS



Bolus with success

Use the recommended bolus doses because additional insulin may have been delivered by the algorithm. Check insulin on board (IOB) before overriding dose recommendations.



Bolus before start of meal


If bolus is given after meal, then the algorithm may have increased insulin delivery and can cause hypoglycemia if bolus dose is not adjusted appropriately.

- + 30-60 min after start of meal: Deliver half of recommended bolus¹
- + >60 min after start of meal: Deliver system-recommended correction bolus¹ only



HELPFUL TIP

The longer insulin has been suspended, the **fewer carbohydrates (5-10 g) are likely needed** to treat low glucose

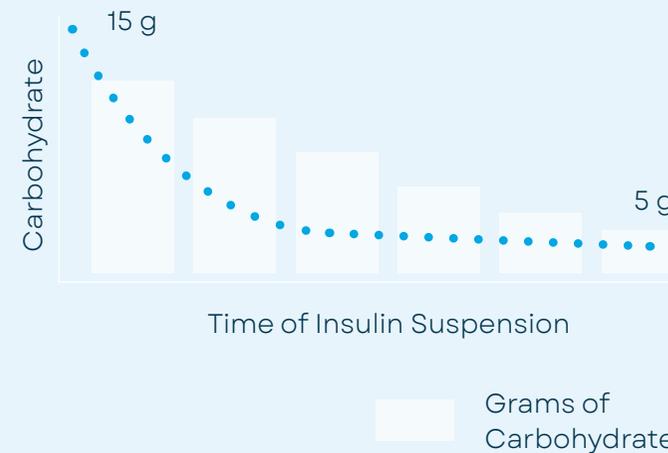

Examples of some foods and the carbs they contain.



= 5 Grapes
5 g Carbs



= 1-2 Glucose Tabs
5-10 g Carbs



= 75-125 ml of Juice
10-15 g Carbs

Amounts are approximate

Sensor/ Share

What are the key aspects of the systems' sensor and sharing capabilities?

COMPATIBLE CGM

Tandem

Dexcom G6

SENSOR WEAR TIME

10 days

NO CALIBRATION REQUIRED



SENSOR WARM UP TIME

2 hours

SENSOR ACCURACY

MARD^{2s}
9%

SHARE FEATURE



Dexcom G6 Follow App



Medtronic

Guardian 4

7 days



Required to enter automation and as system prompts

Up to 2 hours

MARD^{3s}
Adults 10.6%
Peds 11.6%



CareLink Connect App

Responsible Use of Control-IQ Technology

Control-IQ technology does not prevent all high and low blood glucose events, and is not a substitute for meal boluses and active self-management of diabetes. Control-IQ technology will not be able to predict sensor glucose values and adjust insulin dosing if a user's CGM is not working properly or is unable to communicate with their pump. Users should be instructed to always pay attention to their symptoms and blood glucose levels and treat accordingly. Please visit tandemdiabetes.com/tslimX2-use for more information.

*Personal Profile settings are delivered when the 30-minute predicted glucose is within the treatment range. [†]If glucose values are predicted to be above 10.0 mmol/L, Control-IQ technology calculates a correction bolus using the Personal Profile settings and a target of 6.1 mmol/L and delivers 60% of that value. [‡]CGM sold separately. [§]Mean absolute relative difference (MARD) is a statistical measure of accuracy.

References 1. Phillip M, Nimri R, Bergenstal RM, et al. Consensus Recommendations for the Use of Automated Insulin Delivery Technologies in Clinical Practice. *Endocr Rev.* 2023;44(2):254-280. doi: 10.1210/endrev/bnac022. 2. Beck RW, Riddlesworth T, et al. Effect of Continuous Glucose Monitoring on Glycemic Control in Adults with Type 1 Diabetes Using Insulin Injections – The DIAMOND Randomized Clinical Trial. *JAMA.* 2017;317(4):371-378. doi: 10.1001/jama.2016.19975. 3. Christiansen MP, Garg SK, Brazg R, et al. Accuracy of a Fourth-Generation Subcutaneous Continuous Glucose Sensor. *Diabetes Technol Ther.* 2017;19(8):446-456. doi: 10.1089/dia.2017.0087.

Important Safety Information: The t:slim X2 insulin pump with Control-IQ technology (the System) consists of the t:slim X2 insulin pump, which contains Control-IQ technology, and a compatible continuous glucose monitor (CGM, sold separately). The t:slim X2 insulin pump is intended for the subcutaneous delivery of insulin, at set and variable rates, for the management of diabetes mellitus in people requiring insulin. The t:slim X2 insulin pump can be used solely for continuous insulin delivery and as part of the System. When used with a compatible CGM, the System can be used to automatically increase, decrease, and suspend delivery of basal insulin based on CGM sensor readings and predicted glucose values. The System can also deliver correction boluses when the glucose value is predicted to exceed a predefined threshold. The pump and the System are indicated for use in individuals six years of age and greater. The pump and the System are intended for single user use. The pump and the System are indicated for use with U-100 insulin only. The System is intended for the management of Type 1 diabetes.

WARNING: Control-IQ technology should not be used by anyone under the age of six years old. It should also not be used in users who require less than 10 units of insulin per day or who weigh less than 25 kilograms.

The System is not indicated for use in pregnant women, people on dialysis, or critically ill users. Do not use the System if using hydroxyurea.

Users of the pump and the System must: be willing and able to use the insulin pump, CGM, and all other system components in accordance with their respective instructions for use; test blood glucose levels as recommended by their healthcare provider; demonstrate adequate carb-counting skills; maintain sufficient diabetes self-care skills; see healthcare provider(s) regularly; and have adequate vision and/or hearing to recognize all functions of the pump, including alerts, alarms, and reminders. The t:slim X2 pump and the CGM transmitter and sensor must be removed before MRI, CT, or diathermy treatment. Visit tandemdiabetes.com/safetyinfo for additional important safety information.

© 2023 Tandem Diabetes Care, Inc. All rights reserved. Tandem Diabetes Care, the Tandem logo, Control-IQ, and t:slim X2 are either registered trademarks or trademarks of Tandem Diabetes Care, Inc. in the United States and/or other countries. Dexcom, Dexcom G6, Dexcom Follow, and any related logos and design marks are either registered trademarks or trademarks of Dexcom, Inc. in the United States and/or other countries. All third-party marks are the property of their respective owners. ML-1011801_A



(833) 509-3598
tandemdiabetes.ca