AutoSoft™ +

Infusion Set / Dispositif de perfusion / Equipo de infusión /
Infusionsset / Set di infusione / Infusieset / Infusionssæt /
Infusionssett / Infuusiosarja / Conjunto de infusão /
Infúzió szett / Infuzní souprava / Infúzna súprava /
Μجموعة التثريب / סט עירוי / Infuzijski set / Σετ έγχυσης

Contacts / Contactos / Kontakt /
 Contatti / Contactinformatie /
 Kontakter / Kontaktinformasjon /
 Yhteystiedot / Contatos / Elérhetőség /
 Obsah / Kontakty / Επαφές / Stiki /
 جهات الاتصال / אנשי קשר

tandemdiabetes.com/contact

Instructions for Use / Notice d'utilisation / Instrucciones de uso / Gebrauchsanweisung /
 Istruzioni per l'uso / Gebruiksaanwijzing / Brugsanvisning / Bruksanvisning / Käyttöohjeet /
 Instruções de utilização / Használati útmutató / Návod k použití / Návod na použitie /
 Οδηγίες χρήσης / Navodila za uporabo / הוראות שימוש / إرشادات الاستخدام

EN	English	2
FR	Français	8
ES	Español	14
DE	Deutsch	20
IT	Italiano	26
NL	Nederlands	32
DA	Dansk	38
NO	Norsk	44
FI	Suomi	50
SV	Svenska	56
PT	Português	62
HU	Magyar	68
CS	Česky	74
SK	Slovenský	80
EL	Greek	86
SL	Slovenski	94
HE	עברית	100
AR	العربية	106

Instructions for Use

Read all instructions carefully before using the AutoSoft™ + infusion set.

INDICATIONS FOR USE

The AutoSoft + infusion set is indicated for subcutaneous infusion of medication administered by an external Tandem Diabetes Care™ insulin pump. The infusion set is indicated for single use. The infusion set is designed to be used for patients specified in the instructions for use accompanying the external Tandem pump.

INTENDED USE

The infusion set is indicated for subcutaneous infusion of insulin in the treatment of diabetes mellitus. The infusion set has been tested to be compatible with insulin approved for subcutaneous infusion.

CLINICAL BENEFITS

- Continuous subcutaneous infusion of insulin from an external Tandem pump.
- Fewer needle insertions compared with multiple daily injections.

TARGET PATIENT POPULATION

The infusion set is designed to be used for patients specified in the instructions for use accompanying the external pump. Please visit the Tandem Diabetes Care website to find the user guide for your Tandem insulin pump.

GENERAL INSTRUCTIONS

- The infusion set may be worn for a maximum of three days, or per your healthcare professional's instructions.
- If the infusion set is being used for the first time, do the first set-up in the presence of a healthcare professional.

CONTRADICTIONS

This infusion set is indicated for subcutaneous use only. Do not use the infusion set for intravenous infusion. Do not use the infusion set with blood or blood products.

WARNINGS

- Do not use the infusion set if the package is opened or damaged. An opened or damaged package may be contaminated. This can cause an infection.
- Do not use the infusion set if the disconnect cover is removed. Use a new infusion set instead. One purpose of the disconnect cover is as a safety feature against accidental firing of the insertion device which could cause injury.
- Do not change the infusion set just before bedtime unless blood glucose can be checked one to three hours after insertion.
- Do not put alcohol, disinfectants, perfumes, deodorants, cosmetics, or other substances with solvents on the infusion set. These substances may damage the integrity of the infusion set.
- Never point the insertion device toward any body part where insertion is not desired.
- Always rotate to a new position within any of the recommended

insertion sites when changing the infusion set. Reusing an insertion site too often may cause scarring and unpredictable insulin delivery. Choose insertion sites as recommended by a healthcare professional.

- Check the insertion site often through the clear window. Improper insertion and maintenance of the insertion site can cause inaccurate insulin delivery, infection, or site irritation. Replace the infusion set at a new site if the soft cannula is not properly inserted or the adhesive tape becomes loose or displaced.
- Check the cannula housing and tubing for blood. Blood may cause insufficient insulin delivery. This may result in high blood glucose. Replace the infusion set at a new site if there is blood.
- Fill the tubing completely with insulin before insertion. Do not leave air in the tubing. An uncontrolled amount of insulin may be delivered if air is present.
- Check the tubing for clogs or leaks if a blood glucose reading is high. Clogs or leaks may restrict insulin delivery and result in high blood glucose. Replace the infusion set even if a clog or leak is suspected, but not found.
- Do not attempt to clear air or an occlusion in the tubing while it is connected to the body. An uncontrolled amount of insulin may be delivered. This can cause high or low blood glucose. Disconnect the tubing before adjusting it.
- Do not reuse the infusion set. Reuse of the infusion set may cause damage to the cannula or needle and may lead to infection, site irritation, or unpredictable insulin delivery.
- For occlusion alarms, refer to your Tandem pump user guide.

PRECAUTIONS

- Do not use another type of infusion set without consulting a healthcare professional for correct handling. A healthcare professional should always be consulted when choosing an infusion set.
- Choose insertion sites as recommended by a healthcare professional. The choice of site depends on treatment and patient-specific factors such as body composition and physical activity level.
- Check that the insertion site is free of skin irritation, such as redness, scar tissue, or bleeding. Do not insert the infusion set into muscle or over bone. Doing so can cause pain or may damage the infusion set. If the infusion set is improperly inserted, replace the infusion set and insert at a new site.
- If needed, remove body hair around the insertion site to ensure that the adhesive sticks to the skin.
- Do not use an insertion site located under a belt or waistband, or where the site is constrained by clothing or accessories. Insertion at these locations can cause disconnection of the infusion set and interruption of insulin delivery, and lead to hyperglycemia.
- Do not reposition the infusion set on the body after the adhesive is placed on the skin. Repositioning the infusion set may damage the adhesive. Replace the infusion set if the adhesive is damaged.
- Always check blood glucose one to three hours after inserting a new infusion set. This is to confirm accurate insulin delivery. High or low blood glucose can result if insulin delivery is not accurate.
- Check blood glucose several times throughout the day or as recommended by a healthcare professional.

- Consult a healthcare professional for how to correct for missed insulin and for how much time the pump can remain disconnected.

PRODUCT DESCRIPTION

The AutoSoft + infusion set has a 90-degree soft cannula. It is delivered ready to use in a pre-loaded insertion device with automatic needle retraction. It is provided sterile and nonpyrogenic.

INTENDED USERS

Patients, caregivers and healthcare professionals.

USE ENVIRONMENT

Clinical settings (e.g. hospital and clinic) or home environment.

RECOMMENDATIONS

- Check your blood glucose level one to three hours after insertion of the AutoSoft +. Measure your blood glucose on a regular basis. Discuss this with your healthcare provider.
- If your blood glucose level becomes unexplainably high or an occlusion alarm occurs, check for clogs and leaks. If in doubt, change your infusion set since the soft cannula could be dislodged or partially clogged. Discuss plan to bring down your blood glucose level with your healthcare provider should any of these problems arise. Test your blood glucose level to make sure the problem is corrected.
- Carefully monitor your blood glucose level when disconnected and after reconnecting.

- If, during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

STORAGE & DISPOSAL

- Store infusion sets in a cool, dry location at room temperature. Do not store infusion sets in direct sunlight, or in high humidity.
- Store and handle insulin according to the manufacturer's instructions.
- Dispose of the insertion device in a proper sharps container according to local laws, to avoid risk of needle stick injuries.
- Dispose of a used infusion set according to local regulations for biohazardous waste.

SERIOUS INCIDENT

If, during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

ADDITIONAL INFO FOR THE HEALTHCARE PROFESSIONAL

- Consider treatment and patient-specific factors when choosing an insertion site.
- The length of the soft cannula should be based on treatment and patient factors such as physiology and activity level. If the soft cannula is too long, insertion pain or insertion into the bone or muscle may occur. If the soft cannula is too short, leakage or irritation at the insertion site may occur. In both cases, the infusion

must be stopped and a new soft cannula with a different length must be inserted at a new insertion site.

- Inform the patient that the selection of an insertion site is dependent on cannula length.

Components

- A. Top button
- B. Disconnect cover
- C. Insertion device
- D. Raised arrows indicating tubing direction
- E. Clear Window
- F. Cannula housing
- G. Adhesive
- H. Paper backing
- I. Soft cannula
- J. t:lock tubing connector
- K. Tubing
- L. Site Connector
- M. White cap for site connector

Instructions for Use

Read all instructions carefully before using the AutoSoft + infusion set.

- Consult your Tandem pump user guide for important information about pump therapy prior to connecting the infusion set. This information includes connection and filling procedures, possible errors, and potential risks related to pump therapy.
- Follow good hygienic procedures. If this infusion set is being used for the first time, perform the first set-up in the presence of a healthcare professional.

INSERTING

Unfold the front cover of this booklet to view images that are keyed to the steps below.

1. Wash hands with soap and water.
2. Select a recommended insertion site (shown in gray) as indicated by a healthcare professional.
3. Clean the insertion site with a disinfectant as directed by a healthcare professional. Allow to air dry before inserting the infusion set. Remove hair around the insertion site to ensure the adhesive sticks to the skin if needed.
4. Open the package and remove the paper from the tubing.
5. Attach the infusion set tubing connector to the cartridge tubing. Twist clockwise until it is tightly secured. Ensure there is a tight connection between the infusion set tubing and the cartridge tubing.

6. Remove the white cap from the site connector by (a) squeezing the sides of the site connector then (b) pulling the white cap off. Keep the white cap for later use.
7. Fill the tubing with insulin as instructed by your Tandem pump user guide. Do not leave any air in the tubing. The tubing is completely filled when drops of insulin are seen coming out of the site connector.
8. Remove the paper backing from the adhesive. Be careful not to touch the adhesive.
9. Remove the disconnect cover from the insertion device by (a) gently squeezing the sides of the disconnect cover then (b) pulling it away from the insertion device. Keep the disconnect cover for later use. The raised arrows indicate the tubing direction.
10. Stretch the skin until smooth. Then (a) press the insertion device against the skin and (b) press the top button down completely to insert the infusion set.
11. Gently and carefully remove the insertion device.
12. Gently massage the adhesive onto the skin with one finger. Replace the infusion set if the adhesive does not stick to the skin.
13. Gently hold the cannula housing steady with one finger. Then push the site connector straight into the cannula housing until it clicks.
14. Fill the soft cannula with insulin:
 - 6 mm: 0.6 units (0.006 ml)
 - 9 mm: 0.6 units (0.006 ml)

Dispose of the insertion device in an appropriate sharps container according to local laws. Keep the disconnect cover and the white cap for use when the infusion set is disconnected.

DISCONNECTING

- A1. Wash hands with soap and water.
- A2. Gently hold the cannula housing steady with one finger. Then (a) squeeze the sides of the site connector and (b) pull the site connector out of the cannula housing.
- A3. Put the white cap on the site connector.
- A4. Put the disconnect cover on the cannula housing. Then push the cover into the cannula housing until it clicks.

RECONNECTING

- B1. Wash hands with soap and water.
- B2. Use one finger to gently hold the cannula housing steady. Then (a) squeeze the sides of the disconnect cover and (b) pull the disconnect cover out of the cannula housing.
- B3. Remove the white cap from the site connector by (a) squeezing the sides of the site connector then (b) pulling the white cap off. Ensure that there is no air in the tubing. If there are air bubbles in the tubing: Fill the tubing with insulin as instructed in your Tandem pump user guide. Do not leave any air in the tubing.
- B4. Use one finger to gently hold the cannula housing steady. Then, push the site connector into the cannula housing until it clicks.

REMOVING

- C1. Wash hands with soap and water.
- C2. Use one finger to gently hold the cannula housing steady. Then (a) squeeze the sides of the site connector and (b) pull the site connector out of the cannula housing.

- C3. Disconnect the infusion set tubing connector from the cartridge tubing by turning the connector counter-clockwise.
- C4. Carefully lift the adhesive around the cannula housing. Then, pull the soft cannula out of the skin.

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© 2026 Tandem Diabetes Care, Inc. All rights reserved. Tandem Diabetes Care, Tandem logos, t:lock, and AutoSoft are either registered trademarks or trademarks of Tandem Diabetes Care, Inc. in the United States and/or other countries. All third-party marks are the property of their respective owners. / Tous droits réservés. Tandem Diabetes Care, logos Tandem, t:lock, et AutoSoft sont des marques déposées ou des marques commerciales de Tandem Diabetes Care, Inc. aux États-Unis et/ou dans d'autres pays. Toutes les autres marques de tierces parties appartiennent à leurs propriétaires respectifs.
Tandem Diabetes Care, Inc.