


Deliver a Quick Bolus


- 1 Press and hold the silver “T” button on the pump
- 2 Listen for two beeps or feel for vibrations
- 3 Press the silver “T” button for each increment of desired delivery (e.g., press three times to represent three increments)
- 4 Wait for the pump to beep/vibrate for each increment pressed
- 5 Press and hold the silver “T” button until the pump beeps/vibrates to confirm the bolus delivery




Set up Quick Bolus: Tandem Mobi System

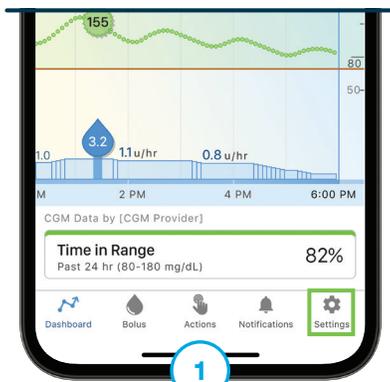


Tap **Settings** from the Navigation bar.

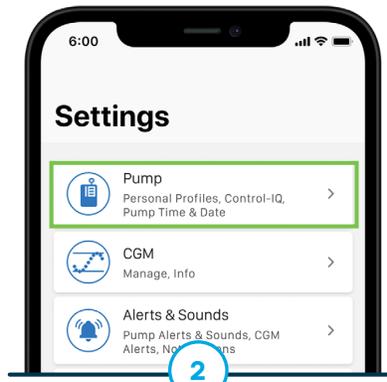


Tap **Pump**.

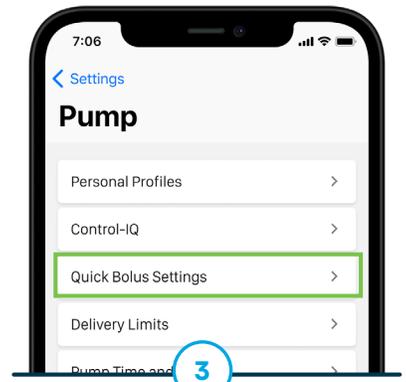


Tap **Quick Bolus Settings**.

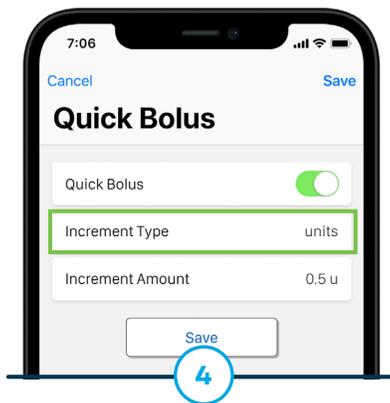

Tap **Increment Type** and then choose units of insulin or grams of carbohydrates.

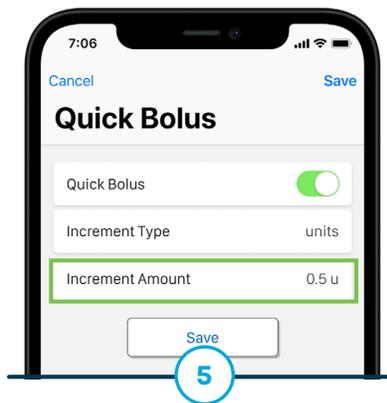

Tap **Increment Amount** and select desired value.

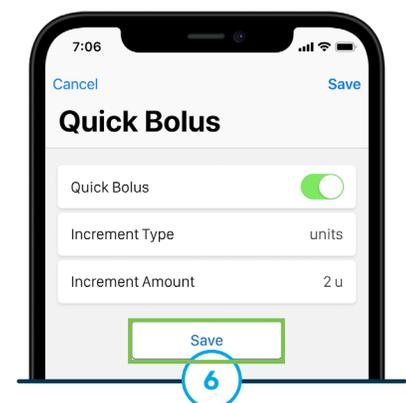

Tap **Save** to confirm.


Set up Quick Bolus: t:slim X2 Insulin Pump




From the Home screen, tap **Options** and then **My Pump**.



Tap **Personal Profiles** and then **Pump Settings**.

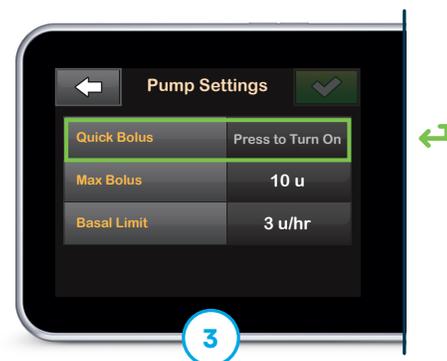


Tap **Quick Bolus**.

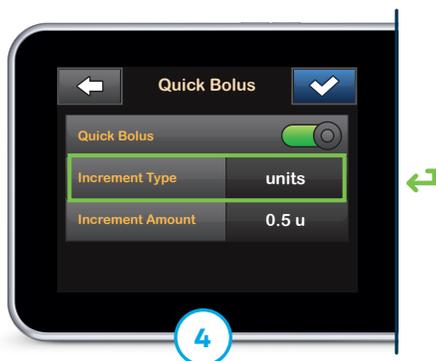

Tap **Increment Type** and then choose units of insulin or grams of carbohydrates.



Tap **Increment Amount** and select desired value.

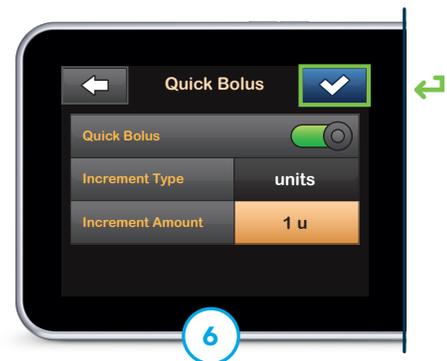

Tap to continue and then tap to save settings.

Visit tandemdiabetes.com/support for additional resources.

These instructions are provided as a reference tool for individuals who are already familiar with the use of a Tandem Diabetes Care insulin pump. Not all screens are shown. For more detailed information on the operation of the t:slim X2 pump or the Tandem Mobi system, please refer to its user guide.

Important Safety Information: RX ONLY. Tandem insulin pumps are intended for the subcutaneous delivery of insulin, at set and variable rates, for the management of diabetes mellitus in persons requiring insulin. Visit tandemdiabetes.com/safetyinfo for additional important safety information.

WARNING: Control-IQ+ technology should not be used in anyone under the age of 2 years old with type 1 diabetes or under the age of 18 years old with type 2 diabetes. It should also not be used in patients who require less than a total daily insulin dose of 5 units of insulin per day or who weigh less than 20 pounds (9 kilograms), as those are the required minimum values needed for Control-IQ+ to operate safely.

© 2025 Tandem Diabetes Care, Inc. All rights reserved. Tandem Diabetes Care, Tandem logos, Tandem Mobi, t:slim X2, and Control-IQ+ are either registered trademarks or trademarks of Tandem Diabetes Care, Inc. in the United States and/or other countries. MAT-0047_1