ENGLISH (en)

Indications for use:

The VarisSoft™ infusion set is indicated for the subcutaneous infusion of insulin administered by an external pump. The infusion set is indicated for single use.

INTENDED USE

The infusion set is indicated for subcutaneous infusion of insulin in the treatment of diabetes. The infusion set has been tested to be compatible with insulin approved for subcutaneous infusion.

PRODUCT DESCRIPTION

VarisSoft is an infusion set with an insertion angle between 20-45°. The VarisSoft cannula and tubing make up the infusion set. This product is delivered sterile and ready for use.

CONTRAINDICATIONS

VarisSoft is neither intended nor indicated for use with blood or blood products (i.V. infusion).

WARNINGS

- Carefully examine the instructions for use before inserting VarisSoft. Failure to follow instructions may result in pain or injury.
- VarisSoft infusion sets with tlock™ tubing connectors should only be used with Tandem Diabetes Care™ catheters featuring the tlock connector.
- The VarisSoft infusion set is only sterile and non-pyrogenic if the packaging is undamaged and unopened. Do not use it if the packaging is already opened or so damaged.
- When using VarisSoft for the first time, do so in the presence of a healthcare provider.
- Inaccurate medication delivery, site irritation or infection may result from improper insertion and maintenance of infusion site.
- Do not re-insert the introducer needle into the infusion set. This could cause a tearing or damage of the soft cannula and may lead to unpredictable medication delivery.
- Reuse of the infusion set may cause site irritation or infection and may lead to inaccurate medication delivery.
- Be sure that the needle guard is removed before insertion is made.

PRECAUTIONS

- Never try to fill or free a clogged tubing while the infusion set is inserted. This may result in unpredictable medication flow.
- Do not leave air in the infusion set. Make sure to fill the tubing completely. Consult pump manufacturer's instructions for use.
- VarisSoft is a single-use item, which must be disposed of after use. Do not clean or re-sterilize.
- Check the infusion set frequently to ensure that the soft cannula remains firmly in place. Replace with a new infusion set at a new site, if the infusion set is not in place. Because the cannula is soft, it will not cause any pain if it slips out, and this may happen without notice. The soft cannula must always be completely inserted to receive the full amount of medication.
- Do not try to use perfumecants, perfumes, deodorants or other products containing disinfectants in contact with the infusion set or the tubing. This may affect the integrity of the infusion set.
- Replace the infusion set if the adhesive tape becomes loose.
- Replace the infusion set every two to three days, or per your healthcare provider's instructions.

- If the infusion site becomes irritated or inflamed, replace the infusion set and use a new site until the first site has healed.
- Use aseptic techniques when temporarily disconnecting VarisSoft. Consult with healthcare provider on how to compensate for missed medication when disconnected.
- People with small or large amounts of subcutaneous fat should choose the insertion angle carefully, as the cannula could be placed in underlying muscle or dermal layer which may lead to unpredictable medication delivery. Normal insertion angle between 20-45°. Consult your healthcare provider concerning this matter.
- Do not change infusion site more than three times, unless blood glucose can be checked one to three hours after insertion.
- If your blood glucose levels become unexpectedly high, or occlusion alarm occurs, check for clogs and leaks. If it is plugged, crimped or partially clogged. Discuss plan to bring down your blood glucose level with your healthcare provider should any of these problems arise. Test blood glucose level to make sure treatment is corrected.
- Carefully monitor your blood glucose level when disconnected and after reconnecting.

- During the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.
- STORAGE & DISPOSAL.**
- Store infusion sets in a cool, dry location at room temperature. Do not store infusion sets in direct sunlight, or in high humidity.
- Store and handle insulin according to the manufacturer's instructions.
- Dispose of the introducer needle in a proper sharps container according to local laws, to avoid risk of needle stick.
- Dispose of a used infusion set according to local regulations for biohazardous waste.

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