Tandem Mobi™

Adhesive Sleeve / Parche adhesivo /
Manchon adhésif / Klebehülse /
Guaina adesiva / Zelfklevende huls /
Selvklæbende hylster / Klebende holder /
Teippitasku / Klisterlapp / Lepící obal /
Manga adesiva / שרוול דבק / غلاف لاصق

TANDEM™
Diabetes Care

United Kingdom

MedEnvoy UK Limited

85 Great Portland Streetl • First Floor

London W1W 7LT • United Kingdom

Switzerland

MedEnvoy Switzerland

Gotthardstrasse 28 • 6302 Zug • Switzerland

European Union

MedEnvoy Gloval B.V

Prinses Margrietplantsoen 33 • Suite 123

2595 AM Den Haag • Netherlands

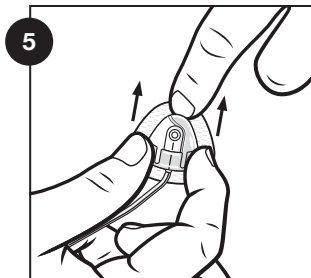
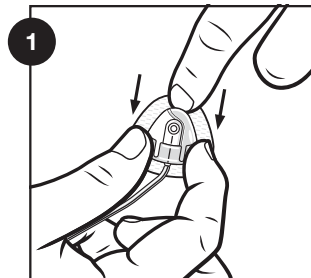
Contacts / Contactos /
Coordonnées / Kontakte / Contatti /
Contactpersonen / Kontakter /
Yhteystiedot / Kontakty / Stiki /
אנשי קשר / جهات الاتصال

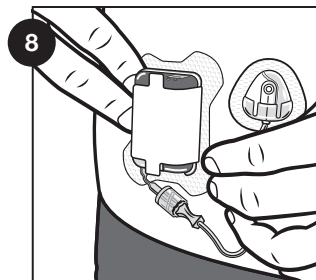
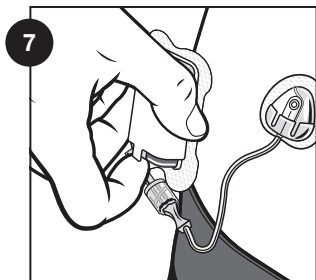
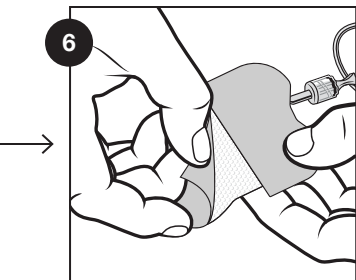
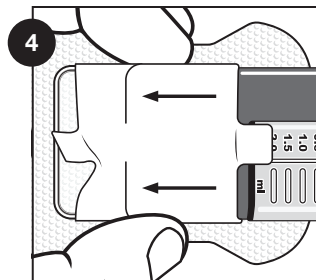
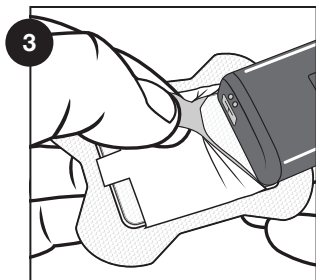
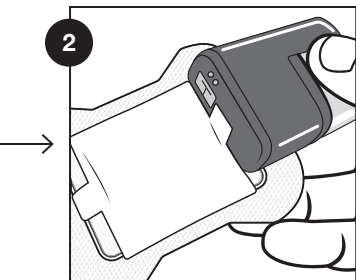
tandemdiabetes.com/contact

Instructions for Use / Instrucciones de uso / Mode d'emploi /
Gebrauchsanleitung / Istruzioni per l'uso / Gebruiksaanwijzing /
Brugervejledning / Bruksanvisning / Käyttöohjeet / Návod k použití /
Instruções de utilização / Návod na použitie / Navodila za uporabo /
تعليمات الاستخدام / וראות

EN English	4
ES Español	6
FR Français	8
DE Deutsch	10
IT Italiano	12
NL Nederlands	14
DA Dansk	16
NO Norsk	18
FI Suomi	20
SV Svenska	22
CS Český	24
PT Português	26
SK Slovenčina	28
SL Slovenščina	30
HE עברית	32
AR الإنجليزية	34

Preparation and Installation / Preparación e instalación / Préparation et installation / Vorbereitung und Einsetzen / Preparazione e installazione / Voorbereiding en installatie / Klargøring og installation / Klargjøring og installasjon / Valmistelu ja asennus / Förberedelser och installation / Příprava a vložení / Preparação e instalação / Příprava a inštalácia / Priprava in namestitev / הכנה והתקנה / التحضير والتركيب

Instructions for Use

The Tandem Mobi™ adhesive sleeve is a non-sterile accessory that may be used to secure the Tandem Mobi insulin pump for up to three days. The sleeve is intended for single use application, do not attempt to reuse or reapply the sleeve.

▲ WARNINGS

- DO NOT use the sleeve if you have sensitive, irritated, fragile, or easily damaged skin or if you have an allergy to one or more acrylics.
- DO NOT place the sleeve on mucous membranes or compromised skin.
- ALWAYS insert the infusion set and connect to the pump before applying the adhesive sleeve to ensure the tubing is not stretched. Failure to follow these steps could result in crimping or dislodgement at the infusion site, which could affect the performance of the cannula. This could lead to hyperglycemia (high blood glucose)

▲ PRECAUTIONS

DO NOT wear or place your pump more than 12 inches above your infusion site. Doing so may result in over delivery of insulin.

Installation

Reference pages 2-3 to view images that are keyed to the steps below.

1. After completing the steps to load a cartridge using the Tandem Mobi mobile app, disconnect the tubing at the infusion site.
2. Place the pump with the Pump button touching the sleeve.
3. Lift and hold the tab at the end of the sleeve.
4. Push the pump through the sleeve until it is fully seated into the tray.
5. Connect the infusion set tubing to the infusion site.
6. Grip the pump with one hand while you peel off the adhesive paper backing.
7. Apply the sleeve onto your skin at the chosen location
8. Smooth out the fabric and press around the edges.

Tip: When placing the sleeve onto your skin, it's helpful to keep the pump button and LED Status Indicator facing up for easier visibility.

Placement Tips

- Ensure the location is clean and dry before applying

the sleeve. Using lotions or moisturizers on skin prior to placement may affect adhesion.

- Assistance may be required when applying your sleeve in a hard-to-reach location.
- Always rotate the application site of the Adhesive Sleeve and avoid consecutive use of the same site.

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© 2025 Tandem Diabetes Care, Inc. All rights reserved. Tandem Diabetes Care, Tandem logos, and Tandem Mobi are either registered trademarks or trademarks of Tandem Diabetes Care, Inc. in the United States and/or other countries. All third-party marks are the property of their respective owners. / Tous droits réservés. Tandem Diabetes Care, logos Tandem, et Tandem Mobi sont des marques déposées ou des marques commerciales de Tandem Diabetes Care, Inc. aux États-Unis et/ou dans d'autres pays. Toutes les autres marques de tierces parties appartiennent à leurs propriétaires respectifs.
🏢 Tandem Diabetes Care, Inc.