

Syringe Kit

Instructions for Use

Syringe Kit

Instructions for Use

DEVICE DESCRIPTION

A vacuum pressure syringe kit.

CONTENTS

The Syringe Kit consists of two 60mL locking syringes, one large bore 1-way stopcock, one 3-way large bore stopcock, and one piece of tubing.

INDICATIONS FOR USE

The VacLok Syringe is intended to be used by a cardiologist or radiologist during angiographic or radiologic procedures.

The flexible tubing and stopcocks are intended to be used as a vacuum-powered bodily fluid suction apparatus, used to aspirate, remove, or sample bodily fluids.

CAUTION


Federal (U.S.A.) Law restricts this device to use by or on the order of a physician. This is intended for single patient use only. Read instructions prior to use.

INSTRUCTIONS FOR USE







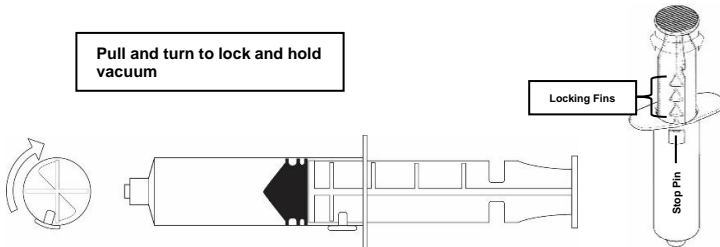





Inspect the device prior to use to verify that no damage has occurred during shipping.

1. To setup the syringe to lock, move the plunger completely forward or completely back and turn the plunger so that the locking fins can engage the stop pin.
2. To create and maintain vacuum, withdraw the syringe plunger to the desired position and rotate the plunger clockwise to position one of the locking fins behind the stop pin.
3. Turn the plunger counterclockwise to release.
4. To aspirate fluids without locking the syringe, move the plunger completely forward or completely back, turn the plunger so that the locking fins will not interfere with the stop pin.

Device is sterilized as stated in the package label.



SYMBOLS

	Batch code		Do not re-use
	Catalogue Number		Caution
	Non-pyrogenic		Use-by-date
	Keep dry		Keep away from sunlight
	Sterilized using ethylene oxide		Date of manufacture
	Manufacturer		Contents



Manufacturer:
 MicroVention, Inc.
 35 Enterprise
 Aliso Viejo, CA 92656 USA
 Tel: (714) 247-8000
www.microvention.com

PD050028 Rev. E 2024-05

© Copyright 2024 MicroVention, Inc. All rights reserved. Vaclok is a registered trademark of Merit Medical Systems, Inc.