English Instructions for Use

Tracess™ Guidewire

DEVICE DESCRIPTION
Tracess™ is a 0.014" diameter stainless steel consisting of a 0.027" diameter distal section of radiopaque pattern and stainless steel. The proximal section is made of hydrophilic polyurethane with a 14 mm diameter shaft section and a 14 mm diameter handle section. The distal 14 mm of the shaft is a sheathless. The distal core consists of nitinol, and the proximal section is a stainless steel shaft section covered with polyurethane/urethane (PTFE).


CONTENTS
One hydrophilic guidewire with Tracess™, Insertion Tool, and Shipping Mandrel.

INDICATIONS FOR USE
Tracess™ Guidewire is intended for general intracavitary use, including the ureters and peripheral vasculature. The guidewire can be directed to facilitate the selective placement of diagnostic or therapeutic catheters. This device is not intended for use in coronary arteries.

CAUTIONS
Do not exceed the maximum length (USA) indicated in this device by an order of a physician.

WARNINGS
The device should only be used by physicians who are familiar with angiographic and interventional procedures. It is important to follow the instructions for proper use to avoid the risk of complications. Do not use the device if the packaging is damaged or tampered with.

PREPARATION FOR USE
Before removing the guidewire, hydrate the hydrophilic segment by flushing heparinized saline for 30 seconds. Use a syringe or an irrigation system to avoid backflow of the saline (Figure 1).

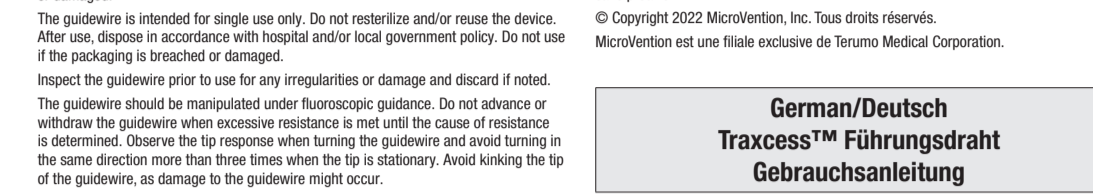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