


Teladoc Health™ Lite® 4 with Boom Camera

Quick Reference Guide

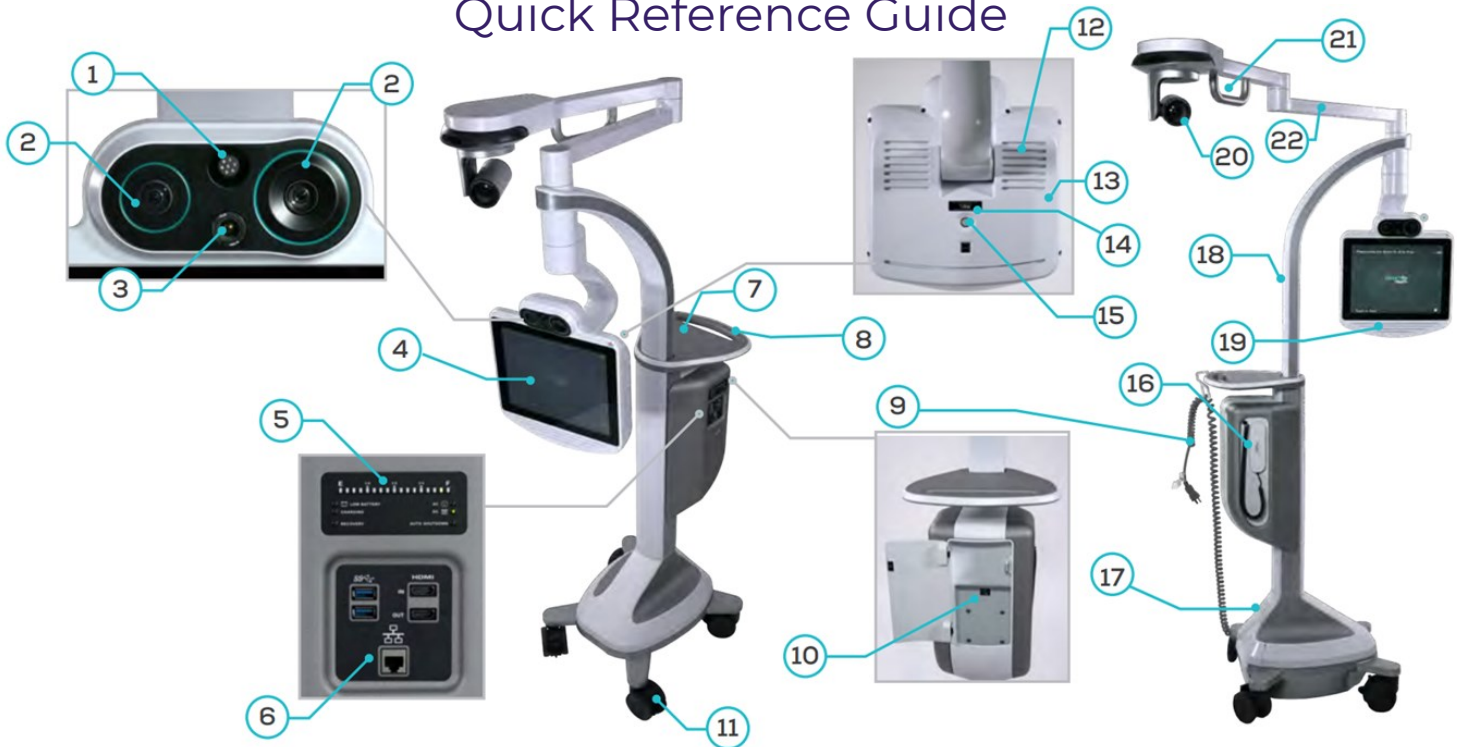

1	Directional Microphone	Used to pick up audio in a narrow zone.
2	Dual High-Definition Cameras	Captures remote video for viewing on the Provider Access Software.
3	Thermal Camera (Optional)	The Teladoc Health Thermal Camera™ is intended for adjunctive use in audio and video telemedicine consultations.
4	Touch Sensitive Display	Enables local controls and shows Provider Access Software user's face.
5	Battery Charging Status	Displays the power and charging status of the Lite 4.
6	Expansion Panel	Includes ports for approved USB peripheral devices, HDMI output, optional HDMI input, and Ethernet.
7	Tray	Work surface for a laptop or other items.
8	Handle	Easily maneuver the Lite 4.
9	Power Cord	Used to power the Lite 4 and recharge the battery.
10	Storage Compartment	Holds miscellaneous items.
11	Wheel Locks	Locks each wheel in place when stationary (four wheels).
12	Air Vents	Air vents help maintain a safe operating temperature for the device. (Do not cover)
13	Back of Touch Display	Location of the power button and additional USB ports.
14	Device Nameplate	Nameplate detailing the device's serial number and regulatory information.
15	Power Button	Used to toggle the power for the Lite 4.
16	Privacy Handset	Used for conversations requiring privacy or intelligibility in loud environments.
17	Serial Number Plate	Contains the device's serial number and support contact phone number.
18	Neck	Can be raised from 71" to a height of 92" by way of lifting or lowering the display.
19	Speakers	Used during sessions to hear the person connected to the device.
20	Boom Camera	Allows for an additional, unobstructed view of the patient from above. (Do not touch)
21	Boom Handle	Used to adjust the boom.
22	Boom	The boom can be adjusted to extend the boom camera above the patient.

Lite[®] 4 with Boom Camera Quick Reference Guide

Powering on the Lite 4

1. Plug the Patient Access Device into a standard grounded AC outlet and allow the system to charge for at least 6 hours to reach a full charge.
2. Power on the Lite 4 by pressing the power button located on the back of the head.

Power Button



Powering off the Lite 4

1. Press the power button on back of the head.
2. Confirm the power off with Patient Access Software by pressing the power button a second time.
3. Unplug the Lite 4 from the wall.

Do Not Turn Off

Turning off disables 24/7 patient care.

Press the power button again to confirm shutdown.

Adjusting the Height of the Lite 4



The Lite 4 head can be placed between 71" and 92" off the floor by raising or lowering the head to the desired height. Adjust the height with two hands on each sides of the display and gently raise or lower the display.

Adjusting the Boom

1. Lock the device's wheels.
2. Set the Head to its lowest position.
3. Use the boom handle to rotate the Boom to the desired position. **Do not touch the Boom Camera.**







Cleaning the Lite 4

Tap Settings  > Cleaning Mode , to disable the screen for 30-seconds to allow cleaning. See the online user guide for details on cleaning the Lite 4, including tested cleaning chemicals.



18 seconds
Cleaning mode

Lite 4 Screen Navigation

Icon	Function
	Tap to preview any of the cameras connected to the device.
	Tap to open device settings.
	Tap to open the device's User Guide.
	Tap to display information about the Lite 4.

Device Network Checkup

The Network Checkup screen runs and displays the results of a test of the device's current network connection.

Tap Settings  > Network Checkup  to run the Network Checkup.

