

Analytical Validation and Initial Clinical Evaluation of a New Blood-Based Diagnostic Test for Alzheimer's Disease

Andrew E. Schade, Adam Abel, Antonio Chambers, Jeff Fill, Heinz Reiske, Ming Lu, Amanda Morris, Michael J. Pontecorvo, Emily C. Collins, Mark A. Mintun, Michael E. Hodsdon

Eli Lilly and Company, Indianapolis, USA

Sponsored by Eli Lilly and Company



Scan the QR code to access this presentation or use URL <https://lillyscience.lilly.com/congress/ctad2023> for a list of all Lilly content presented at the congress. Other company and product names are trademarks of their respective owners.

OBJECTIVE

- To report the analytical validation and initial clinical performance of a plasma P-tau217 immunoassay test on a Quanterix SP-X platform for identifying amyloid PET-positive patients.

CONCLUSIONS


- This P-tau217 immunoassay using the Quanterix SP-X platform
 - has undergone systematic analytical validation,
 - demonstrates high positive and negative agreement with amyloid PET and,
 - could prove to be a useful diagnostic test to identify the presence or absence of amyloid pathology.

Clinical Trials on Alzheimer's Disease (CTAD); Boston, Massachusetts, USA; October 24-27, 2023

BACKGROUND


- Accessible, minimally invasive methods to detect AD pathology are lacking
 - Blood-based biomarkers have been gaining interest in the diagnostic work-up of neurodegenerative diseases, including AD

- Among the most interesting plasma biomarkers for AD is a post-translationally modified fragment of tau protein, P-tau217

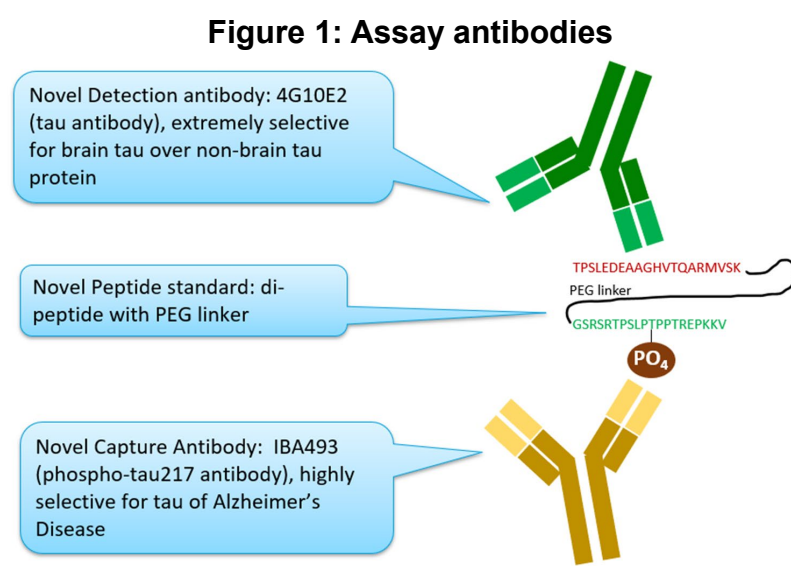

- Two previously characterized antibodies, 4G10E2 and IBA493, measure brain-specific tau-fragments [1]

METHODS

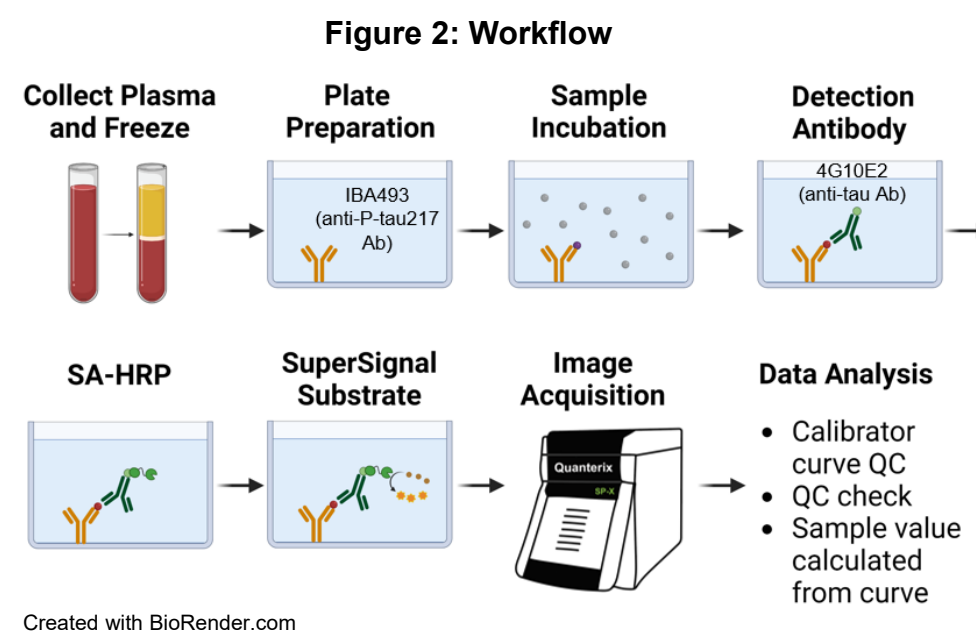

Created with BioRender.com

Plasma P-tau217 chemiluminescent immunoassay

- Formal analytical validation consisting of three Quanterix manufactured P-tau217 kit lots, nine different operators, and six Quanterix SP-X imagers was performed in our CAP-accredited CLIA lab
- QC Acceptance Criteria
 - Controls run twice on a plate
 - High, medium, and low controls are on every run in front and after patient samples on each plate
 - 4/6 controls must pass and at least one control must pass at each level
 - ≤ 20% U/mL duplicate CV
 - ± 20% RE from target U/mL
 - Ratio of controls must be within 0.85-1.15
 - Back and front controls are compared to check for plate bias
 - Average %RE of all controls passing duplicate %CV are used
 - Standard Curve Criteria and Curve Fitting:
 - 7 standards and a zero run on every plate
 - Power (Log-Log) Regression is used to convert raw values to U/mL
 - 4/5-PL curve fit increased both intra- and inter-run variability
 - Power Regression maintained/reduced both intra- and inter-run variability
 - 5/7 non-zero standards must pass
 - ≤20% raw duplicate CV
 - ±20% RE from target U/mL
 - If multiple fail, the most egregious is removed first

ANALYTICAL VALIDATION

Precision

- 23 patient-derived precision samples, individual results ranged from 0.09 U/mL to 3.35 U/mL
- Assessed 80 precision runs across 25 days and multiple imagers with 3 total assay kits
- Total precision was ≤20% CV
 - Sample %CV ranges from:
 - 7.2 to 36.1% with QC applied
 - 7.3 to 14.3% with QC applied and outlier removal
 - Within laboratory precision was 11.5% CV (95% CI 10.98, 12.01) with QC applied
- No significant lot-to-lot differences were observed

Table 1: Summary of within lab precision by reagent lot


Lot	Within Lab Precision (%CV)	95% CI
Lot 3	12.3	(11.25, 13.33)
Lot 4	9.2	(8.49, 9.80)
Lot 5	12.1	(11.07, 13.17)

Table 2: Summary of reagent lot contrasts

Mean Contrast	Ratio	90% CI
L3/L4	0.98	(0.97, 0.99)
L3/L5	0.97	(0.96, 0.99)
L4/L5	0.99	(0.98, 1.00)

Figure 3: Concentration per patient sample

Data from all passing plates without outlier removal

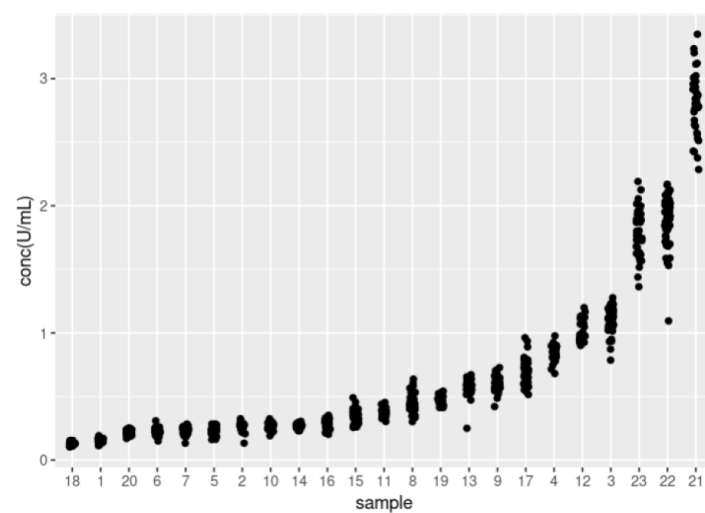

Instrument Comparison

- Multiple instruments demonstrated consistent results
 - A similar fail rate was observed for 5 of 6 instruments
 - One instrument was removed from the study due to a high fail rate

Parallelism and dilution linearity

- Parallelism and dilution linearity support the use of a standard 1:2 dilution of plasma sample in diluent

Analytical Measurement Range

- 0.080 to 2.814 U/mL

Sample stability

- Samples are stable for up to 7 freeze/thaw cycles with room temperature and refrigerated stability for at least 72 hours

Analytical Specificity and Interference

- Irrelevant antibodies were assessed by using irrelevant capture and detection antibodies paired with original antibodies which resulted in errors and did not produce data
- There was no endogenous interference from tau441 (non-phosphorylated tau) (up to 20 ng/mL tested) or lipemia
- Hemolysis greater than 2+ was found not acceptable

Sensitivity

- Analytical sensitivity was established as
 - LoB of 0.041 U/mL
 - LoB was established using sample diluent buffer. A total of 178 blank reportable results were generated across 3 instruments, 3 reagent kit lots, 7 analysts and up to 3 unique days.
 - LoD of 0.070 U/mL
 - LoD experiments were performed by measuring 6 K2EDTA plasma samples with P-tau217 concentrations <0.2 U/mL and were tested in 10 replicates per run, 2 runs per day for 3 days to reach a total of 60 measurements per sample.
 - LLOQ 0.080 U/mL
 - LLOQ was defined through an acceptable precision approach according to CLSI guidance EP17A2E. Four separate low level samples were run 5 times per plate, across 15 plates (5 plates per lot), for a total of 75 replicate measurements per sample.

Figure 4: Aqueous Blank Concentrations Reported per Instrument (colored by analyst)

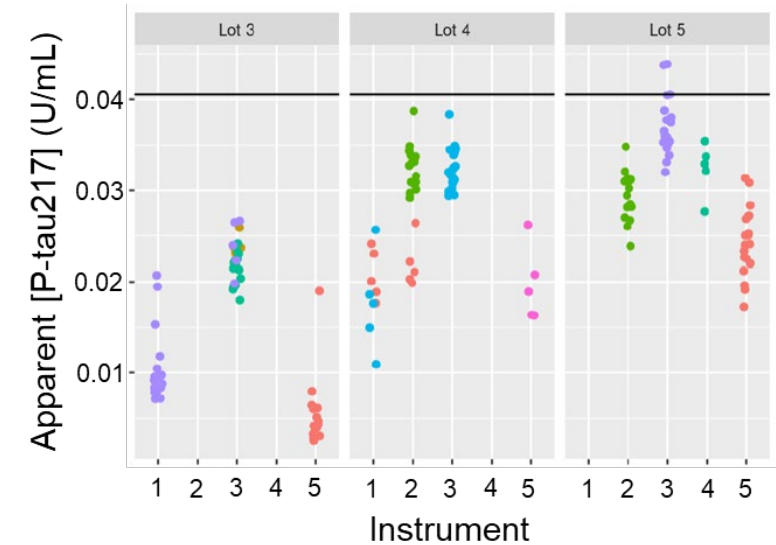


Table 3: LoB and LoD

Lot	LoB	LoD
Lot 3	0.025	0.059
Lot 4	0.035	0.065
Lot 5	0.041	0.062

Table 4: LLOQ All lots combined

Donor ID	Mean (U/mL)	SD	CV	N
1	0.076	0.014	18.4%	75
2	0.051	0.013	24.5%	75
3	0.089	0.017	18.7%	75
4	0.076	0.016	20.4%	75

INITIAL CLINICAL EVALUATION

Predicting amyloid PET positivity

- Concentration of P-tau217 in plasma samples from a subset of individuals screened in TRAILBLAZER-ALZ 2 were compared to florbetapir SUVR
- The AUC of the plasma samples assayed was 91.6% (95% CI 0.90, 0.94) for predicting PET positivity
- Plasma P-tau217 concentrations used for ROC analysis from patients with known PET status follow closely overlapping, log-normal distributions.
- Using two post-hoc chosen thresholds, the PPV was 94%, the NPV was 86%, with 18% of samples being between the upper and lower thresholds

Table 5: Demographics

Demographic	N=1124
Age, mean (SD)	72.87 (6.47)
Female, n (%)	602 (53.7)
MMSE, mean (SD)	24.66 (2.52)
APOE ε4 Carrier, n (%)	573 (51.3)
Amyloid Positive (≥1.1 SUVR), n (%)	702 (64.3)

Figure 5: ROC Curve


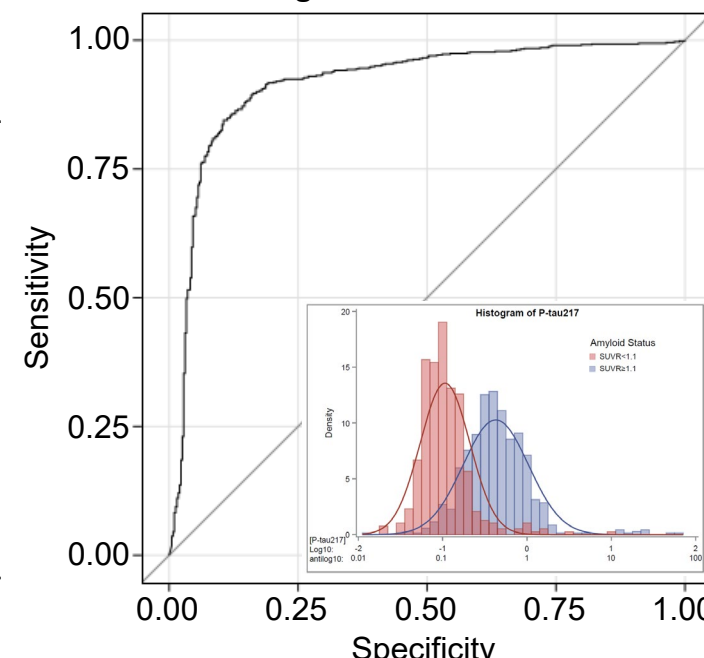


Figure 6: Florbetapir SUVR vs Plasma P-tau217 (15 data points outside y-axis limits)

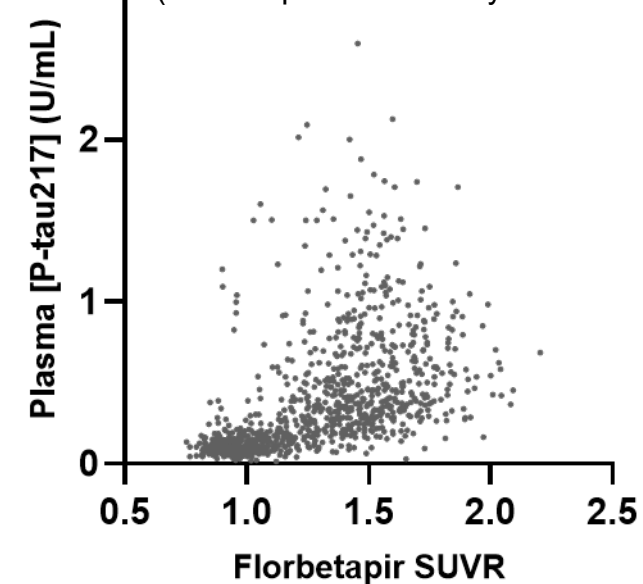

Acknowledgments: The authors would like to thank all the patients with AD, their families, and their caregivers who participated in this study, along with trial site investigators and personnel, and members of the data monitoring committee. We also thank Paul Faya for his contributions to the statistical analyses. Writing and BioRender assistance were provided by Staci Engle and Maanasa Chavani.

Disclosures: All authors are employees and minor stockholders of Eli Lilly and Company.

Abbreviations: AD=Alzheimer's disease; AUC=area under the curve; CI=confidence interval; CV=coefficient of variation; LLOQ=lower limit of quantitation; LoB=limit of blank; LoD=limit of detection; N=number of participants; NPV=negative predictive value; PET=positron emission tomography; PPV=positive predictive value; ROC=receiver operating characteristic; SD=standard deviation; SUVR=standardized uptake value ratio.

References: [1] Palmqvist S, et al. *EMBO Mol Med* 2019;11(12):e11170.

Copyright ©2023 Eli Lilly and Company. All rights reserved.