


Blood Track RN Steps

Scan ID badge

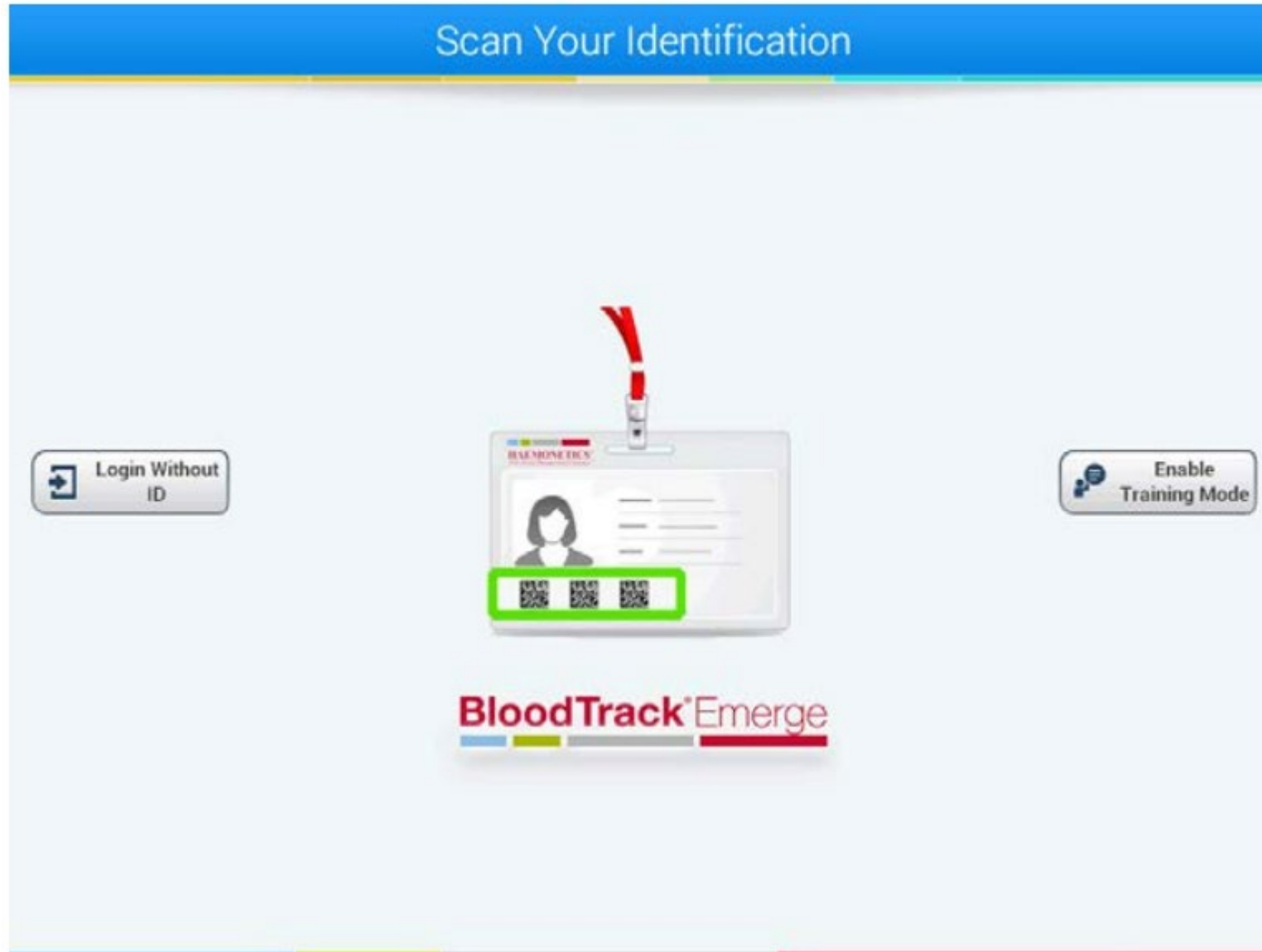

Select Taking Out



Select the type of product (Red Cells or FFP)

Select a Product

Red Cells

Fresh Frozen Plasma

Albumin (5%)

IVIG (G)

Octaplas

Cancel


Scan patient label

Enter Personal Health Number

Personal Health Number

Other Number

Search Cancel



The image shows a software interface for entering a Personal Health Number. At the top, a blue header contains the text "Enter Personal Health Number". Below this, there are two input fields: "Personal Health Number" and "Other Number". Underneath the input fields are two buttons: "Search" (with a magnifying glass icon) and "Cancel" (with a circle and slash icon). At the bottom of the screen, there is a virtual keyboard with a numeric row (0-9, -, =, and a delete key), a QWERTY row, an ASDF row, and a ZXCV row. The keyboard also includes arrow keys on the left and right sides. The interface is designed for a patient to scan a label and then enter their personal health number.

Confirm patient details

Correct Patient?

Personal Health Number: 532757

Other Number: 812 345 6786

Last Name: GANDLEY

First Name: Margaret

Birth Date: 17-Jul-1938


Gender: Female

Yes

No

4. For O Positive RBCS when prompted, select **Male** or **Female Older Than 55** from the options displayed:

Male | Female Older Than 55 | Female Younger Than 55 | Don't Know

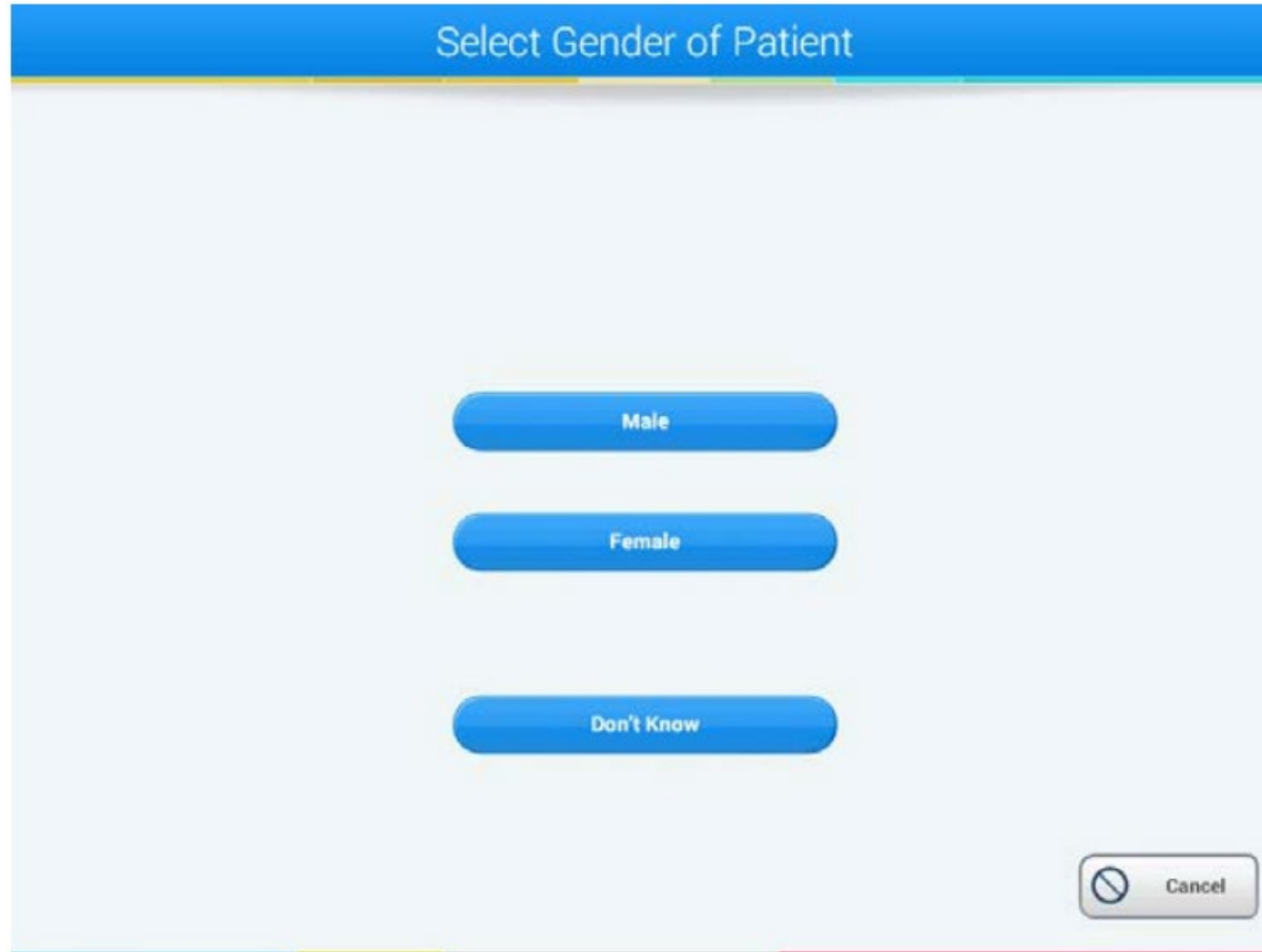

Select Gender of Patient

Male

Female


Don't Know

Cancel

Enter Number of Emergency Units to Dispense


Device compartment unlocks, a blue light will display in the compartment, signaling which compartment is unlocked

Remove Unit(s) from Tray (Tray(s) lights-up)

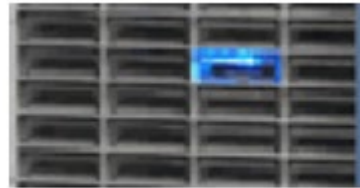

Be sure to scan the Unit Pool Number and the emergency tag *when obtaining products.*


Scan Unit Number



Patient
532757
(812 345 6786)
GANDLEY
Margaret
17-Jul-1938
Female
O Pos

Logout


Scan Emergency Tag



Please scan the unit number barcode followed by the emergency tag barcode.

You must scan the two barcodes within 10 seconds.

If these items are not scanned, the system will timeout, and an alert will be generated in the Blood Bank.

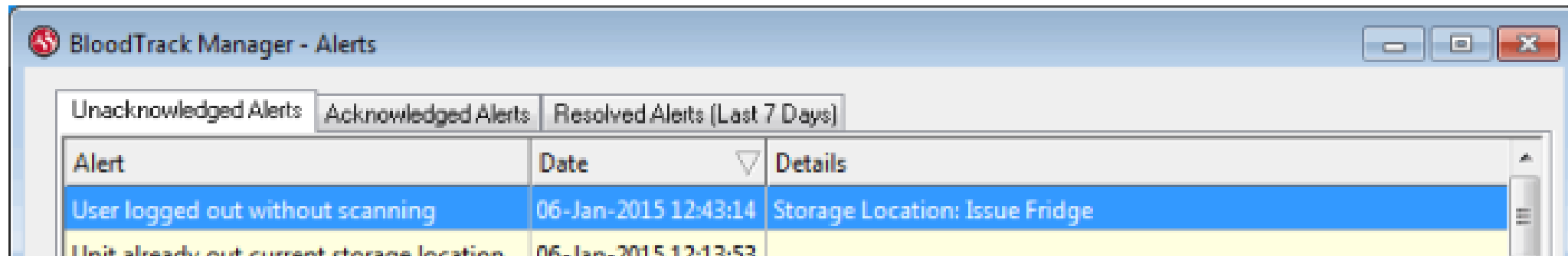


The screenshot shows a window titled "BloodTrack Manager - Alerts". It has three tabs: "Unacknowledged Alerts", "Acknowledged Alerts", and "Resolved Alerts (Last 7 Days)". The "Unacknowledged Alerts" tab is selected. Below the tabs is a table with three columns: "Alert", "Date", and "Details". The first row is highlighted in blue and contains the text "User logged out without scanning", "06-Jan-2015 12:43:14", and "Storage Location: Issue Fridge". The second row is highlighted in yellow and contains the text "Unit already out current storage location", "06-Jan-2015 12:13:53", and an empty "Details" cell.

Alert	Date	Details
User logged out without scanning	06-Jan-2015 12:43:14	Storage Location: Issue Fridge
Unit already out current storage location	06-Jan-2015 12:13:53	

MTP packs

Scan ID badge

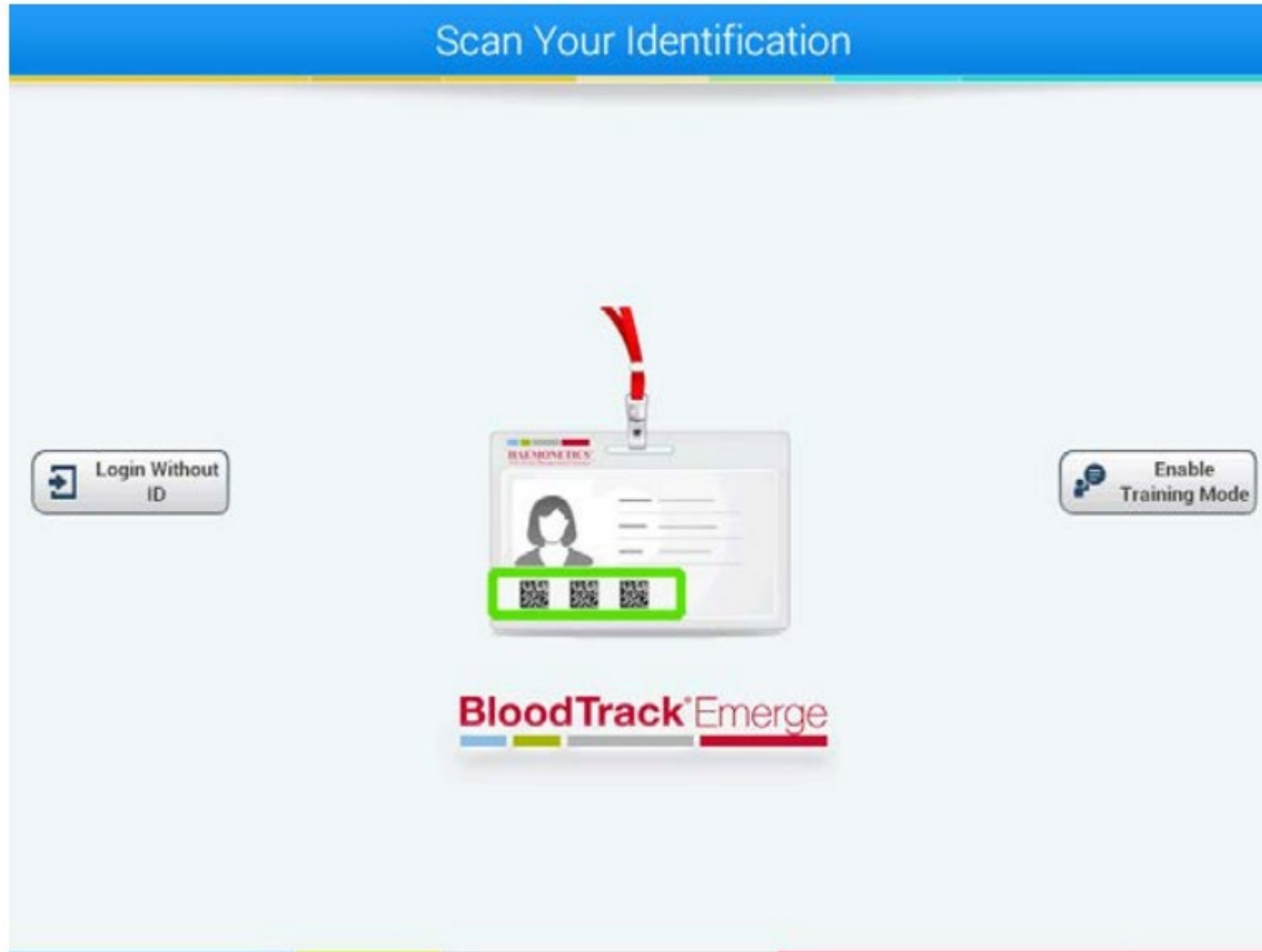

Select Touch Here for MTP Packs



Scan patient label

Enter Personal Health Number

Personal Health Number

Other Number

Search Cancel

· 1 2 3 4 5 6 7 8 9 0 - = ✕

q w e r t y u i o p [] \

a s d f g h j k l ; ' ←

↑ z x c v b n m , . / ↑

Confirm patient details

Correct Patient?

Personal Health Number: 532757

Other Number: 812 345 6786

Last Name: GANDLEY

First Name: Margaret

Birth Date: 17-Jul-1938

Gender: Female

Yes

No

Select the type of MTP Pack to be removed

Select a Massive Transfusion Pack

Peds Pkg1

ICU/OR Adult Pkg1

OR MTP Adult Pkg2

Cancel

Open the door to the storage location, remove the MTP Pack and close the door.
After removing the MTP Pack, the BloodTrack software will prompt to scan the MTP Pack Number



Scan the MTP Pack Number.

Once all units from a virtual MTP Pack have been removed, a message will display: You have successfully taken out all of the units for MTP pack Peds Pkg 1.

