


Single crown workflow with iTero Lumina™ scanner

These steps can be completed for a single crown procedure or multi-unit crowns, with the option to copy the details from crown to crown.

01 Select New Scan

Tap **New Scan** to start.

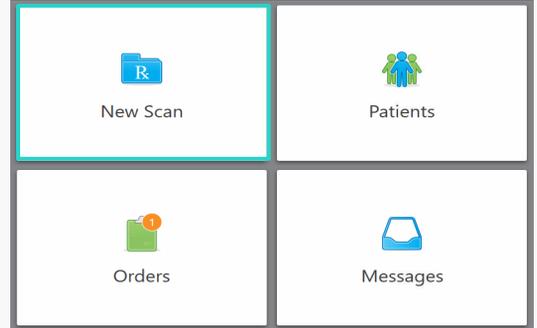

02 Complete the Rx

1 Fill in the patient's name, date of birth, and chart number.

A screenshot of the 'Patient' information form. At the top left is the label 'Patient:'. At the top right are a blue pencil icon and a 'Clear' button. The form contains several input fields: 'First Name: Test', 'Last Name: Test', 'Date of Birth:', 'Gender: Male Female' (with 'Female' selected), and 'Chart Number:'. Each field has a horizontal line for text entry.

2 In the procedure field drop-down, select **Fixed Restorative**. Then, select the partner lab or MyiTero.com account for fabrication. The due date is optional, however specify if you are sending it directly to the lab.

A screenshot of the 'Order' form. At the top left is the label 'Order:'. Below it is a 'Procedure *' dropdown menu with 'Fixed Restorative' selected. To the right is a 'Type' dropdown menu. Below the 'Procedure' dropdown is a 'Due Date:' field with a calendar icon. To the right of the 'Due Date' field is a 'Send to' dropdown menu.


3 Toggle the Pre-Treatment scan, if relevant.

A screenshot of the 'Scan Options' form. At the top left is the label 'Scan Options:'. Below it are four toggle switches: 'NIRI Capture:', 'New Sleeve Attached:', 'Pre-Treatment Scan:', and 'Multi-Bite:'. The 'Pre-Treatment Scan:' toggle is highlighted with a red box and is currently turned off.


Single crown workflow with iTero Lumina™ scanner

02 Complete the Rx - cont.


- 4 Tap the relevant tooth on the tooth diagram and select **Crown** from the drop-down menu.

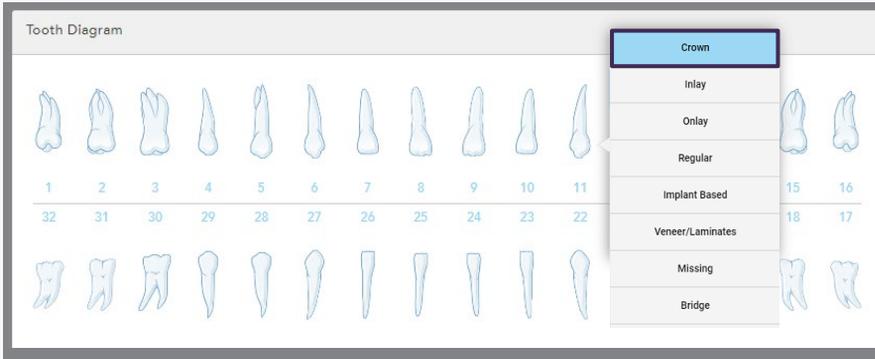

- 5 Select all the relevant information about the crown, including specification, material, shade system and additional information, if needed.

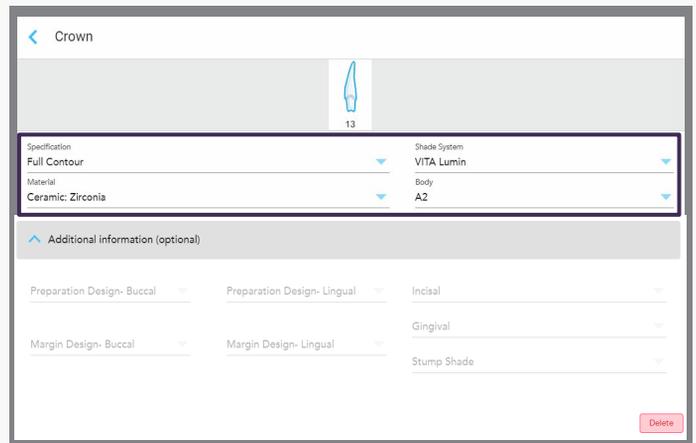

- 6 When scanning for multiple crowns, you can select **Copy from tooth XX** (select tooth number) to copy the information over to another tooth.

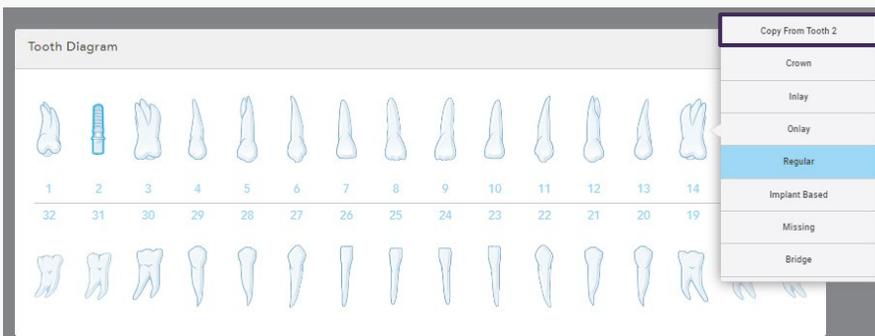


- 7 Upload the attachments from MyiTero.com, if needed. And add any additional notes relevant to your lab.

Single crown workflow with iTero Lumina™ scanner

03 Scan mode

If the scanning is performed **without** the pre-treatment scan, follow the steps below:

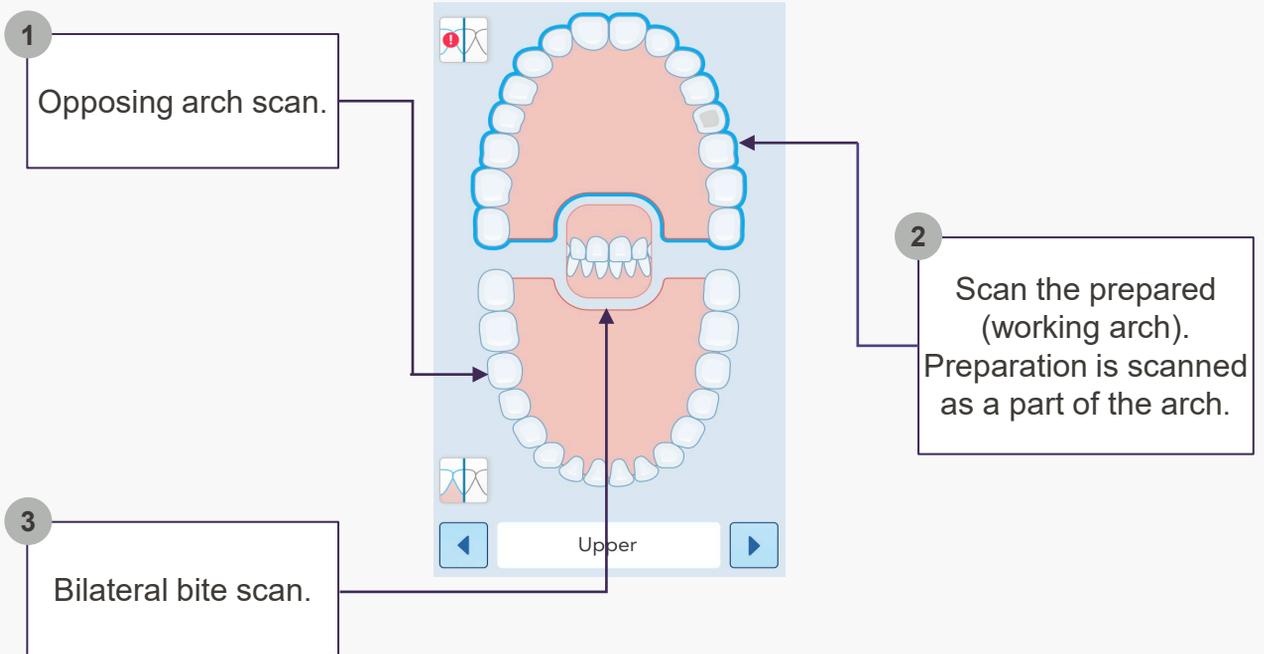

Scanning tips

- Scan occlusal, lingual and buccal of each arch.
- If the scanning is performed with the pre-treatment scan, it should be performed first. It will be marked as '1' on the tooth chart.
- Fill in all the dark blue on and around the prepped tooth while scanning.
- Use the prep editor to adjust the prep only if needed.

Single crown workflow with iTero Lumina™ scanner

Before moving to the view mode



Make sure you review the model in colour and monochrome mode and ensure that all the relevant surfaces were captured.



Use the prep editor to adjust the prep only if needed by following these steps:

1 Tap the prep editor icon  located above the viewfinder.

2 Tap Select  and circle the area to be rescanned.

3 Rescan the area and make sure the surface and the margin line have been scanned correctly..

04 View mode

Tap the view icon  in the upper part of the screen to move to the view mode.



Check the occlusal relationship with iTero™ Occlusogram to make sure the patient bites down all the way.

05 Send mode



Send the completed scan to the lab.

An active Lumina comprehensive service plan is required to access the restorative workflow on both the iTero Lumina and the iTero Lumina Pro scanner models. Ongoing fees apply.

