Cara Care für Reizdarm: Intended Use | v1

August 20, 2020

Deutsch

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- **Reizdarmsyndrom** (ICD-10-GM: K58, K58.1, K58.2, K58.3, K58.8, F45.32)
 - Behandlung/Therapie
 - Diagnosehilfe (aid in diagnosis)

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English version

Description

Cara Care is an interactive software-based medical application. It can be used as digital therapy or as an aid in diagnosis for irritable bowel syndrome (ICD-10-GM: K58.1, K58.2, K58.3, K58.8, F45.32).

The use of Cara Care is based on anamnestic and medically validated questionnaires that collect patient data initially and at regular intervals. These data are incorporated into an algorithm based on medical guidelines. The algorithm defines content that contains therapeutic recommendations. Those recommendations are individualized for each patient.

The content of Cara Care is organized in the form of modules that correspond to the therapeutic measures in compliance with the medical standard. Modules on nutrition, on audio-guided hypnosis, on the control of emotions, cognitions and behavior as well as on psychoeducation are based on the current recommendations regarding nutritional therapy, gut-directed hypnosis, and cognitive-behavioral therapy. Content is provided in the form of guided self-help interventions.

Furthermore, symptom severity and quality of life related to irritable bowel syndrome is assessed via validated questionnaires. The results can be presented to a health care practitioner as a report and can thus aid in diagnosis.

In addition, the patient can document parameters such as symptoms, medication, stool consistency, or lifestyle factors with the help of a tracking feature. The patient can use a visual representation and interpretation of these data to gain information on positive and negative factors influencing symptoms and quality of life.

Cara Care can be used on the recommendation of a physician or on the user's own initiative. The generated data can be exported by the user and presented to the relevant health care practitioners.

Cara Care does not make diagnostic decisions. Cara Care does not replace medical treatment by a physician. Consultation of a physician for the purpose of diagnosis and therapy is strongly recommended.

Indications

- Irritable bowel syndrome (ICD-10-GM: K58, K58.1, K58.2, K58.3, K58.8, F45.32)
 - Treatment
 - Aid in diagnosis

Contraindications

- Severe acute symptoms/acute abdomen due to (list not intended to be exhaustive):
 - Trauma
 - Cholecystitis
 - Cholangitis
 - Hepatitis
 - Liver rupture
 - Portal vein thrombosis
 - Congested liver
 - Budd-chiari-syndrome
 - Gastric or duodenal ulcer
 - Acute coronary syndrome
 - Myocardial infarction
 - Mallory-Weiss syndrome
 - Boerhaave syndrome
 - Pneumonia
 - Lung embolism
 - Pleuritis
 - Spleen infarct
 - Spleen rupture
 - Appendicitis
 - Incarcerated hernia
 - Retention of urine
 - Renal/bladder stones/renal colic
 - Urocystitis
 - Salpingitis
 - Adnexitis
 - Ovarial cyst rupture
 - Extrauterine pregnancy/tubal rupture
 - Testicular torsion
 - Sigma diverticulitis
 - Peritonitis
 - Pancreatitis
 - Pancreas rupture
 - Ileus
 - Acute gastroenteritis
 - Mesenteric infarction
 - Coprosthiasis
 - Aneurysm rupture
 - Aortic dissection
 - Pyelonephritis
 - Renal infarction
 - Perforation of a hollow organ
 - Acute porphyria
 - Sickle cell anemia
 - Cancer
- women in pregnancy
- persistent alcohol or drug abuse
- persons under the age of 18

Patient Population

Sex	Female, male, other
------------	---------------------

Weight	BMI >16 kg/m ² ; <40 kg/m ²
Race	any
Age	18-70 years
Health	See indications/contraindications
Patient state	Stable, conscious, able to operate, average health literacy