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.....  
Ort, Datum

.....  
Unterschrift (Betreiber/in)

## Procedure for regenerating the mixed-bed resin cartridge

---

Dear Madam / Sir,

please read the following information and steps before initiating the order for regenerating the mixed-bed resin cartridge. Failure to observe this information can result in the delay of the procedure.

### Order procedure

- From the time the order is received until the mixed-bed resin cartridge is returned, the order to regenerate the mixed-bed resin cartridge takes around **three weeks**.
- Once the order has been received, the mixed-bed resin cartridge should be collected within **five working days**. If this is not the case, please contact MELAG.
- The mixed-bed resin cartridge must be shipped in the MELAG transport packaging.
- For safe transport, please observe the instructions in "Storage and transport" in the MELAdem 53/MELAdem 53 C user manual.
- If you have requested a transport packaging, inform MELAG as soon as the packed mixed-bed resin cartridge is ready for collection. MELAG will then initiate the collection.
- The mixed-bed resin cartridge must be completely empty and closed tightly for transport, see "Preparing the mixed-bed resin cartridge for transport". Leaking liquid can damage the transport packaging and delay the transport.

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Department: Order processing Regeneration

Phone no. +49 30 757911 805233

### Order form

1. Fill in the order form on the following page **completely** and **legibly**.
2. Send the completed order form by fax or e-mail to your stockist/depot. **PLEASE NOTE:** Only the stockist can place an order with MELAG.

### Preparing the mixed-bed resin cartridge for transport

1. Unscrew the water connections of the mixed-bed resin cartridge after releasing the pressure.
2. **Empty the mixed-bed resin cartridge.**  
To do so, place the mixed-bed resin cartridge upside down in a small tray. Wait until no more water escapes.
3. Screw the yellow caps **firmly** onto the connection fittings.
4. Remove all labels from the transport packaging.
5. Pack the mixed-bed resin cartridge in the transport packaging.
6. Enclose a copy of the completed order form with the mixed-bed resin cartridge to be sent in.

**Order form for regeneration**

Name of the stockist/depot

Office (place) of the stockist/depot

Name of the practice

Contact person of the practice

Street address

E-mail

Postcode

Place

Phone number

Serial number MELAdem 53/53 C

Is the MELAG transport packaging available?

Yes

No - Please send transport packaging (at a price of € 30.00 plus VAT).

**PLEASE NOTE:** When regenerating for the first time you receive the transport packaging free of charge.

Opening hours of the practice (excl. breaks):

A precise indication of the availability in the practice is required for the collection and return delivery of the mixed-bed resin cartridge.

Monday

Tuesday

Wednesday

Thursday

Friday

Planned closing times/holidays within the next four weeks?

Other comments (e.g. collection/return delivery, defects etc.):

.....  
Place, date

.....  
Signature (operator)