

DEKODA

QUICK START GUIDE

Download the Kohler Health app

01 Download the Kohler Health app from the App Store or Google Play.




The Kohler Health app requires Bluetooth and WiFi connection to use with your Dekoda device.



02 Log in or create your Kohler Health account.

03 Follow the in-app instructions to set up and install your Dekoda device.

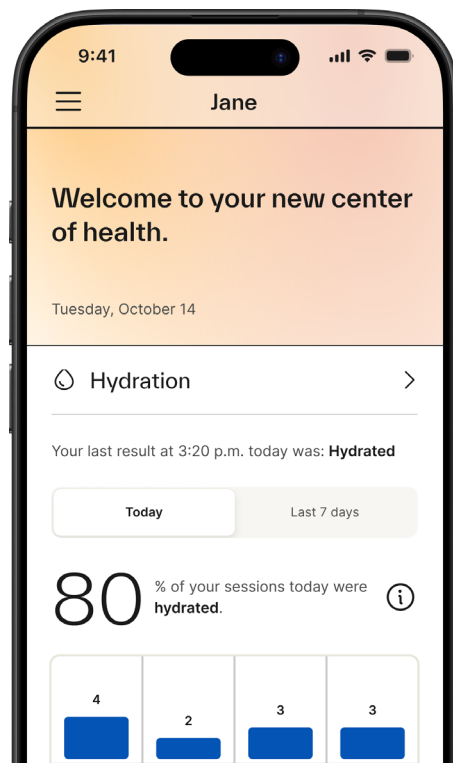

About your Battery

Your Dekoda comes with a partially charged battery. **Please fully charge the battery before starting the installation.**

If you have an outlet near your toilet, we recommend keeping Dekoda plugged in for the best experience.

Color	Solid Light	Blinking Light
● Green	Fully charged	Charging



Get to Know your Dekoda



Unsure about toilet fit?

Scan to view our compatibility guide. Email us at support@kohlerhealth.com for assistance.

Status light

Color	Solid Light	Blinking Light
○ White	Ready	Waking up
● Purple	Battery low	
● Blue		Pairing mode
● Red	Error	Wi-Fi disconnected
● Blue & Green		Firmware installing

Power button

Press to turn Dekoda on/off.

Proximity sensor

Automatically wakes the device on motion detection.

Adjustable clamp

Adjusts to fit your toilet.





Do not spray cleaners directly onto your device. Use only cleaning wipes for best results.

Battery

Can be charged while attached or detached from the device.

Battery status light

Color	Solid Light	Blinking Light
● Green	Fully charged	Charging

USB-C charging port

For use with the included power cable and charging adapter.



Sensors

Magnetic pad

Pivots for stability.
Remove only to clean.

Get to Know your Remote

Primary button


Confirm an action or scan your fingerprint to start a session.

Status light

Color	Status
○ White	Ready to scan
● Blue	Pairing mode
● Green	Fingerprint recognized
● Red	Can't read fingerprint

AA battery compartment

Slide to open.

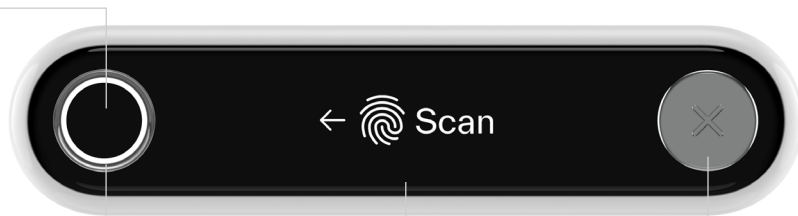

Screen

Follow on-screen messaging as needed.

Cancel button

Cancel any ongoing activity.



Included Accessories

Wall-mount remote holder

An optional remote holder that mounts on the wall near your toilet for convenient one-touch activation.



Charging cable

Power adapter

AA batteries for remote

Remote holder screws,
anchors and 3M™ tape



For more detailed information or help,
please scan the QR code to visit our support page.

KOHLER® Health