

My
personalized
plans, finished
in the moment.



Get initial ClinCheck® plans

56% faster*

with new ClinCheck® Signature.

- Minimize back and forth with CAD designers
- Maximise consistency and control
- Get more time back for what matters most

ClinCheck® Signature Quick Start Guide


A collection of our latest treatment planning innovations that helps you confidently deliver personalized, consistent treatment plans in the moment, giving you the time to do what matters most.

Suite of treatment planning innovations

- Flex Rx submission form
- ClinCheck® Pro treatment planning software
- Plan Editor
- 3D controls
- ClinCheck® Live Update

*-Based on Invisalign comprehensive treatment option primary orders from doctors who were activated with a treatment planning preferences template. Total orders analyzed is 487,926. The comparison is between orders submitted using a treatment planning preferences template and orders submitted without. Time is measured from when an order is received until the ClinCheck plan is available for doctor review. The time taken for orders submitted using a treatment planning preferences template is 56% less. The median value was used. Orders were from the period April 2022 through May 2023. Includes Invisalign trained orthodontists and dentists in the NA, LATAM, EMEA, and APAC regions. Data on file with Align Technology, Inc.

ClinCheck® Treatment Planning *hierarchy*

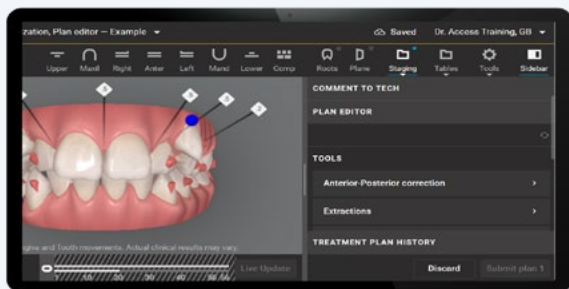

3D Controls

with ClinCheck® Live Update

Plan Editor

with ClinCheck Live Update

Flex Rx submission form

Global Clinical Preference template

Can be customised
with Case-specific
clinical preferences

Can be
customised with
Template Editor

Case Submission *with* Flex Rx

04 Select treatment option

Patient list Patient file

Patient type: Adult Product type: Invisalign clear aligners Treatment options

Select treatment option

Comprehensive Package	Moderate Package	Lite Package
Unlimited stages Selectable additional aligner options Flex Rx is supported dual arch 5 year(s), unlimited additional aligners included £1810 3 year(s), 3 set(s) of additional aligners included £1710 Select	Up to 20 stages 2 set(s) of additional aligners included dual arch single arch Select	Up to 14 stages 2 set(s) of additional aligners included Select

“Flex Rx is supported”
will appear in
supported treatment
options with Flex Rx.

05 Complete Flex Rx

Welcome to Flex Rx

Show me around

Some cases not supported by Flex Rx
Some cases may require submission with the Traditional Rx
Switch to Traditional Rx

Uploaded Records

None

Private Notes

6/10000

Write private notes here (only visible to you).

Private notes are only visible to you and will be displayed in the Notes section of the ClinCheck® software

Preferences

Global Clinical Preferen... View Edit for all cases

Case specific clinical preferences (optional)

No case specific clinical preferences added

Arch to treat

Both

Restrictions, extractions, and missing teeth

Tooth movement restrictions: none
Attachments restrictions: none
Extractions: none
Missing teeth: none

Eruption compensation

Eruption compensation: none
Molar tab: none

Flex Rx Case-specific preferences option

- Personalise for patient specific needs
- Manage exceptions to your preferences template
- Manage missing teeth
- Schedule IPR
- Manage crowding

invisalign®

Patients Account Store Education Support Marketing Site

Welcome to Flex Rx

Show me around

Some cases not supported by Flex Rx
Some cases may require submission with the Traditional Rx
Switch to Traditional Rx

Uploaded Records

None

Private Notes

6/10000

Write private notes here (only visible to you).

Private notes are only visible to you and will be displayed in the Notes section of the ClinCheck® software

Preferences

test View

Case specific clinical preferences (optional)

IPR x Collapse

IPR

Follow chosen preferences template

Click here to view your predefined clinical preferences for "IPR"

Select segments where IPR is allowed

Right Left

Limit anterior IPR per contact

If unchecked, maximum limit of 2.5 mm IPR will be applied

Limit posterior IPR per contact

If unchecked, maximum limit of 2.5 mm IPR will be applied

IPR timing (rounding)

Perform alignment after IPR

Perform alignment before IPR

Delay IPR

Schedule IPR

Based on access to contacts

Every (certain number) stages

At specific stages

Arch to treat

Both

Restrictions, extractions, and missing teeth



Global Clinical Preferences *template* (Global template)

This template contains the Align default preferences to be applied to your cases if you do not have any defined clinical preferences.



- The Global Clinical Preferences template was created based on treatment results from years of experience in successfully preparing **millions of ClinCheck® treatment plans** for Invisalign patients.
- The clinical preferences from the Global Clinical Preferences template are shaped by a **multifaceted approach that encompasses** a combination of factors, including **data analysis, established protocols, feedback from customers (including faculty)** and input from internal Align teams across the globe.
- Modification to these preferences can be done through Template editor.

Clinical preferences page


Clinical preferences page consists of three options:

- **Flex Rx Template**
- **Traditional**
- **General settings**

The screenshot displays the 'Clinical Preferences' interface. At the top, there are three tabs: 'Preferences for Flex Rx', 'Preferences for Traditional Rx', and 'General Settings'. The 'Flex Rx Protocol' tab is active, showing a 'Welcome to your new clinical preferences' message. Below this, there are sections for 'Custom and reference templates' and 'Flex Rx Protocol'. On the left, a sidebar lists various preference categories: General, Deep bite, Anterior open bite, Posterior cross bite, Crowding, and Place it. The 'Global clinical preferences' section is expanded, showing settings for 'Passive Aligners', 'IPR on first ClinCheck treatment plan', 'Delay stage to start IPR', 'Delay attachment placement', 'Delay stage of extraction', and 'Pontics for open spaces'. The 'Tooth numbering system' section is also visible, with options for Universal, Palmer, and FDI systems. The 'Aligner trimming' section includes options for 'Trim aligners to 1/2 way between gingival margin and CEJ line' and 'Trim aligners to CEJ line'. The 'Terminal molar distortion' section provides instructions on how to handle distorted terminal molars.

how to use the Template Editor


- All options in the Template editor are pre-filled based on the preferences in the Global Clinical Preferences template
- When first launched, all options will be set **using the current traditional clinical preferences from IDS and combined with Align defaults preferences** in the Global Clinical Preferences template
- These preferences can be adjusted at any time, and will be reflected in the Global Clinical Preferences template instantaneously
- The updated Global Clinical Preferences template will be used in all cases submitted with Flex Rx. It does not impact cases already submitted with Flex Rx

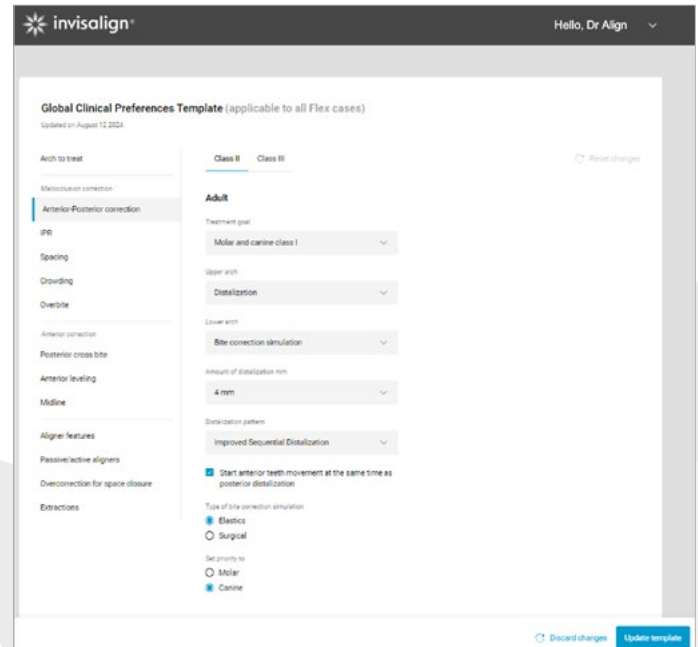

Plan Editor

New ways to:

- Personalize for patient specific needs.
- Handle exceptions to IPP
- Schedule IPR
- Adjust AP approach or goal
- Manage spacing
- Simulate extraction
- Plus more

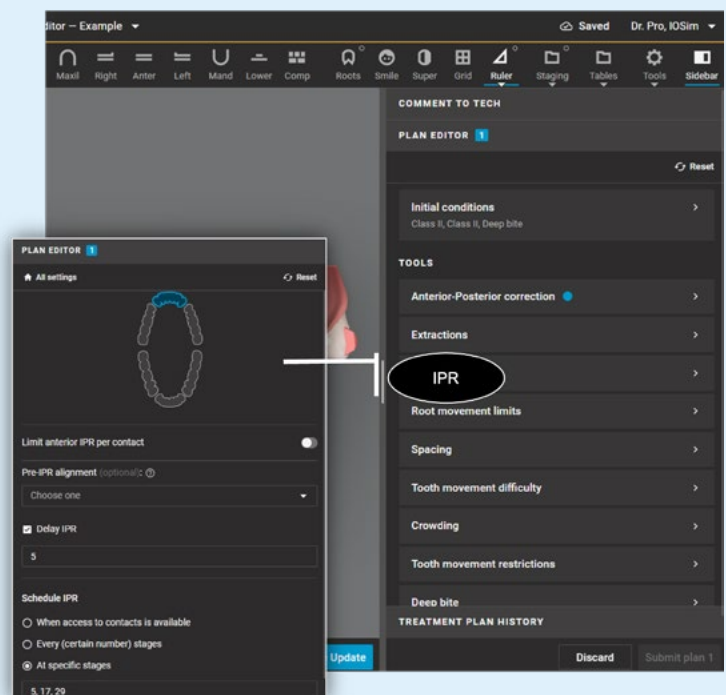

A suite of Invisalign® treatment planning tools built from 25+ years of innovation and 20 million smiles.

Consistent plans you can trust.

- ✓ Invisalign® Personalized Plan
 - Quick start with the Global template.
 - Personalize your plan as desired.

Plans are sent back to CAD designer less than ½ the time¹

Customization when you need it.

- ✓ Case-specific preferences in Flex Rx
- ✓ ClinCheck® Pro visualization software
- ✓ Plan Editor
- ✓ 3D controls

Doctors approved their cases 82% faster²

1. Based on Invisalign comprehensive treatment option primary orders from doctors who used Invisalign® Personalized Plan in at least 50% of their comprehensive orders. Total orders analyzed is 10,992. Modifications are measured as the number of times a treatment plan was sent back to a CAD designer before plan approval. The comparison is between the aggregate of orders with Invisalign® Personalized Plan and ClinCheck Live Update and the aggregate of orders without Invisalign® Personalized Plan and ClinCheck Live Update. The modification rate is 57% less for the aggregate of orders with Invisalign® Personalized Plan and ClinCheck Live Update. Orders were approved from November 2022 through April 2023.

2. Based on Invisalign comprehensive treatment option primary orders (n=50,114 cases) from doctors (n=165 orthodontists and dentists) participating in a limited market release who only used ClinCheck Live Update for 3D controls as compared to those who only used CAD designers for their ClinCheck modifications to treatment plans in the period June 1, 2021 through December 23, 2021. Time measurement is based on the doctor's first opening of the ClinCheck plan to final ClinCheck approval.

➤ Learn more at learn.aligntech.com



Align Technology, Inc.
2820 Orchard Parkway, San Jose, CA USA 95134



Align Technology B.V., Herikerbergweg 312,
1101CT, Amsterdam, The Netherlands



0344