Moje

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Patient type: Adult Product type: Invisalign clear aligners Treatment options

Select treatment option

Comprehensive Package	Moderate Package	Lite Package
Unlimited stages Selectable additional aligner options Flex Rx is supported	Up to 20 stages 2 set(s) of additional aligners included	Up to 14 stages 2 set(s) of additional aligners included
dual arch 5 year(s), unlimited additional aligners included 3 year(s), 3 set(s) of additional aligners included £1810 £1710	dual arch single arch	
Select		

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Welcome to Flex Rx

Show me around

Some cases not supported by Flex Rx. Some cases may require submission with the Traditional Rx. [Switch to Traditional Rx](#)

Preferences Global Clinical Preferen... View Edit for all cases

Case specific clinical preferences (optional) No case specific clinical preferences added

Arch to treat Both

Restrictions, extractions, and missing teeth
Tooth movement restrictions: none
Attachments restrictions: none
Extractions: none
Missing teeth: none

Eruption compensation
Eruption compensation: none
Molar tab: none

Uploaded Records 0/10000

Private Notes 0/10000
Write private notes here (only visible to you).
Private notes are only visible to you and will be displayed in the Notes section of the ClinCheck® software

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invisalign®

Patients Account Store Education Support Marketing Site

Welcome to Flex Rx

Show me around

Some cases not supported by Flex Rx. Some cases may require submission with the Traditional Rx. [Switch to Traditional Rx](#)

Preferences test View

Case specific clinical preferences (optional) IPR x

IPR

Follow chosen preferences template
Click here to view your predefined clinical preferences for "IPR"

Select segments where IPR is allowed

Right Left

Limit anterior IPR per contact
If unchecked, maximum limit of 0.5 mm IPR will be applied

Limit posterior IPR per contact
If unchecked, maximum limit of 0.5 mm IPR will be applied

IPR timing (roundtripping)
Perform alignment after IPR
Perform alignment before IPR
Delay IPR

Schedule IPR
Based on access to contacts
Every (certain number) stages
At specific stages

Arch to treat Both

Restrictions, extractions, and missing teeth

Uploaded Records 0/10000

Private Notes 0/10000
Write private notes here (only visible to you).
Private notes are only visible to you and will be displayed in the Notes section of the ClinCheck® software

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Align Technology, Inc.
Align Technology B.V., Herikerbergweg 312,