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Share Case

Enter treatment goal

Share your treatment objectives and patient's chief concern for effective treatment planning

Overall treatment goal

Describe patient and your treatment goals including any restorative plan if it's applicable

Patient's chief concerns

☒ Crowding

☐ Spacing

☐ Open bite

☐ Deep bite

☒ Overjet

☐ Crossbite

☐ Underbite

☐ Misshapen teeth

☐ Misaligned teeth

☐ Narrow arch

☒ Other

Enter comments \*

Clinical considerations

☐ RCT

☐ Bone resorption

☐ Short roots

☐ Gingival recession

☐ Extractions

☒ Other

Enter comments \*

Treatment plan limitations

☐ Extractions

☐ IPR

☐ TADs

☐ Elastics

☐ Other

☒ I confirm that the patient whose data I am submitting, or the patient's legal guardian if applicable, has given valid consent for this data to be shared with and processed by Align and with third parties involved in the development of the patient's Invisalign treatment.

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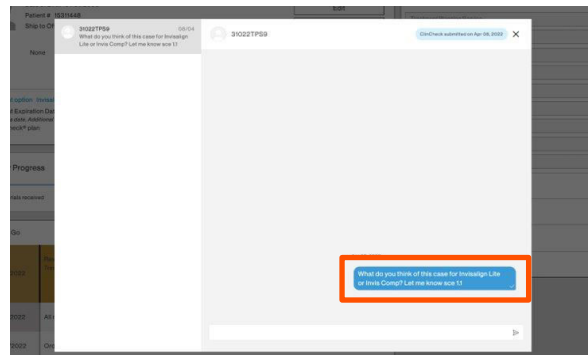
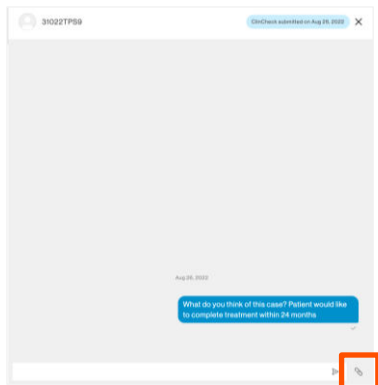
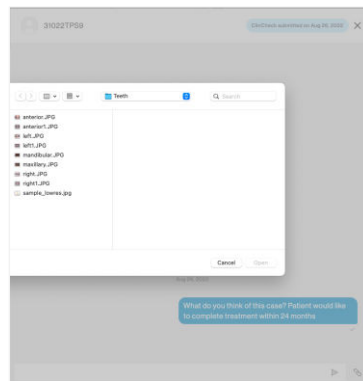
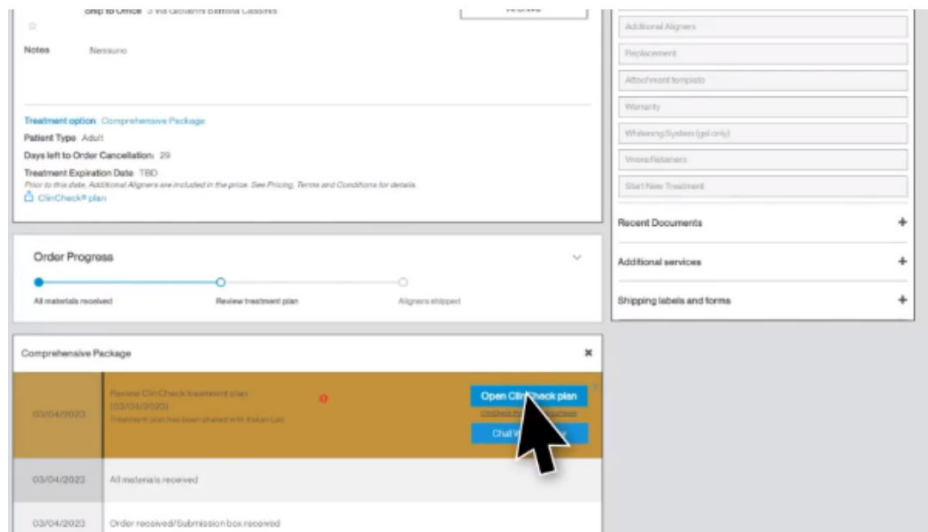
**Are you sure you want to cancel review of the case?** ✕

Your case is under Provider review. Canceling the review will be counted towards the usage of the service.

Please provide reason for cancellation:

- ☐ Needs finishing improvements
- ☐ Teeth are not tracking
- ☐ Patient has new restoration or dental work
- ☐ Patient was not complaint
- ☐ Treatment plan change (specify in treatment instructions)
- ☒ Other

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invisalign® Hello, Dr. Bom (impersonated by mbattini)

Patients Account Store Education **Subscription Program** Support TPS Virtual Care

Order history

Treatment planning services

Dec 2024 - Dec 2024 Status: ▾

| Assignment date ▾ | Provider name ▾ | Package or Service ▾ | Amount Paid ▾ | Payment status ▾ | Receipt |
|-------------------|-----------------|----------------------|---------------|------------------|---------|
| Dec 16, 2024      | New UK TPS Test | Case Assessment      | USD 0.00      | Program Pack     |         |
| Dec 6, 2024       | New UK TPS Test | Case Assessment      | USD 0.00      | Program Pack     |         |

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© Align Technology, Inc. All rights reserved. [Privacy](#), [Terms of Use](#), [Software as a Service](#)