

Specimen Instructions

Extracted Nucleic Acid

NUCLEIC ACID TYPE	SUBMISSION FORMAT	CONCENTRATION*	VOLUME	SHIPPING INSTRUCTIONS
DNA	Nuclease-free water	Picogreen: ≥ 3.5 ng/ μ l UV: ≥ 10 ng/ μ l	≥ 60 μ l	Domestic: ship overnight, ambient International: ship overnight, frozen on dry ice
RNA	Nuclease-free water	Ribogreen: ≥ 20 ng/ μ l	≥ 30 μ l	Domestic/International: ship overnight, frozen on dry ice

* Please specify concentration on requisition form.

Additional Submission Requirements

Please submit concurrent or recent laboratory test results (e.g., CBC / differential, flow cytometry results, final bone marrow pathology report) if available (these documents may be faxed after specimen shipment; fax to 617.418.2290).

Shipping Instructions

1. Ensure that all samples are labeled with two patient-specific identifiers (e.g. patient name and DOB).
2. Place the specimens, Test Requisition Form, insurance information and any other attachments into your shipping pack.
3. Call 800.463.3339 to request a pick-up or drop the package at your site's designated FedEx pick-up location and ship sealed shipping pack to:

Foundation Medicine, Inc.
7010 Kit Creek Road
Morrisville, NC 27560
Phone: 888.988.3639

(Note: Saturday delivery, when appropriate, must be specifically marked on the airbill in order to ensure proper handling.)

INTERNATIONAL ONLY: Make three copies of the commercial invoice for FedEx courier and include with air waybill in FedEx documents pouch.

FoundationOne® Heme is a laboratory developed test that was developed and its performance characteristics determined by Foundation Medicine. FoundationOne Heme has not been cleared or approved by the U.S. Food and Drug Administration. For more information on our laboratory developed tests (LDTs) please see their respective Technical Specifications at <http://www.foundationmedicine.com>.