Ultradent Products Lumaloc™ EN

Indirect Bonding Outer Tray Material Description and Indications:

Ultradent Products Lumaloc is a vinyl polysiloxane outer tray matrix for bracket transfer used in indirect bonding.

Lumaloc is not to be used for conventional impressions due to the final hardness.

Instructions for Use:

1. Use Lumaloc at room temperature.
2. Place Lumaloc cartridge in a 1:1 / 2:1 Impression Dispensing Gun.

3. Remove turquoise cartridge cap from dual barrel syringe by twisting counter clockwise and discard.

NOTE: Prior to attaching tip, verify uniform flow from both syringe orifices by expressing a small amount onto a mixing pad. Use caution not to cross-contaminate orifices.

4. Attach the turquoise mixing tip by lining up internal cap stems with syringe orifices. Twist clockwise to lock and verify flow.

5. Place desired amount according to the procedure where Lumaloc is being used. Working time at 23°C/73°F including mix – 30 seconds

Intra-oral working time at 35°C/95°F including mix – 20 seconds

6. Mixing tip acts as the cap. Do not remove from syringe until next use.

Impression Disinfection:

Lumaloc can be sprayed or soaked in a solution of 5.25% sodium hypochlorite for up to 10 minutes without distortion or damage to the impression or surface. See manufacturer's instructions when using other disinfectants.

Precautions and warnings:

1. Carefully read and understand all instruction before use.

2. Mixing tip is for single use ONLY.

3. Use only recommended tip.

4. Store at room temperature.

5. If product comes in contact with eyes, rinse with copious amounts of water for several minutes. If symptoms persist, consult an eye specialist.

6. If product is inhaled or ingested, seek medical attention immediately.

7. If allergic reaction, dermatitis or rash develops consult a physician.

8. Dispose of tips and empty syringes properly.

9. When using this product, do not remove mixing tip until next use.

10. Disinfect syringe between uses if contaminated with saliva or used intraorally.

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U.S. Federal law restricts the sale of this device by or on the order of a dentist.

Keep out of reach of children.

For immediate reorder and/or complete descriptions of Ultradent's product line, refer to Ultradent's catalog or call Toll Free 1-800-552-5512. Outside U.S., call (801) 572-4200 or visit www.ultradent.com.

Ultradent Products Lumaloc™

Manufactured for:
Ultradent Products Inc.
505 West Ultradent Drive (10200 South)
Salt Jordan, Utah 84095 USA

ULTRADENT
PRODUCTS, INC.

© Copyright 2017 Ultradent Products, Inc. All Rights Reserved.
190324 013118

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